

Rinse Aid®

Item No. 34563

Rinse Aid for Machine Washing

Specifically Designed for use in all Automatic Washers, Washer Sterilizers, Cart Washers and Tunnel Washers.

Prevents Spotting on Surfaces of all Surgical Instruments and Glassware.

Rinse Aid® prevents adhesion of scale and mineral deposits on the surface of all surgical instruments and glassware. Rinse Aid® contains powerful wetting agents that lower the surface tension on instruments during the rinse cycle, allowing water to drain quickly and completely. Rinse Aid® prevents spotting caused by hard-water, leaves no residue and assists in drying, leaving instruments and glassware sparkling clean.

Features

- Designed for all automatic washers
- Prevents adhesion of mineral deposits
- Prevents spotting
- Assists in drying
- Leaves no residue

Directions for Use

For Automatic Cleaning Devices

1. Calibrate the washer dilution at 1 - 2 ounces per one (1) gallon (8-16ml/liter) of water depending on water quality and application.

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.

