

## Ruhof ENDO-BAG®

Item No. 345EDB

### *Cleaning Bag for Small Removable Scope Parts*

The Endo-Bag is perfect for keeping scope buttons and valves with the scope in the washer disinfector during the cleaning and disinfection process.

Perfect for use in Endoscopy Suites, Operating Rooms, Sterile Services, Outpatient Surgery, and clinics where small items require cleaning in an automated process.



### Features

- Keeps the scope and its accessories together during the cleaning process.
- Protects the accessories during the cleaning process
- Assists in the traceability of the scope
- Can be used for any small item in any type of washer disinfector
- Designed to withstand High Level Disinfectants

### Specifications:

- 6" width x 5" height
- Durable soft mesh nylon material
- Drawstring closure

### Directions for Use

1. Place the small items for cleaning in open bag.
2. Holding the cord, gather the top of the bag to close the top.
3. Press the button on the toggle and pull it along to the closed bag to hold it in place.
4. Place the loop of the cord formed by closing the bag around the control knobs of the scope.

**Please Note:** This product has a life span and will require replacing if there is any sign of deterioration of the fibers of the bag.