

Surgislip®

Item No. 34571

Concentrated Instrument Lubricant and Rust Inhibitor For Use In All Washers

The only concentrated silicone-free instrument lubricant and rust inhibitor clinically tested to be thoroughly steam penetrable, steam sterilizable and ETO sterilizable.



Surgislip® is a non-oily, non-sticky, non-toxic, water soluble mineral oil lubricant that prevents rusting, staining, spotting and tightness of surgical instruments. Designed for use in all washers, Surgislip® forms a protective coating on surgical instruments, keeping them in excellent working condition and restoring articulation to box-locks and joints. Used routinely as part of the Ruhof System, Surgislip® will greatly reduce repair and replacement costs.

Features

- Perfect for use in all washers
- Prevents rust, spots, stains and corrosion, prolonging the life of instrument
- Inhibits damage due to stiff or grinding instruments, reducing repair and replacement cost
- Restores articulation in box-locks and joints
- High-grade mineral oil lubricant and rust inhibitor
- Steam penetrable, steam sterilizable and ETO sterilizable
- Non-oily, non-sticky, non-toxic and silicone free

Directions for Use

Manual Soaking

Surgislip® must be used before sterilization. Add 13 ounces (384 ml) of Surgislip® to one (1) gallon (4 liters) of hot water (50°-80°C or 122°-176°F) into Ruhof Instru-Bin®, basin or sink. Dip instrument (DO NOT RINSE), wrap and sterilize.

Automatic Cleaning Devices

Surgislip® should be used routinely on all surgical instruments following washer manufacturer guidelines at a dilution of 1-2 ounces of Surgislip® per gallon of warm water (1-2ml/liter).

Discard used solution

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.



