



andorate®

SUCTION CHANNEL CLEANING ADAPTER

*Instructions for Use.
Read carefully prior to use.*

 Manufactured for Ruhof Corporation
393 Sagamore Avenue
Mineola, NY 11501
1-800-537-8463
www.ruhof.com

**Not Made with natural rubber latex.
Does not contain DEHP**

<u>Part Number</u>	<u>Description</u>
GAR075	Suction Channel Cleaning Adapter (Non-sterile)

For Olympus® GI Endoscope

INTENDED USE

The device is intended to aspirate reprocessing fluids from the distal end of the endoscope through the instrument channel.

WARNINGS AND PRECAUTIONS

- This device is intended to be used with Olympus® GI Endoscopes.
- Do Not Use the **ANDORATE® Suction Channel Cleaning Adapter** during patient procedures. Insufficient aspiration and leakage cause cross-contamination.
- Endoscopic cleaning procedures should only be performed by persons having adequate training with endoscope cleaning techniques.
- Do not attempt to reuse, reprocess, refurbish, remanufacture or resterilize this device. GA Health Company Limited did not design this device, nor is it intended to be reused, reprocessed, refurbished, remanufactured, or resterilized. Performing such activities on this disposable medical device presents a safety risk to patients (i.e. compromised device integrity, cross-contamination, infection).

DIRECTION FOR USE

1. Prior to clinical use, familiarize yourself with the device and read all the Instructions for Use.
2. Inspect the package for shipping or handling damage. If damage is evident, **DO NOT USE THIS DEVICE, SAVE THEM OR RETURN, AND CONTACT YOUR LOCAL PRODUCT SPECIALIST**
3. Remove the device from the package.
4. Attach the **ANDORATE® Suction Channel Cleaning Adapter** to the instrument channel port (Figure 1).
5. Connect the suction tube from the suction pump to the suction port of the endoscope. Turn on the suction pump as per the Olympus Reprocessing Manual.
6. Immerse both the endoscope's distal end and the weighted end of the suction channel cleaning adapter in the detergent solution (Figure 2).
7. Cover the suction cylinder with the finger and aspirate detergent solution for approximately 30 seconds.
8. Release the finger from the suction cylinder.
9. Turn the suction pump OFF.
10. Disconnect the suction tube and the suction channel cleaning adapter.

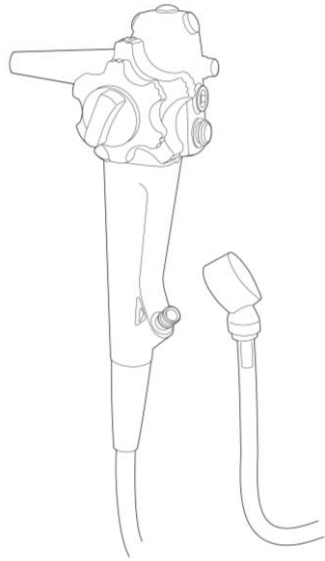


Figure 1 – Channel Port

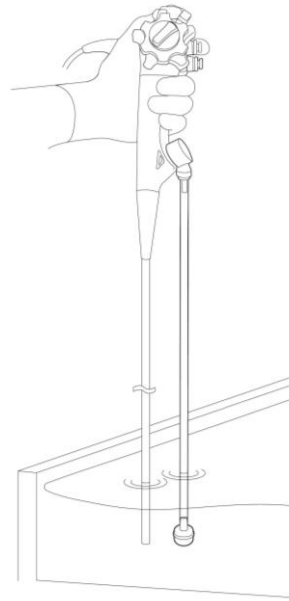


Figure 2 – Cleaning of Channel Port

PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of the product in accordance with accepted medical practice and applicable local, state and federal laws and regulations.
















WARNING

An issued or revision date for these instructions is included for the user’s information. In the event that two years have elapsed between this date and product use, the user should contact GA Health Company Limited to determine if additional information is available. Report any serious incident that has occurred in relation to the device to the distributor or manufacturer or competent authority of the member state.

ANDORATE® is a registered trademark of GA Health Company Limited.

Olympus® is a registered trademark of Olympus Corporation.

EXPLANATION OF SYMBOLS USED ON LABELS AND INSTRUCTIONS FOR USE

	Use-by date
	Batch or lot code
	Manufacturer
	Date of Manufacture
	Medical Device
	Catalogue number Product Code
	EC Conformity Quality Assurance
	Do not re-use Single use
	Product not made with natural rubber latex
	Do not contain DEHP
	Consult Instructions for Use
	Authorized representative in the European Community
	Keep dry
	Keep away from sunlight
	Federal law (U.S.A.) restricts this device to sale, distribution and use by or on the order of a physician.