

# ScopeValet™

## Endoscopy Care Products

### ScopeValet™ Transport Bag

Item No. 345SVBT - Transparent  
Item No. 345SVB - Red

The ScopeValet™ TRANSPORT BAG is a safe and easy way to transport an endoscope from the procedure room to the reprocessing room after the endoscopy. The drawstring closure ensures staff safety by securely keeping the soiled scope enclosed during transfer.



Available in two color choices: **TRANSPARENT** (for improved visibility of contents) or **RED**.

#### Features


- Easy set up before procedures providing full surface coverage for large and small cart surfaces. There is no need to use Chux pads or towels
- A strong leak-proof liner and absorbent inner pad which keep bio-hazard fluids contained and protects your endoscope from being damaged during transportation
- 40" x 33" (104 cm x 84 cm) bag size with 16" (41 cm) wide pad fits perfectly over a standard endoscopy cart
- Ample bag size accommodates all endoscopes and keeps them safe from being impaired when the flexible shafts are coiled too tightly
- Easy-to-use cinching cords which, when pulled, convert the surface pad into your enclosed transport container eliminating the need for transport buckets, pillow cases and giant zip-lock bags
- Full compliance with infection control guidelines for endoscope transportation
- Biohazard tag warn staff of possible infectious waste exposure
- A "Used Instrument" tag informs personnel of the valuable medical device which is enclosed
- Available in two color choices: **TRANSPARENT** - offering clear identification of the enclosed endoscope - or **RED**.
- Biodegradable; Disposable; One time use



Tag informs personnel of the valuable medical device which is enclosed.

#### Instructions for Use

1. Remove packaging and spread the ScopeValet™ Transport Bag across the procedure cart surface, placing the soiled scope in the center of the pad.
2. Take hold of the drawstring closures at each end of the pad bringing them together into one hand over the center of the pad.
3. While lifting the scope and pad off of the cart surface pull up on the cords with one hand and down along the cords with the other hand to easily cinch the pad into a transport bag.
4. Simply release cinched cords and draw open the Transport Bag to remove used scope for reprocessing.

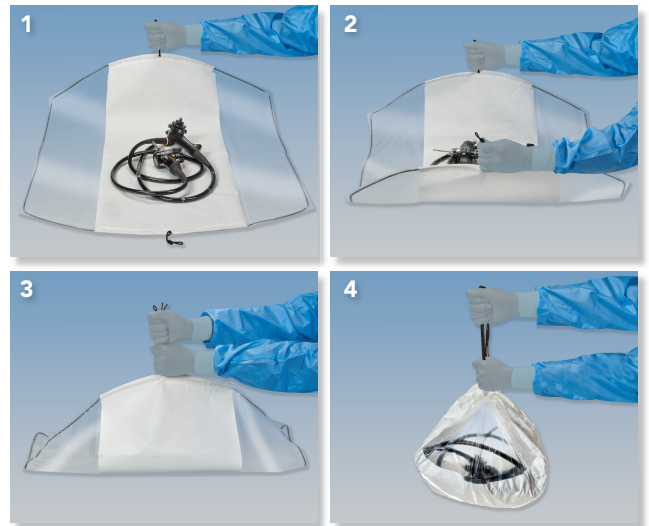

#### Contraindication:

This product is for one time use only. Its reuse may present risks to the safety of individuals, due to possible uncontrolled contamination.