## APPLYING AND REMOVING YOUR CONTACTS

### Applying step by step

- Thoroughly wash your hands with soap and dry using a clean towel.
- Place the lens on the tip of your right index finger. Be sure the lens is positioned correctly.
- Use your other hand to gently hold your upper eyelid from blinking. Pull down your lower eyelid with the other fingers of your applying hand.
- Look up at the ceiling and gently place the lens on the lower part of your eye.
- Slowly release your eyelid and close your eye for a moment. Then blink a few times to center the lens.

### Removing step by step

- Thoroughly wash your hands with soap and dry using a clean towel.
- Look up at the ceiling and gently pull down holding your lower eyelid.
- Using your index finger gently slide the lens to the lower white part of your eye.
- Gently squeeze the lens between your thumb and index finger and remove it.

### Chemical Lens Disinfection

- Clean the contact lenses with a recommended cleaning solution.
- Thoroughly rinse lenses with a recommended rinsing solution.
- Store lenses in a thoroughly clean contact lens case in a recommended storage solution.
- Leave lenses in unopened storage case. Rinse thoroughly with fresh saline solution prior to next wear.

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## CARE REGIMEN AND SAFETY PRECAUTIONS

### Handling Precautions:

- **DO NOT** use if the sterile blister package is opened or damaged.
- **CAREFULLY** follow the handling, insertion, removal and wearing instructions provided with your contact lenses and those prescribed by the Eye Care Professional.
- **ALWAYS** wash and rinse hands thoroughly before handling lenses. Do not get soaps, lotions, cosmetics, sprays or deodorants in contact with your eyes or lenses.
- **ALWAYS** apply makeup after contacts have been put in your eyes.
- **ALWAYS** handle lenses gently and avoid dropping them.
- **DO NOT** touch the lens with finger nails.
- **NEVER** use tweezers or other tools to remove your lenses from the lens container unless specially indicated for that use. For the lens and solution directly in the palm of your hand.

### Lens Wearing Precautions:

- **IMMEDIATELY** remove your lenses if they become red or irritated.
- **DO NOT DRIVE** while wearing Primal Brand costume lenses. They may obscure vision.
- **NEVER** wear contact lenses beyond the amount of time recommended by the Eye Care Professional. Contact lenses should not be worn for longer than 8 hours a day as this will increase the risk of infection or permanent damage.
- **DO NOT** sleep while wearing lenses. It is recommended to remove lenses during a period of rest overnight or longer.
- Ask the Eye Care Professional about wearing contact lenses during sports activities.
- **AVOID** going in the water while wearing lenses. This includes lakes, the ocean, swimming pools, hot tubs and even the shower. The water may contain unseen threats to your eyes.
- **AVOID** environments that are extremely dry, dusty, handling of harmful chemical substances, exposure to moisture, or extreme changes in temperatures. This may cause irritation or infections of the eye. Consult the Eye Care Professional if you are subject to these environments.
- **AVOID** using contact lenses if you suffer any eye diseases or are allergic to materials contained in the lens.
- **ALWAYS** inform your employer of being a contact lens wearer. Certain jobs require the use of eye protection or may require the individual not wear contact lenses.
- Normal daily wear of contact lenses assumes a minimum 4-hour period of non-lens wear each day. Do not share your lenses with anyone as this may spread microorganisms which could result in serious eye health problems.

### Solution Precautions:

- **ALWAYS** use the recommended product for cleaning, rinsing and storage of contact lenses. Different lens care products cannot always be used together.
- **ALWAYS** use fresh, unexpired lens care solutions.
- If the lens sticks (stops moving or cannot be removed)

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**PRIMAL CONTACTS**  
Soft (Hydrophilic) Contact Lenses  
Eye Care Instructions

**IMPORTANT: Please read carefully and keep this information for future use.**

Rev date : EO-IFU-001(Rev.6)

**DESCRIPTION**

Primal® Brand Contact Lenses are available as corrective and non-corrective lens designs.

The Lenses are composed of 62% Polyacraon and 38% water a silicone-containing hydrogel.

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Primal (non-corrective) contact lenses parameters:

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**INDICATIONS (USES) AND WEARING RESTRICTIONS**

Primal® Brand Soft (Hydrophilic) Power Contact Lenses is indicated for the optical correction of myopia a lens for daily wear.

Primal® Brand Fashion Lenses is indicated for non-corrective use in the colour enhancement of the iris and have no therapeutic effects for daily wear.

**CONTRAINDICATIONS (REASONS NOT TO USE)**

**DO NOT USE THE PRIMAL LENS WHEN ANY OF THE FOLLOWING CONDITIONS EXIST:**

- Inflammation or infection in or around the eye or eyelids.
- Any eye disease, injury, or abnormality that affects the cornea, conjunctiva, or eyelids.
- Any previously diagnosed condition that makes contact lens wear uncomfortable.
- Any systemic disease that may affect the eye or be exacerbated by wearing contact lenses.
- Allergic reactions of ocular surfaces of the eye (adnexa) that may be induced or exaggerated by wearing contact lenses or use of contact lens solutions.
- Allergy to any ingredient, such as mercury or Thimerosal, in a solution which is to be used to care for your contact lenses.
- Severe dry eye.
- Any active corneal infection (bacterial, fungal, protozoal or viral).
- Not to be used if you are unable to follow the lens care regimen or unable to obtain assistance to do so.

Should side effects occur due to the use of Primal® Brand contact lenses, remove the contact lens immediately, and consult your Eye Care Professional before resuming the use of contact lenses.

Primal is a registered trademark of

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