

INSTALLATION INSTRUCTIONS FOR DETECTO PORTABLE WHEELCHAIR SCALE RAMPS

UNPACKING

Before starting to assemble, please unpack and remove all wrappings and fillers. Carton contains:

Item Description	Part Number	Qty.
6550 Ramp Weldment	0065-0675-0A	2
1/4-20 X 1/2" Flat-Head Socket Cap Screw	6021-1401	4
5/32" Short Hex Key	M220-0080-00	1
0.5 ml Packet High Strength Thread Locker	6560-1046	1

ASSEMBLY

Refer to Figures No. 1, 2, and 3, then follow the instructions for attaching the ramps.

1. Fold and lock column onto scale platform. Lift scale by its handles and place platform on its side in the storage position, supported by the metal stand.
2. Attach each ramp in the following sequence:
 - Begin with top rubber mount. Apply thread locker to first screw. Then, while holding ramp against top rubber mount, hand thread screw into top rubber mount (Figure No. 1).
 - Next, apply thread locker to second screw, and while holding ramp against bottom rubber mount, hand thread screw into bottom rubber mount.
 - Tighten both screws with supplied 5/32" Short Hex Key (Figure No. 2).**NOTE:** When installing screws into rubber mount, be careful not to over-tighten.
3. Repeat Step 2 for ramp on other side of platform. Finished assembly should look like Figure No. 3.



Figure No. 1



Figure No. 2



Figure No. 3



Printed in USA

0065-0720-0M Rev A 01/18



203 E. Daugherty, Webb City, MO 64870 USA
Ph: 417-673-4631 or 1-800-641-2008
Fax: 417-673-2153

Technical Support: 1-866-254-8261
E-mail: tech@cardet.com