

**UNPACKING**

Before starting to assemble, please unpack and remove all wrappings and fillers. Carton contains:

Item Description	Part Number	Qty.
6400, 6500, 6550, 6560, and 6570 Ramp Weldment	0065-0675-0A	2
6600 and BRW1000 Ramp Weldment	0065-0731-0A	2
8500 and 8550 Ramp Weldment	0065-0821-0A	2
.25-20 X .438" Flat-Head Socket Cap Screw Zinc Plated	6021-1812	4
5/32" Short Hex Key	M220-0080-00	1
0.5 ml Packet High Strength Thread Locker	6560-1046	1

**ASSEMBLY**

Refer to Figures No. 1, 2, and 3, then follow the instructions for attaching the ramps.

**NOTE: On the 6550, 6560, 6570, and 8550, ensure the column(s) are locked onto scale platform.**

1. Lift scale and place the platform on its side in the transport position. Have an assistant hold the platform while installing the ramps.
2. Attach each ramp in the following sequence:
  - Begin with top rubber mount. Apply thread locker to first screw. Then, while holding ramp against top rubber mount, hand thread screw into top rubber mount (Figure No. 1).
  - Next, apply thread locker to second screw, and while holding ramp against bottom rubber mount, hand thread screw into bottom rubber mount.
  - Tighten both screws with supplied 5/32" Short Hex Key (Figure No. 2).

**NOTE:** When installing screws into rubber mount, be careful not to over-tighten.

3. Repeat Step 2 for ramp on other side of platform. Finished assembly should look like Figure No. 3.



Figure No. 1



Figure No. 2



Figure No. 3



Printed in USA

0065-0720-0M Rev C 02/19

Cardinal Scale Manufacturing Co.  
203 E. Daugherty, Webb City, MO 64870 USA  
Ph: 417-673-4631 or 1-800-641-2008  
Fax: 417-673-2153

Technical Support: 1-866-254-8261  
E-mail: tech@cardet.com