

DS600 Electronic Bluetooth Enabled Personal Scale

Instructions for Use

WARNING: To prevent slipping, do not place scale on a wet surface or use while wet.

SPECIFICATIONS

- High precision strain gauge sensors
- Capacity: 11-400 lb (5-180 kg)
- Division: 0.2 lb (0.1 kg)
- Auto step on/Auto off
- Low battery/Overload indications
- Unit: lb / kg / st
- Power: 4xAA batteries (not included)
- Working temperature: 50-104°F
- Large Scale platform (12 ¾" x 12 ¾")
- Bluetooth 4.0

OPERATION

1. Place the scale on a hard, flat surface. Do not use on carpet or other soft surfaces.
2. Stand on the scale and it will automatically measure your weight.
3. Stand still. The scale will flash your weight and automatically transmit it to your Bluetooth 4.0 device.
4. The scale will automatically turn off after 8 seconds of inactivity.

TO CONVERT WEIGHT UNIT

1. Press the “Unit” button on the back side of the scale.

ERROR MESSAGES

1. “EEEE” indicates that the scale is overloaded. Step off scale immediately to avoid damaging the scale.
2. “Lo” indicates that the batteries are low. Replace the batteries as soon as possible.

ADVISE FOR USE & CARE

1. The glass platform will be slippery when it is wet. Keep it dry!
2. Stand still during the course of weighing.
3. DO NOT strike, shock or drop the scale.
4. Treat the scale with care as it is a precise instrument.
5. Clean the scale with a damp cloth and avoid water getting inside the scale. DO NOT use chemical/abrasive cleaning agents.
6. Keep the scale in a cool place with dry air.
7. Always keep your scale in a horizontal position.
8. If the scale fails to turn on, check whether the batteries are installed or battery power is low. Insert or replace the batteries.
9. If there is an error on the display or the scale does not shut off after a long period of time, remove the batteries for 3 seconds and reinstall.
10. The scale is intended to weigh people. Do not weigh other items on the scale.

ERROR RANGES

100lb: +/-0.6lb 200lb: +/-0.8lb 300lb: +/-1.0lb