

Reorder No. 12950

 Manufactured for:  
Dynarex Corporation  
10 Glenshaw Street  
Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com)  
Made in China


# ***Metal Swing Rail***

***(Compatible with Dynarex  
D-Series LTC Beds)***




## MOUNTING INSTRUCTIONS



1. Before installing the rails, the bed should be in the flat position. The carton should include 2 bed rails as shown.

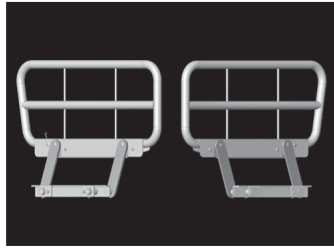

2. Unclip and remove the metal pins from the rail mounting brackets. Hook the rail bracket onto the edge of the bed decking. Align the rail bracket mounting holes with the bed decking mounting holes.

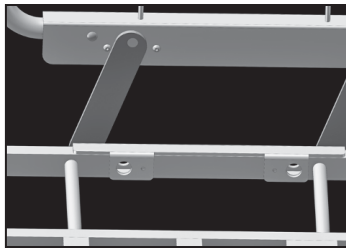

3. Re-insert the metal pins through the mounting holes and secure with the snap rings. Make sure the snap rings are positioned in the groove of the metal pins. Repeat the same procedure to install the other rail.

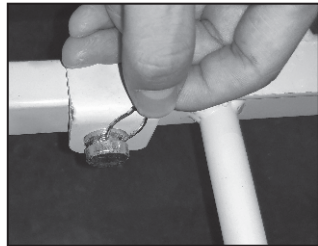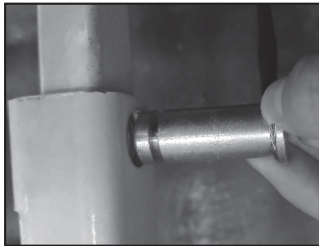

The rails can be positioned at the head or foot section.

## RAIL OPERATION

**RAISE BED RAIL:** Pull the bed rail upwards until the rail is securely locked in place.

**LOWER BED RAIL:** Squeeze the release lever at the bottom corner of the bed rail and slowly lower the bed rail to its lowest position.

## WARNINGS

Read the instructions carefully before installing.

Before operating, ensure that the bed rails are assembled and installed correctly as instructed. Ensure that all the attaching hardware are tightened and that the bed rails are securely in place. Inspect the hardware frequently to make sure all the components are tightened securely.

**DO NOT** use the bed rails as a handle for getting into or out of the bed. This could result in personal injury and damage to the bed rails.

**DO NOT** apply side pressure to the bed rails. This could deform or even break the bed rails.

**DO NOT** use the bed rails as a push handle for moving the bed. This could deform or even break the bed rails.

**DO NOT** use the bed rails if proper installation cannot be achieved. Contact your dealer or manufacturer.

## LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

\*This product meets or exceeds all federal safety and environmental regulations.



**SYMBOL GLOSSARY**  
For an explanation of symbols used in Dynarex packaging, visit [dynarex.com/symbols.php](http://dynarex.com/symbols.php)

**WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)