

SiliGentle™

Silicone Foam Dressing



INSTRUCTIONS FOR USE

Description

SiliGentle™ Silicone (Bordered) Foam Dressing consists of a silicone wound contact layer, a super absorbent pad layer, and a vapor permeable and waterproof film. The triple-layer construction facilitates dynamic fluid management to provide optimal moist wound environment, which leads to the promotion of faster wound healing and may help to reduce the risk of maceration. The gentle silicone layer can be moved without losing its adherence. Also, this layer can minimize pain and trauma to wound and its surrounding skin. The absorbent pad layer provides superior absorption and locks exudates away.

Indications

SiliGentle™ Silicone (Bordered) Foam Dressing is indicated for the management of partial and full thickness wounds, including:

1. Pressure ulcers
2. Leg and foot ulcers
3. Diabetic ulcers
4. Venous ulcers
5. Pressure sores
6. Traumatic wounds
7. Donor sites
8. Skin tears

For external use only.

Instructions for Use

1. Clean the wound in accordance with normal clinical protocol.
2. Remove excess moisture from surrounding skin.
3. Select an appropriate dressing according to wound size.
4. Remove release films and apply adherent side to the wound – do not stretch.
5. For best results, the dressing should overlap the wound margins by at least 1-2 cm.

Warnings

1. Do not use SiliGentle™ Silicone (Bordered) Foam Dressing together with oxidizing agents such as hypochlorite solutions or hydrogen peroxide.
2. If reddening or sensitization occur discontinue use.
3. SiliGentle™ Silicone Foam Dressing can remain in place for up to 7 days, dependent on the wound conditions and exudate levels. It is recommended that the dressing be changed every 24 hours initially, then proceeded to less frequent changes as necessary.

Disposal

Disposal should be handled according to local environmental procedures.

Specifications

Code	Size	Size
3055	2 in. × 2 in. (5 cm × 5 cm)	Non-Border
3056	4 in. × 4 in. (10 cm × 10 cm)	Non-Border
3057	6 in. × 6 in. (15 cm × 15 cm)	Non-Border
3058	2 in. × 2 in. (5 cm × 5 cm)	Bordered
3059	4 in. × 4 in. (10 cm × 10 cm)	Bordered
3060	6 in. × 6 in. (15 cm × 15 cm)	Bordered
3061 (Heel)	10 in. × 9 in. (25.4 cm × 22.9 cm)	Bordered
3062 (Sarcal)	7 in. × 7 in. (17.78 cm × 17.78 cm)	Bordered
3063 (Large Sarcal)	9 in. × 9 in. (23 cm × 23 cm)	Bordered

Manufactured for:
Dynarex Corporation
 10 Glenshaw Street
 Orangeburg, NY 10962
 USA • www.dynarex.com

Made in China



SYMBOL GLOSSARY
 For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

R190308