

























⚠ WARNING: For complete operation and safety information, refer to the *EPC500ST Procedure Chair Operation Manual*, available at www.hausted.com.

	WAKE PENDANT	The pendant will go into <i>SLEEP MODE</i> after five minutes of no activity – before pressing the desired function button, press any button to first wake up the pendant.
	LOCK ALL FUNCTIONS	Press and hold LOCK button for three seconds to lock all functions. After five LED flashes, all four <i>UP</i> LEDs illuminate steady green, indicating they are now locked; an audible signal also indicates when locked and beeps up to three times until LOCK button is released.
	UNLOCK ALL FUNCTIONS	Press and hold LOCK and BATTERY buttons for one second to unlock all functions. A quick LED flash indicates they are now unlocked; an audible signal also indicates when unlocked.
	UNLOCK INDIVIDUAL FUNCTION	Press and hold LOCK button and press each <i>UP</i> button to unlock each function individually. As each button is released, its LED will go out, indicating that function is unlocked.
	BATTERY	When the battery discharges to 20% capacity or less, a continuous beep sounds during motor operation, the LED above the <i>BATTERY</i> button illuminates steady green, and the control box LED illuminates amber.
	LOCK / UNLOCK BRAKE	Press the <i>BRAKE</i> button to lock or unlock all four caster wheels (toggle brakes on / off). <i>Note: To prevent unintended movement, brakes lock automatically after chair is stationary for 30 consecutive seconds.</i>
	STEER MODE	Press the <i>STEER MODE</i> button to toggle <i>STEER LOCK</i> position on / off. When <i>STEER LOCK</i> position is on, all four caster brakes will unlock, and foot-end casters will remain straight so that chair steers in a straight line. <i>Note: STEER MODE can be engaged even if brakes are locked.</i>
	EXTEND CHAIR LENGTH – BUT FIRST, SUPINE	To extend chair length, first press and hold the <i>SUPINE</i> button until the chair is completely horizontal.
	EXTEND CHAIR LENGTH	Then, at foot end, press and hold the black toggle button below the <i>EXTEND</i> label, shown at left. <i>Note: The motorized leg extension can only be extended when the leg is in the horizontal position.</i>
	CHAIR POSITION – BUT FIRST, RETRACT	Before transitioning to <i>CHAIR</i> position, retract chair length: at foot end, press and hold the black toggle button below the <i>RETRACT</i> label, shown at left. <i>Important: Hold this button until the chair length retracts completely.</i>
	CHAIR POSITION	Press and hold the <i>HOME</i> button to transition to <i>CHAIR</i> position. The chair automatically raises back section, lowers leg section, and tilts seat section simultaneously; chair also automatically adjusts height. No LED. <i>Note: The leg extension must first be fully retracted; if not, the HOME button will beep when pressed, but nothing will move.</i>
	TRENDELENBURG POSITION	Press and hold the <i>TRENDELENBURG</i> button to transition to <i>TRENDELENBURG</i> position. The chair automatically lowers back section, raises leg section, and tilts seat section backward simultaneously; chair also automatically adjusts height. No LED.
	REVERSE TRENDELENBURG	Press and hold the <i>REVERSE TRENDELENBURG</i> button until desired position is achieved. Chair automatically lowers back section, raises leg section, and tilts seat section forward simultaneously; chair also automatically adjusts height. No LED.

  <small>Height</small>	CHAIR HEIGHT UP	Press and hold HEIGHT button to raise or lower chair height.	  <small>Back</small>	BACK UP	Press and hold BACK button to raise or lower back section.
	CHAIR HEIGHT DOWN			BACK DOWN	
  <small>Seat Tilt</small>	SEAT TILT UP	Press and hold SEAT TILT button to raise or lower seat tilt.	  <small>Leg</small>	LEG UP	Press and hold LEG button to raise or lower leg section.
	SEAT TILT DOWN			LEG DOWN	

MEMORY AND PRESET FUNCTIONS		
   		
M	MEMORY	Position chair to desired position. Enter MEMORY MODE by pressing and holding MEMORY button for three seconds until LED flashes. Once MEMORY LED flashes, simultaneously press and hold MEMORY button and desired PRESET button (1, 2, or 3) until beeping stops (MEMORY LED will stop flashing and go out and PRESET LED will illuminate). Once MEMORY button and PRESET button are released, the position saves, LEDs go out, and MEMORY MODE exits. (After entering MEMORY MODE , PRESET buttons that illuminate are already programmed, but can be overwritten; PRESET buttons that don't illuminate are not yet programmed.)
1	PRESET 1	Press and hold PRESET 1 button until saved pre-programmed position is achieved. LED illuminates steady green while pressed, goes out when released.
2	PRESET 2	Press and hold PRESET 2 button until saved pre-programmed position is achieved. LED illuminates steady green while pressed, goes out when released.
3	PRESET 3	Press and hold PRESET 3 button until saved pre-programmed position is achieved. LED illuminates steady green while pressed, goes out when released.

IMPORTANT: PLUG CHAIR INTO POWER OUTLET EVERY NIGHT WHEN NOT IN USE.

