
























**⚠ WARNING:** For complete operation and safety information, refer to the *ESC2EYEST Procedure Chair Operation Manual*, available at [www.hausted.com](http://www.hausted.com).

	<b>WAKE PENDANT</b>	The pendant will go into <i>SLEEP MODE</i> after five minutes of no activity – before pressing the desired function button, press any button to first wake up the pendant.
	<b>LOCK ALL FUNCTIONS</b>	Press and hold <b>LOCK</b> button for three seconds to lock all functions. After five LED flashes, all four <i>UP</i> LEDs illuminate steady green, indicating they are now locked; an audible signal also indicates when locked and beeps up to three times until <b>LOCK</b> button is released.
	<b>UNLOCK ALL FUNCTIONS</b>	Press and hold <b>LOCK</b> and <b>BATTERY</b> buttons for one second to unlock all functions. A quick LED flash indicates they are now unlocked; an audible signal also indicates when unlocked.
	<b>UNLOCK INDIVIDUAL FUNCTION</b>	Press and hold <b>LOCK</b> button and press each <i>UP</i> button to unlock each function individually. As each button is released, its LED will go out, indicating that function is unlocked.
	<b>BATTERY</b>	When the battery discharges to 20% capacity or less, a continuous beep sounds during motor operation, the LED above the <b>BATTERY</b> button illuminates steady green, and the control box LED illuminates amber.
	<b>LOCK / UNLOCK BRAKE</b>	Press the <b>BRAKE</b> button to lock or unlock all four caster wheels (toggle brakes on / off). <i>Note: To prevent unintended movement, brakes lock automatically after chair is stationary for 30 consecutive seconds.</i>
	<b>STEER MODE</b>	Press the <b>STEER MODE</b> button to toggle <b>STEER LOCK</b> position on / off. When <b>STEER LOCK</b> position is on, all four caster brakes will unlock, and foot-end casters will remain straight so that chair steers in a straight line. <i>Note: STEER MODE can be engaged even if brakes are locked.</i>
	<b>EXTEND CHAIR LENGTH – BUT FIRST, SUPINE</b>	To extend chair length, first press and hold the <b>SUPINE</b> button until the chair is completely horizontal.
	<b>EXTEND CHAIR LENGTH</b>	Then, at foot end, press and hold the black toggle button below the <b>EXTEND</b> label, shown at left. <i>Note: The motorized leg extension can only be extended when the leg is in the horizontal position.</i>
	<b>CHAIR POSITION – BUT FIRST, RETRACT</b>	Before transitioning to <b>CHAIR</b> position, retract chair length: at foot end, press and hold the black toggle button below the <b>RETRACT</b> label, shown at left. <i>Important: Hold this button until the chair length retracts completely.</i>
	<b>CHAIR POSITION</b>	Press and hold the <b>HOME</b> button to transition to <b>CHAIR</b> position. The chair automatically raises back section, lowers leg section, and tilts seat section simultaneously; chair also automatically adjusts height. No LED. <i>Note: The leg extension must first be fully retracted; if not, the HOME button will beep when pressed, but nothing will move.</i>
	<b>TRENDELENBURG POSITION</b>	Press and hold the <b>TRENDELENBURG</b> button to transition to <b>TRENDELENBURG</b> position. The chair automatically lowers back section, raises leg section, and tilts seat section backward simultaneously; chair also automatically adjusts height. No LED.

  <small>Height</small>	<b>CHAIR HEIGHT UP</b>	Press and hold <i>HEIGHT</i> button to raise or lower chair height.	  <small>Back</small>	<b>BACK UP</b>	Press and hold <i>BACK</i> button to raise or lower back section.
	<b>CHAIR HEIGHT DOWN</b>			<b>BACK DOWN</b>	
  <small>Seat Tilt</small>	<b>SEAT TILT UP</b>	Press and hold <i>SEAT TILT</i> button to raise or lower seat tilt.	  <small>Leg</small>	<b>LEG UP</b>	Press and hold <i>LEG</i> button to raise or lower leg section.
	<b>SEAT TILT DOWN</b>			<b>LEG DOWN</b>	

MEMORY AND PRESET FUNCTIONS		
   		
<b>M</b>	<b>MEMORY</b>	Position chair to desired position. Enter <i>MEMORY MODE</i> by pressing and holding <i>MEMORY</i> button for three seconds until LED flashes. Once <i>MEMORY</i> LED flashes, simultaneously press and hold <i>MEMORY</i> button and desired <i>PRESET</i> button (1, 2, or 3) until beeping stops ( <i>MEMORY</i> LED will stop flashing and go out and <i>PRESET</i> LED will illuminate). Once <i>MEMORY</i> button and <i>PRESET</i> button are released, the position saves, LEDs go out, and <i>MEMORY MODE</i> exits. (After entering <i>MEMORY MODE</i> , <i>PRESET</i> buttons that illuminate are already programmed, but can be overwritten; <i>PRESET</i> buttons that don't illuminate are not yet programmed.)
<b>1</b>	<b>PRESET 1</b>	Press and hold <i>PRESET 1</i> button until saved pre-programmed position is achieved. LED illuminates steady green while pressed, goes out when released.
<b>2</b>	<b>PRESET 2</b>	Press and hold <i>PRESET 2</i> button until saved pre-programmed position is achieved. LED illuminates steady green while pressed, goes out when released.
<b>3</b>	<b>PRESET 3</b>	Press and hold <i>PRESET 3</i> button until saved pre-programmed position is achieved. LED illuminates steady green while pressed, goes out when released.

**IMPORTANT: PLUG CHAIR INTO POWER OUTLET EVERY NIGHT WHEN NOT IN USE.**

Information contained herein is subject to change.  
The most current and complete product information can be found on our website:  
[www.grahamfield.com](http://www.grahamfield.com) / [www.hausted.com](http://www.hausted.com)



© 2021, GF Health Products, Inc. All Rights Reserved.  
Hausted is a trademark of GF Health Products, Inc.,  
One Graham-Field Way, Atlanta, GA 30340-3140, U.S.A.  
GF Health Products, Inc. is an ISO 13485:2016 / MDSAP  
Certified Company.