



Operating Manual

Hausted® Fluoro-Track
Fluoroscopy Stretcher
Model: 493RPC
GFH1300089RevC21



A WORD FROM GF HEALTH PRODUCTS, INC.

This manual contains important information on proper use and maintenance of the Hausted® Fluoro-Track Fluoroscopy Capable Stretcher Model: 493RPCST. All personnel involved in the use and maintenance of this equipment must carefully review and comply with the warnings, cautions and instructions contained in this manual. These instructions are important to protect the health and safety of personnel operating a Hausted Fluoro-Track Fluoroscopy Capable Stretcher Model: 493RPCST and should be retained in a conveniently accessible area for quick reference.

Instructions for uncrating and connecting utilities, as well as equipment drawings, have been furnished. If missing, contact GF Health Products, Inc. for replacement copies, with the serial and model number of the unit.

GF Health Products, Inc. carries a complete line of accessories for use with this stretcher. A representative will gladly review these with you.

Indications for Use

The Hausted Fluoro-Track Fluoroscopy Capable Stretcher Model: 493RPCST is intended for use in patient treatment, transport and recovery.

Service Information

A thorough preventive maintenance program is essential to safe and proper unit operation. This manual contains maintenance schedules and procedures which should be followed for satisfactory equipment performance.

We encourage you to contact GF Health Products, Inc. with any maintenance concerns.

Advisory

A listing of the safety precautions to be observed when operating and servicing this equipment can be found in Section 1 of this manual. Do not operate or service the equipment until you have become familiar with this information.

Any alteration of this equipment not authorized by GF Health Products, Inc., which could affect its operation, will void the warranty, could violate national, state, and local regulations, and could jeopardize your insurance coverage.

WARNING - COPYING PROHIBITED

This manual is protected by Federal Copyright Law, which provides for damages of up to USD \$20,000, as well as criminal fines and imprisonment, for unauthorized copying.

Info: Column 1 below applies only if product was purchased outside the U.S.

	
	EC Authorized Representative: CEpartner4U BV ESDOORNLAAN 13 3951DB MAARN The Netherlands +31(0)6 516 536 26

	Manufactured by: GF Health Products, Inc. One Graham-Field Way Atlanta GA 30340-3140 1.770.368.4700 Main 1.770.368.2386 Fax www.grahamfield.com www.Hausted.com
---	--

Equipment not suitable for use in the presence of a flammable anesthetic mixture with air or oxygen or nitrous oxide.

The base language of this document is English. any translations must be made from the base language document.


TABLE OF CONTENTS

1	Listing of Warnings and Cautions.....	1-1
2	Uncrating Instructions.....	2-1
3	Operating Instructions.....	3-1
3.1	Braking and Steering Operation	3-1
3.1.1	Applying the Brakes.....	3-1
3.1.2	Releasing the Brakes	3-1
3.1.3	Applying the Steering Lock/Fifth Wheel.....	3-1
3.1.4	Releasing the Steering Lock/Fifth Wheel.....	3-1
3.2	Litter Top Height Adjustment.....	3-2
3.2.1	Height Adjustment	3-2
3.2.2	Lowering Litter Top	3-2
3.2.3	Trendelenburg Adjustment.....	3-2
3.2.4	Reverse Trendelenburg Adjustment	3-3
3.3	Retracto Rail Operation.....	3-3
3.3.1	Raising the Rail	3-3
3.3.2	Half Height.....	3-3
3.3.3	Lowering the Rail.....	3-3
3.4	Fowler Backrest Operation	3-4
3.4.1	Raising the Backrest.....	3-4
3.4.2	Lowering the Backrest	3-4
3.5	Sliding Top Operation.....	3-5
3.5.1	Sliding Top Operation.....	3-5
4	Preventive Maintenance for the User	4-1
5	Recommended Optional Accessories.....	5-1
6	Warranty.....	6-1
7	Disposal and Key to Symbols	7-1




The following Safety Precautions must be observed when operating or servicing this Hausted® Fluoro-Track Fluoroscopy Capable Stretcher Model: 493RPCST. WARNING indicates the potential for personal injury and CAUTION Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury or damage to equipment. For emphasis, certain Safety Precautions are repeated throughout the manual. It is important to review ALL Safety Precautions before operating or servicing the unit.

Strictly following these Safety Precautions enhances your ability to safely and effectively utilize the unit and helps the customer avoid improper maintenance methods which may damage the unit or render it unsafe. It is important to understand that these Safety Precautions are not exhaustive; customers are encouraged to develop their own safety policies and procedures to enhance and complement these Safety Precautions.

WARNING – LACERATION HAZARD:

-  When cutting bands, always use a tool specifically designed for that purpose. This will help to avoid personal injuries frequently incurred when bands are cut and tension released.

WARNINGS – PERSONAL INJURY

-  Do not sit on end – tipping may occur.
-  This stretcher has a warning label located at the head end stating: Maximum patient weight 500 lb (226 kg) EVENLY DISTRIBUTED.
-  Patient entry, egress and transfer should always be done with the brakes locked.



The brakes should always be locked and patient side rails up when patient is not in transport.



Be sure both rails are in the upright locked position before leaving patient.

WARNING – CAUTION – PROPER OPERATION HAZARD:



Floors must be smooth and level to maintain optimum fifth wheel steering. Fifth wheel steering functions can be influenced by floor irregularities (bumps or dips) greater than 1/2" (13 mm) across the span of the stretcher.



Ensure patient and caregiver (or attendant) body and extremities are clear of pinch points before operating the rail.



This product can expose you to chemicals including Di(2-ethylhexyl)phthalate (DEHP) which is known to the State of California to cause cancer or birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov/furniture.



IMPORTANT - REPORT ANY SHIPPING DAMAGE IMMEDIATELY:

Inform shipper of any damages - leave carton intact. Leave the equipment in the receiving area until inspection is complete.

CAUTION - POSSIBLE EQUIPMENT DAMAGE:

This crate contains fragile expensive medical equipment. Uncrate and handle carefully.

If after uncrating the equipment you find any damage (no matter how slight), show it to your supervisor.



WARNING - PERSONAL INJURY HAZARD:

When cutting bands, always use tool specifically designed for that purpose. This will help avoid personal injuries frequently incurred when bands are cut and tension is released.

IMPORTANT: Follow each step in the order shown in these instructions.

UNPACKING INSTRUCTIONS:

Your Hausted equipment has been carefully packed at our manufacturing plant to ensure safe shipment to your medical facility. There are several procedures you must follow to put your new equipment in service. These procedures only take a few minutes to complete and are required to ensure proper operation of the equipment.

1. Cut the two bands around the shipping carton.
2. Remove the top half of the carton and cut one side of the bottom half.
3. Remove the equipment from the carton.
4. Check to see if all features of the equipment work properly. If all the features work, advance to step 5. If any of the features of the equipment do not work properly, call GF Health Products, Inc. for service at: 1.770.368.4700.
5. Clean the equipment using mild detergent to remove any dirt accumulated during shipment, and place the equipment into service.

OPERATING INSTRUCTIONS

3.1 BRAKING AND STEERING OPERATION

3.1.1 Applying the Brakes

The four wheel central braking system is activated by depressing the red pedal at any of the four corners of the unit (Figure 3-1). To fully engage the brakes, the pedal should be pressed to approximately 45°. All four caster wheels should be locked from swiveling and rotating.



FIGURE 3-1

3.1.2 Releasing the Brakes

Depress the green pedal at any of the four corners of the unit, until the pedal is in a horizontal position (Figure 3-2). All four wheels should rotate and swivel freely.

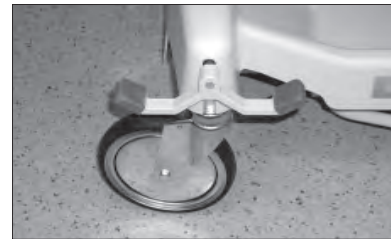


FIGURE 3-2

NOTE: Brakes should always be activated when unit is not in transport"

3.1.3 Applying the Steering Lock/Fifth Wheel

From any corner of the stretcher, depress the green pedal downward into the locked position. (Figure 3-3). Push the stretcher forward. Either one caster at the foot end will lock into a non-swivel mode, or the optional fifth wheel will lower and apply pressure to the floor. This allows straight steering by the attendant.



FIGURE 3-3

WARNING - PROPER OPERATION HAZARD:

Floors must be smooth and level to maintain optimum fifth wheel steering. Fifth wheel steering functions can be influenced by floor irregularities (bumps or dips) greater than 1/2" (13 mm) across the span of the stretcher.

3.1.4 Releasing the Steering Lock/Fifth Wheel

Depress the red pedal at any of the four corners of the unit, until the pedal is in a horizontal position (Figure 3-2). All four wheels should rotate and swivel freely, and/or the optional fifth wheel will lift off the floor.

3.2 LITTER TOP HEIGHT ADJUSTMENT

3.2.1 Height Adjustment

Press the pump pedal either at base side (Figure 3-4A) or foot end (Figure 3-4B) to the floor, then release. Repeat this process until desired height is obtained. Use smooth strokes on either pump pedal to ensure patient comfort.



FIGURE 3-4A

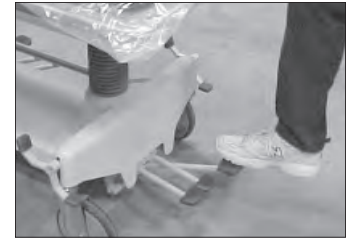


FIGURE 3-4B

3.2.2 Lowering the Litter Top

Press down on the two release pedals either at base side (Figure 3-5A) or foot end (Figure 3-5B) at the same time until desired height is obtained.



FIGURE 3-5A

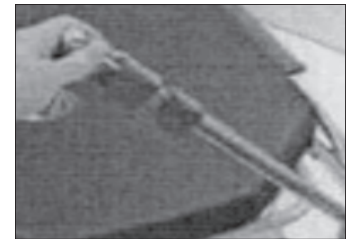


FIGURE 3-5B

NOTE: Do Not Stand On Release Pedals.

3.2.3 Trendelenburg Adjustment

Place the unit at maximum height, see “height adjustment.” Press down on the release pedal at base side nearest the head end (Figure 3-6A) or right hand release pedal at foot end (Figure 3-6B) until desired position is obtained then remove pressure.



FIGURE 3-6A



FIGURE 3-6B

3.2.4 Reverse Trendelenburg Adjustment

Place the unit at maximum height, see “height adjustment.” Press down on the release pedal at base side nearest the foot end (Figure 3-7A) or left hand pedal at foot end (Figure 3-7B) until desired position is obtained then remove pressure.

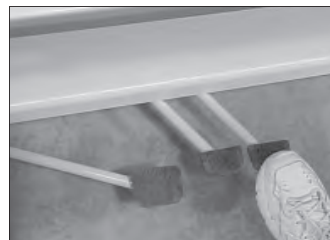


FIGURE 3-7A



FIGURE 3-7B

3.3 RETRACTO RAIL OPERATION

⚠ WARNING:
Always ensure side rails are in upright locked position before leaving patient unattended.

3.3.1 Raising the Rail

Grasp rail top cap in the middle of the rail (Figure 3-8A) and lift.



FIGURE 3-8A

3.3.2 Half Height

Grasp the rail (Figure 3-8A) and lift up on the red trigger under the litter top (Figure 3-8B) while lowering the rail. When rail starts to move down, release trigger. Lower rail until it locks into half height position.



FIGURE 3-8B

3.3.3 Lowering the Rail

Grasp the rail (Figure 3-8A) and lift up on the red trigger under the top (Figure 3-8B) while lowering the rail. Hold the trigger up until the rail is all the way down.



FIGURE 3-9

⚠ WARNING:
Ensure patient and caregiver (or attendant) body and extremities are clear of pinch points before operating the rail (Figure 3-9).

NOTE: Be sure both rails are in the upright locked position before leaving patient.

3.4 FOWLER BACKREST OPERATION

3.4.1 Raising the Backrest

Grasp the fowler backrest tube and the gas spring activating handle (Figure 3-10), squeeze the handle and the tube together and lift. Once the desired incline is achieved, release the handle. (Amount of manual lifting required will vary depending on patient size; the gas springs are intended only to assist you in lifting the patient).



FIGURE 3-10

3.4.2 Lowering the Backrest

Grasp the fowler backrest tube and the gas spring activating handle (Figure 3-10). Place your opposite hand on the backrest (Figure 3-11). Squeeze the handle and the tube together while supporting tube. Adjust incline by either slowly lowering patient or by pressing the fowler down. (The amount of manual lifting or supporting required will vary, depending on patient size; the gas springs are intended only to assist you in lifting or lowering the patient.)



FIGURE 3-11

3.5 SLIDING TOP OPERATION

3.5.1 Sliding Top Operation

Pull out on red release trigger pin with index finger (Figure 3-12) to unlock red release handle. While holding release trigger pin outward with the index finger, at same time grasp the red release handle (Figure 3-13) and lift upward. With the release handle lifted upward, grasp the stretcher top frame with the other hand (Figure 3-14) and slide the stretcher top to extend toward the head end or foot end of the stretcher. When top is in desired position, let go of the release handle to lock in place. On the release lever side of the stretcher near the head end, there is a top position label and red indicator to record the precise position of the stretcher top (Figure 3-15).

NOTE: Return stretcher top to center before moving or transporting patient.



FIGURE 3-12



FIGURE 3-13



FIGURE 3-14



FIGURE 3-15

Component	Cleaning Procedure	Schedule	Cleaning Agent *	Special Notes
Pads / Mattresses	Wipe with damp cloth to remove any foreign material	After each use	Routine hospital grade disinfectants, soap and water	Use only medium strength cleaners Do not steam clean or pressure wash
Stretcher	Wipe with damp cloth to remove any foreign material		Routine hospital grade disinfectants, soap and water	Lubricate pivot points after cleaning
Mechanical stretcher components	Wipe with damp cloth to remove any foreign material		Routine hospital grade disinfectants, soap and water	Lubricate pivot points after cleaning
Mechanical accessories	Wipe with damp cloth to remove any foreign material		Routine hospital grade disinfectants, soap and water	Lubricate pivot points after cleaning
Procedure		Schedule	Material	
Lubricate all moving and sliding parts and hinge points		Every 6 months	Lithium-based grease	
⚠ NEVER LUBRICATE GAS SPRING, MOTOR OR MECH-LOCK SHAFTS ⚠				
Inspect all fasteners to ensure proper fit, position and tightness, including nuts, bolts, etc.		Every 3 months	Proper size wrench and screwdriver	
Inspect all surfaces and remove any sharp or burred areas; apply touch-up paint where required			Metal file, proper color paint (specify color when ordering)	
* Disinfecting and Cleaning Upholstery - ALWAYS follow manufacturer's recommended dilution				
Disinfectants for vinyl products	Phenolic disinfectants are the best choice for vinyl			
	Properly diluted quaternaries are also acceptable for vinyl			
	Quaternary / Isopropyl disinfectants ARE NOT recommended for vinyl			
Disinfectants for urethane products (Standard Upholstery)	Quaternary disinfectants are recommended for urethane			
	Quaternary / Isopropyl disinfectants are recommended for urethane			
	Phenolics SHOULD BE AVOIDED on urethane			
Disinfectants for all products	All fabrics may be cleaned with a 1:10 dilution of household bleaches containing 5.25% sodium hypochlorite as recommended by the Centers for Disease Control in Atlanta, Georgia; there is no harmful effect on the fabric			
	Disinfectants applied at full concentration or in highly concentrated solutions will decrease the useful life of fabric			
	Iodophor-type disinfectants used on fabric may result in staining			
Soils or Stains	Use neutral soapsuds and lukewarm water; DO NOT use harsh cleansers, solvents or detergents			
Hard-To-Clean Spots	Use standard household / vinyl cleansers and a soft bristle brush on troublesome spots or stains; presoak heavy, dried-on soil			
Laundering	Laundering Vinyl-laminated, Polyurethane-coated, or Rubber-coated fabric IS NOT recommended; laundering may substantially decrease the useful life of the fabric			

▲ NOTICE – POSSIBLE EQUIPMENT DAMAGE HAZARD: Steam cleaning and pressure washing of stretcher is not recommended and can void warranty.

Info: For more detailed information, contact GF Health Products, Inc. at 1.770.368.4700.

Info: GF Health Products, Inc. offers customized Preventative Maintenance Service Programs for Hausted products; contact your GF sales representative for further information.

Optional Accessories	
Number	Description
00001200	Patient safety strap with buckle (Qty. 1)
00001400	Patient safety strap with Velcro (Qty. 1)
00001800	IV Pole - telescoping, removable, stainless steel, 27" x 54" (69 cm x 137 cm)
00005400	Lateral x-ray cassette holder, holds cassettes up to 14"-17" (36 cm- 43 cm)
00006B00	Head/Footboard, stationary
00007B00	Head/Footboard, with extension
000E1700	IV Pole, fixed height, removable, stainless, 42" (107 cm)
000N4500	Monitor shelf - removable, folding
00CR6B00	Head/Footboard with chart holder, stationary
00CR7B00	Head/Footboard with chart holder and extension
00N16A00	Heel stirrups with mounting adapters (pair)
00N16C00	Knee crutch stirrups with mounting- adapters (pair)
00N16E00	Knee crutch and heel stirrups with mounting adapters (pair)
03165460	Mattress, 2" x 24-1/2" x 76" (5cm x 62cm x 193cm) conductive foam
03165560	Mattress, 3" x 24-1/2" x 76" (8cm x 62cm x 193cm) conductive foam
06189360	PressureCare mattress 4" x 24-1/2" x 76" (10cm x 62cm x 193cm) conductive foam
06536600	Universal patient tray
06576500	Footboard with monitor shelf and chart holder
06589700	Mobile IV stand attachment
06883600	Footboard with monitor shelf, chart holder, and extension
06892160	Enhanced PressureCare mattress 5" x 24 1/2" x 76" (13cm x 62cm x 193cm) with Fusion II material, hunter green
06920000	Slick Patient Transfer System 4" x 24-1/2" x 76" (10cm x 62cm x 193cm) The Slick 2 part system consists of transfer mattress, and base mattress.
06925700	Paper roll holder
07562600	Patient protecting side rail covers, padded (pair)
07565600	Push handles, self-storing (pair)
07633400	Armboard with 1" pad and surgical mounting adapter
12845000	Vertical oxygen tank holder
12877000	IV Pole, telescoping with cable attachment
12884000	Drainage bag hooks (pair)
13193400	Burgundy color bumpers and striping for department identification
13193500	Teal color bumpers and striping for department identification
13193600	Plum color bumpers and striping for department identification
13317000	Under fowler utility tray (stainless steel)

 It is recommended that only accessories approved by GF Health Products, Inc. be used with this device.

To order accessories, or for more detailed information on accessories, please contact GF Health Products, Inc. at 1.770.368.4700.

GF HEALTH PRODUCTS, INC. LIMITED WARRANTY FOR HAUSTED BRAND STRETCHERS AND CHAIRS

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use and maintenance of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable.

The warranted components and time period are set forth below :

COMPONENT	PARTS WARRANTY	LABOR WARRANTY*
Frame.....	5 years.....	1 year
Casters	1 year	1 year
Electrical components	1 year	1 year
Hydraulics.....	1 year	1 year
Optional accessories.....	1 year	1 year
Patient weighing system	1 year	1 year
Pneumatic gas springs	1 year	1 year
Upholstered pads.....	1 year	1 year
Replacement parts**.....	90 days	

* The Labor Warranty applies only to products purchased within the U.S. Labor is not warranted for products purchased outside the U.S.

** The warranty period is as designated above. If a part is replaced under warranty, the original warranty period will not be affected. All other replacement parts will be subject to the warranty period specified.

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

Obtaining Warranty Service Inside the U.S.: A GF Customer Service Representative must authorize warranty service. Please contact the GF Customer Service department by calling 1.770.368.4700 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

Obtaining Warranty Service Outside the U.S.: The Purchaser must contact the Distributor from whom the product was purchased, commensurate with the guidelines set forth by that Distributor. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a consumable nature including, but not limited to: filters, fuses, gaskets and lubricants;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS.

THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

CERTAIN STATES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE STATES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's Terms and Conditions, found on GF's website and which may be updated from time to time, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this Hausted product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Outward freight damage must be notated on the appropriate shipping documents.
- 3) Claims for any short shipment or concealed damage must be made within three (3) days of the invoice date.
- 4) Additional international, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.

DISPOSAL

Hausted equipment and accessories can be disposed of.

















We recommend disassembling and dividing the equipment and components into different waste groups such as: metal, cable, electronic, recoverable resource and plastic for recycling or combustion.

Most plastic components are provided with a plastic types code and fiber content to aid sorting of plastic parts.

Product	Metal Scrap	Cable Scrap	Electronic Scrap	Plastic Recycling or Combustion
493RPC	X	X	None	X

KEY TO SYMBOLS

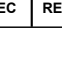The following symbols are used on Hausted product labels.

	Protective Earth		Manufacturer
	Earth Ground		Keep Dry
	General Warning Sign		Fragile, Handle with Care
	CE Mark		Electrical and Electronic Equipment
	ETL		Consult Instructions for Use
	European Authorized Representative		Caution
	Disconnect before Service		Pinch Point
	Medical Device		Unique Device Identifier

www.hausted.com

www.grahamfield.com

Hausted®

www.hausted.com
www.grahamfield.com

by graham-field



Proudly Made in the USA

1.770.368.4700

Manufactured by:
GF Health Products, Inc.
One Graham-Field Way
Atlanta GA 30340-3140

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.hausted.com / www.grahamfield.com

Follow us on   

© 2013, GF Health Products, Inc. All Rights Reserved.
Hausted is a trademark of GF Health Products, Inc.,
One Graham-Field Way, Atlanta GA 30340-3140
GF Health Products, Inc. is an ISO 13485:2016 / MDSAP Certified Company