

# CALIBRATION

**This scale has been factory calibrated, and does not require calibration prior to use.** If required, the scale can be calibrated. Calibration of your scale is performed using kilograms (KG). 20 kilograms total are required for calibrating, applied in 5 kg increments.

**Note: Only weights certified and traceable to national standards should be used for calibration procedures.**

ACTION	DISPLAY SHOWS
With the scale off, press and hold the <b>ZERO/CLEAR</b> as well as the <b>kg/LB</b> button (553KL) or the blank blue button (553KG). While holding down both buttons, momentarily press the <b>ON/OFF</b> button to power on the scale. When "CAL" appears on the display, release all buttons. Numbers will appear indicating that you can calibrate.	"CAL" followed by "XXXXX"
Press the <b>ZERO/CLEAR</b> button. "C-5" will flash on the display and then return to "0."	"C-5" then "0"
Place 5 kg on the scale and when the calibration number stabilizes, press the <b>ZERO/CLEAR</b> button. "C-10" will flash on the display and then the display will return to "0".	"C-10" then "0"
Place an additional 5 kg on the scale for a total of 10 kg. When the calibration number stabilizes, press the <b>ZERO/CLEAR</b> button. "C-15" will flash on the display and then the display will return to "0".	"C-15" then "0"
Place an additional 5 kg on the scale for a total of 15 kg. When the calibration number stabilizes, press the <b>ZERO/CLEAR</b> button. "C-20" will appear on the display, followed by "0"	"C-20" then "0"
Place an additional 5 kg on the scale for a total of 20 kg. When the calibration numbers stabilize, press the <b>ZERO/CLEAR</b> button. "END" will appear on the display and the scale will automatically shut off.	"END"