

A – INSTALLATION INSTRUCTIONS

1. INTRODUCTION

If your Korebalance™ Premiere or Korebalance™ has been installed by an authorized technician, then all of the assembly work described below will already have been done for you. This part of your manual will be for reference only.

2. PACKAGING

The Korebalance™ Premiere and Korebalance™ are shipped in one box:

- Box #1 contains:
 - Item 1. Korebalance™ assembly with shield;
 - Item 2. (4) End caps;
 - Item 3. All-in-one touchscreen computer;
 - Item 4. (4) Computer mounting screws;
 - Item 5. Country-specific power cord;
 - Item 6. Wireless keyboard with trackpad;
 - Item 7. Wireless keyboard receiver;
 - Item 8. Color inkjet printer;
 - Item 9. Printer USB cable;
 - Item 10. Printer power cord;
 - Item 11. Korebalance™ Premiere owner's manual;
 - Item 12. Korebalance™ technical documentation DVD.

3. ASSEMBLY

The Korebalance™ Premiere and Korebalance™ require some assembly.

You will need these tools:

- Phillips head screwdriver;
- Rubber mallet.

Read these instructions completely before you begin assembly:

- a. Remove contents from all boxes;
- b. With the help of another person, place the Korebalance™ assembly (Item 1) in the desired location for use. Allow clearance of approximately 2 feet (0.6m) from a wall or other object;
- c. Position an end cap (Item 2) so that it is aligned with an open tube end and lightly tap it into place with a rubber mallet or the palm of your hand. Repeat for the other three end caps;
- d. Position the all-in-one computer (Item 3) so that its mounting holes are aligned with the mounting holes in the bracket attached to the vertical assembly and secure with the four mounting screws (Item 4);
- e. Connect the upper end of the extension cord to the computer at the position marked “DC Input”;
- f. Connect the upper end of the USB cable to the computer at any open USB port;
- g. If necessary, adjust the leg levelers on the base assembly so that the unit is stable and does not wobble;
- h. Connect the power cord (Item 5) to the receptacle at the front of the base assembly. Note: make sure the on/off switch is in the “off” position before connecting the power cord;
- i. After the unit is fully assembled and all fasteners are tight, connect the power cord to the wall outlet and turn the on/off switch to the “on” position;
- j. Complete the calibration procedure and the Korebalance™ is ready for use!
- k. (Optional Assembly) The wireless keyboard (Item 6) can be installed by following the keyboard manufacturer’s instructions. Connect the keyboard receiver (Item 7) to the computer at any open USB port;
- l. (Optional Assembly) The color printer (Item 8) can be installed by following the printer manufacturer’s instructions. Connect the printer USB cable (Item 9) to the computer at any open USB port. Make sure the printer on/off switch is in the “off” position before connecting the printer power cord (Item 10) to the wall outlet.