

# Maintenance

## Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

## Cleaning

### Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintain instructions are available in your user guide located on the CD that came with your table or referring to Upholstery Care and Maintenance.

### Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces. Use of chlorine based disinfectant can cause damage to the hand control.

## Calling for Service

If service is required, contact your Midmark dealer.  
To contact Midmark directly:  
1-800-Midmark (1-800-643-6275)  
8:00 am until 5:00 pm Mon-Fri (EST)

- **Français** : Référez-vous à CD-Rom



Midmark Corporation  
60 Vista Drive  
Versailles, OH 45380-0286  
Phone: 937-526-3662  
Fax: 937-526-5542  
midmark.com



Because we care.

## Quick Reference Guide

### 630 Barrier-Free® Universal Procedures Table

Model Number:  
Serial Number:

Dealer Info:

Purchase Date:

Upholstery Color:

003-1706-00 Rev. C 10/23/13

This guide outlines basic operation of the table. The enclosed cd provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



© Midmark Corporation 2006

### To adjust the headrest at pivot points A and/or B...

- Pull upper handle outward to release.
- Move pivot point(s) to desired position.
- Push upper handle inward to lock in place.

### Note

The handle tension is adjustable. Contact your service provider if an adjustment is required.



### To adjust the headrest at pivot point C...

- Pull lower handle outward to release.
- Move pivot point to desired position.
- Push lower handle inward to lock in place.

### Note

The Foot Up function is disabled when the treatment pan is pulled out. (You will hear several "beeps" if Foot Up is activated with pan pulled out).

### To resume Foot Up operation...

Push treatment pan all the way in, then press Foot Up button.



Treatment Pan

Brake Pedal

MA729800p

### Rotational Base (optional)

This feature allows you to rotate the table base 350°.

### To rotate the table base...

- Press & release brake pedal.
- Rotate the table to desired position.

### To lock base in position...

Press & release brake pedal.

Note: The base will lock automatically after two minutes.

**To operate the Base / Back / Tilt / Foot functions...**

Press & hold the appropriate button.

**ATTENTION:** The Foot Up function is disabled when the treatment pan (under seat) is pulled out.

Example: The Foot Up button is called out in the illustration.


**Upholstery Heater (optional)**

The upholstery heater warms the seat & back sections of the upholstered top.

**To turn the upholstery heater ON / OFF...**

Press & release the upholstery heater button on the hand control.

(The orange light on the table base will illuminate when the heater is ON).

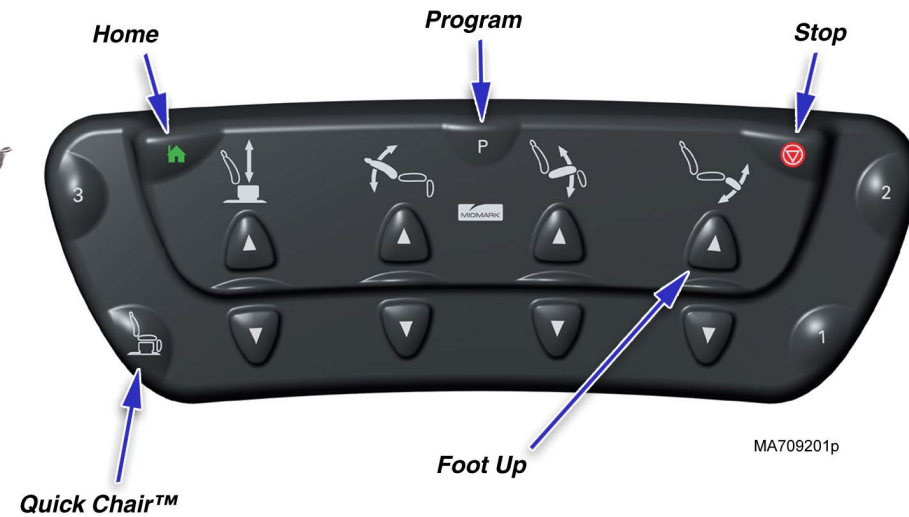

**Programming Table Positions (programmable models only)**

This feature allows you to easily recall frequently used table positions.

**To program a table position...**

- Move the table to desired position.
- Press the Program button. [single "beep"]
- Press the desired Position button (1, 2, 3, or 4). [three "beeps"]

Tip: You must press the Position button within 5 seconds of pressing the Program button.

**To recall a programmed position...**

Press & hold the desired Position button (1, 2, 3, or 4).

**Note**

**To stop movement of any automatic table function...**

Press the Stop button on the hand or foot control.

**Home Function**

This function automatically lowers the base so that your patient may easily mount or dismount the table.

**To lower the base to 23 in. (58 cm)...**

Press, then immediately release the Home button.

**To lower the base to 19 in. (48 cm)...**

Press & hold the Home button until you hear a single "beep".

**Quick Chair™ Function (w/foot control only)**

This function activates the Base Down, Tilt Down, Foot Down, & Back Up functions at the same time.

**To activate the Quick Chair™ function...**

Press & hold the Quick Chair™ button on the foot control.