


USER MANUAL - INTERVENTIONAL SUPINE PILLOW SYSTEM

PRODUCT USE

Primarily designed to facilitate cervical transformational injections, but useful for other antero-lateral procedures such as facet injections, medial branch blocks, stellate ganglion blocks, scalene brachial plexus blocks and discography.

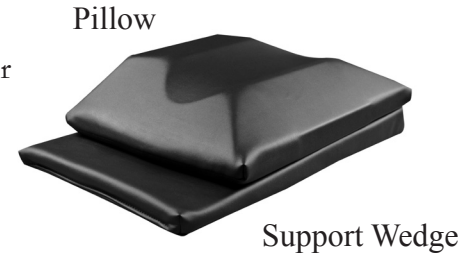

SAFETY INSTRUCTIONS





CAUTION

- Not intended to immobilize the head. Patient may move the head during procedures.
- Maximum Operating Weight: 20 lbs. (9 kg)

DIRECTIONS FOR USE

The **Pillow** is designed to cradle the head and prevent rollback. The beveled surfaces create supportive surfaces for the patient's neck muscles and improve access for procedures.

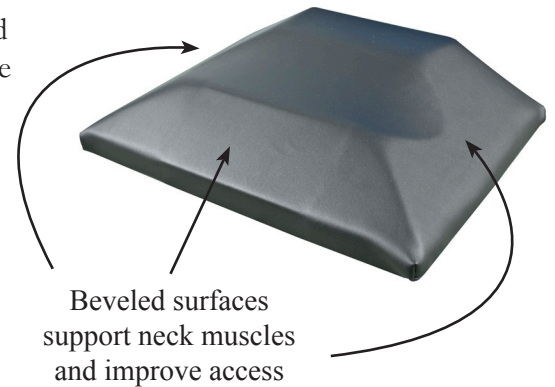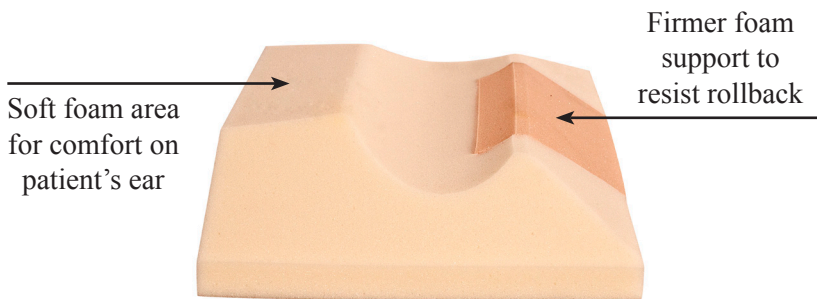

The **Support Wedge** is designed to both raise the head and create a longer transitional area of support under the back for greater relaxation. The thinner end should be located inferiorly and may be moved superiorly or removed.



Positioning options for the system



Increase the rotation by inserting the **Support Wedge**



Raise head & increase the cervical flexion by inserting the thicker end of the **Support Wedge**



Reversible for Left or Right procedures



USER MANUAL - INTERVENTIONAL SUPINE PILLOW SYSTEM CONT...

CLEANING & DISINFECTION

Use appropriate sterilized field materials to isolate the patient from the Interventional Supine Pillow System. This also minimizes cleaning and disinfecting procedures, prolonging the useful life of the system.



Use a 10% sodium hypochlorite (bleach) solution or *Recommended Disinfectants* on all surfaces. Clean all sides of each pad. Follow the directions on the disinfectant and wipe off excess.

Recommended Disinfectants

Protex, MadaCide, Accell TB, Virox®

Note: Damage caused by unapproved substances will not be covered under the warranty.

MODEL NUMBER LOCATION

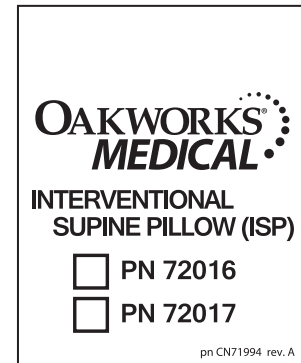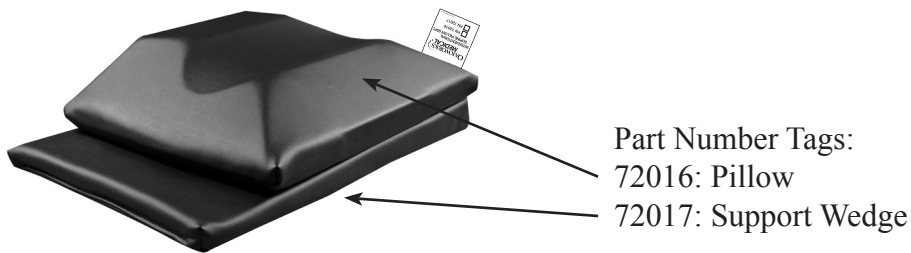


WARRANTY INFORMATION

Use appropriate sterilized field

SPECIFICATIONS

Dimensions (X Y Z)	Pillow - 10.5" x 14" x 3.25" (27cm x 36cm x 8cm) Support Wedge - 10.5" x 16.5" x 1.5" (27cm x 42cm x 4cm)
Maximum Weight Rating	20 lbs. (9 kg)
Radiolucency	Pillow - Max of .70mm AL @ 100 kVp. HVL of 3.6mm Support Wedge - Max of .70mm AL @ 100 kVp. HVL of 3.6mm
Environmental Conditions	Temperature: 50°F (10°C) to 100°F (38°C) Humidity: 20% to 60% RH Atmospheric Pressure: 98 to 105 kPa

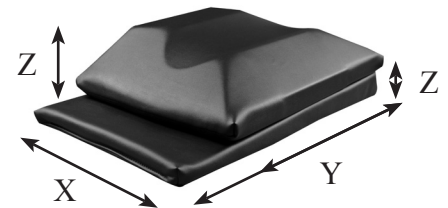

MAINTENANCE & SERVICE

Inspections

- Inspect the pads for areas where ingress of fluids is possible due to damage to the covering. Discard if found.

REPLACEMENT PARTS

- PN 72016: **Pillow**
- PN 72017: **Support Wedge**