


Maintenance

To adjust the headrest at pivot points A and/or B...

- Pull upper handle outward to release.
- Move pivot point(s) to desired position.
- Push upper handle inward to lock in place.

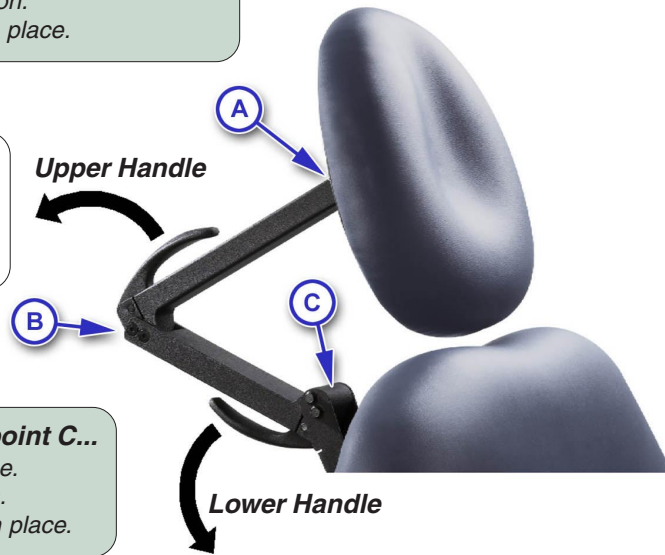

Note

The handle tension is adjustable. Contact your service provider if an adjustment is required.

To adjust the headrest at pivot point C...

- Pull lower handle outward to release.
- Move pivot point to desired position.
- Push lower handle inward to lock in place.

Note

The Foot Up function is disabled when the treatment pan is pulled out. (You will hear several "beeps" if Foot Up is activated with pan pulled out).

To resume Foot Up operation...

Push treatment pan all the way in, then press Foot Up button.



Brake Pedal

MA709300p

Rotational Base (optional)

This feature allows you to rotate the chair base 350°.

To rotate the chair base...

- Press & release brake pedal.
- Rotate the chair to desired position.

To lock base in position...

Press & release brake pedal.

Note: The base will lock automatically after two minutes.

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces. Use of chlorine based disinfectant can cause damage to the hand control.

Calling for Service

If service is required, contact your Midmark dealer. To contact Midmark directly:
1-800-Midmark (1-800-643-6275)
8:00 am until 5:00 pm Mon-Fri (EST)



Français : Référez-vous à www.midmark.com

Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
Phone: 937-526-3662
Fax: 937-526-5542
midmark.com



Because we care.

Quick Reference Guide

230

Universal Procedures Chair

Model Number:

Serial Number:

Dealer Info:

Purchase Date:

Upholstery Color:

003-1705-00 Rev. EA1 10/12/18

This guide outlines basic operation of the chair. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



To operate the Base / Back / Tilt / Foot functions...
Press & hold the appropriate button on the hand control.

Note: The Foot Up function is disabled when the treatment pan (under seat) is pulled out.

