
INSTRUCTIONS

EN

English

1. **Select** the area for the placement of the Grip-Lok.
2. **Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
3. **Remove** right and left release liners from one side. Do not remove the small release liner from the center.
4. **Place** the Grip-Lok with the exposed adhesive side onto skin surface with the insertion site above the center.
5. **Remove** right and left release liners from the top side. Do not remove the small release liner from the center.
6. **Insert** the catheter and fold wings down to the exposed adhesive.
7. **Wrap** one side of the exposed adhesive over the wing and hub then the other side.
8. **Press** adhesive around the catheter to conform to the hub and wing shapes.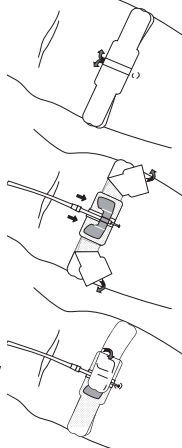

Direction: for added skin protection, skin prep may be used prior to application of securement device.

Note: Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

Note: Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.

Note: Use of an alcohol swab may help removal of securement device from the skin.

Note: Hypoallergenic.

2020-11-01

TIDI® Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA

Phone: +1 920 751 4300
tidiproducts.com/securement

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Grip-Lok is a registered trademark of TIDI Products, LLC.

Grip-Lok®

Dialysis Fistula Needle
Securement Device

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