
INSTRUCTIONS EN
English

1. Select the correct size Grip-Lok for your application.

REF	Size	Line/Tubing Diameter
32XXS-NSB	Small	4.5 - 13.5 Fr
33XXM-NSB	Medium	9 - 24 Fr
34XXL-NSB	Large	16 - 40 Fr

2. Select the area for the placement of the Grip-Lok. Note: The line or tube should be already inserted in the patient.
3. Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
4. Open the top flap, remove the interior liner and hold the flap open.
5. Slide the Grip-Lok under so that the line or tube is in the center of the exposed adhesive.
6. Secure the top flap over the line or tube and apply gentle pressure to the center fabric area over the adhesive.
7. Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
Direction: for added skin protection, use skin prep pad prior to application of stabilization device.
Note: Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.
Note: Use of an alcohol swab may help removal of securement device from the skin.
Note: re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.
Note: CAUTION: Intended for further processing.
Note: Hypoallergenic.

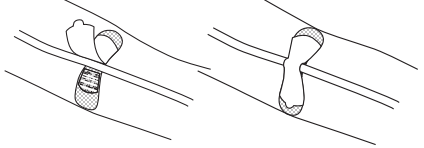2020-11-01

TIDI® Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA
Grip-Lok is a registered trademark of TIDI Products, LLC.
Phone: +1 920 751 4300
tidiproducs.com/securement

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