



Posey® Seizure Side Rail Pads

Application Instructions

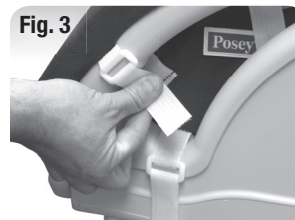

Posey Seizure Pads are form fitted to fit snugly over the existing bed side rails with a 1" (3 cm) layer of medium density foam to help safeguard against accidental patient injury. The outer shell is constructed from lightweight sheer nylon denier to help prevent patient "rub" burns and allows unobstructed movement of the side rails to the lowered position. The nylon cover is backed with a urethane waterproof coating that easily wipes clean with liquid disinfectant. Open windows in upper section allow for easy access to controls on most bed models. Navy Blue.

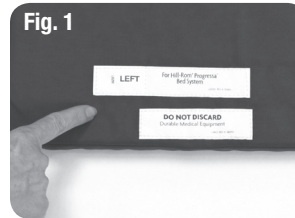

Cat. No. Description Pads

Hill-Rom® Beds

| | | |
|-----------------|---|---|
| REF 5725 | Fits Advanta™ and Century Plus® beds | 4 |
| REF 5729 | Fits Advanta™ (Pre- 02/2004), Advance®, Century 8500, and Century CC Beds | 4 |
| REF 5742 | Fits CareAssist® and Advanta™ 2 Beds | 4 |
| REF 5743 | Fits VersaCare® Med / Surg Beds | 4 |
| REF 5744 | Fits Affinity III® and IV® Maternity Beds | 2 |
| REF 5746 | Fits TotalCare® Med / Surg Beds | 4 |
| REF 5747 | Fits Resident® LTC Beds | 2 |
| REF 5756 | Fits Progressa® Beds | 4 |
| REF 5760 | Fits Hill-Rom Low® Beds | 2 |
| REF 5761 | Fits Affinity II® Maternity Beds | 2 |

Stryker® Beds

| | | |
|-----------------|--|---|
| REF 5755 | Fits Secure I and II® Beds with Rounded Side Rails | 4 |
| REF 5762 | Fits Secure I and II® Beds with Rectangular Side Rails | 4 |
| REF 5763 | Fits GO Bed II® with Rectangular Side Rails | 4 |
| REF 5764 | Fits In-Touch™ Critical Care Beds | 4 |
| REF 5766 | Fits S3™ Beds | 4 |

Other Manufacturers

| | | |
|-----------------|--|---|
| REF 5760 | Fits Carroll® Beds with Care Assist Arms | 2 |
| REF 5765 | Fits Med-Mizer® Beds with Med Support Arms | 2 |

Monitoring

⚠WARNING Posey Seizure Side Rail Pads will not prevent a determined patient from exiting the bed, going over or attempting to go between or through side rail gaps. Not all hospital beds allow for full articulation when this product is applied. Slowly articulate the head of the bed to ensure compatibility. Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

Application Instructions:

1. Inspect product set prior to use. Do not use soiled or damaged products.
2. Identify each of the components in the set. Each pad has sewn-in tags, identifying its intended location from the perspective of lying down in the bed (Fig. 1).
3. Place each of the pads onto the proper side rail and shift it into position (Fig. 2).

4. Securely fasten all the hook-and-loop fasteners along the edges of each pad. Some pads may have additional hook-and-loop fasteners in their mid-sections, for those pads please see Fig. 3 and Fig. 4.
5. Lower each of the side rails to check clearances. Note that there may be some resistance as the rails are completely lowered. This should be expected as there are close manufacturing tolerances between the bed and the side rail. The shear nylon covers are chosen to minimize any potential binding.
6. Check for proper fit. Modified beds may not be compatible with Posey Seizure Side Rail Pads.

Disposal

⚠WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

Cleaning Instructions:

Wipe clean with soap and water, or disinfectants that DO NOT contain the chemicals Phenol or Benzyl. DO NOT submerge pads under any liquid solution.



Advance, Advanta, Affinity, CareAssist, Century Plus, Progressa, Resident, TotalCare and VersaCare, are registered trademarks of Hill-Rom® Services, Inc. Med-Mizer is a registered trademark of Med-Mizer Inc. Go-Bed II, In-Touch, Secure, S3 and Stryker are registered trademarks of the Stryker® Corporation.