

STEP 1:

Prepare alarm:

- Install batteries if using batteries as primary or backup source of power.
- If using the optional AC Adapter (sold separately), connect the AC adapter to the On Cue[®] PRO, then plug the other end into the wall.
 - DO NOT plug the AC Adapter into the wall outlet in advance of plugging into the Alarm.
 - DO NOT allow the On Cue PRO to remain in the OFF position while plugged into the AC Adapter.
 - Unplug the AC Adapter from the wall outlet when not in use.
- Immediately turn on the alarm.



Note: Turning off the alarm unpairs the wireless chair sensor pad. **Never** turn off the wireless nurse call adapter or it will unpair from the alarm.

STEP 2:

Prepare Nurse Call Adapter:

- Install batteries if using batteries as primary or backup source of power.
- If using the optional AC Adapter (sold separately), connect the AC adapter to the Nurse Call Adapter then plug the other end into the wall.
- Immediately turn on the Nurse Call Adapter
- Tap to pair the wireless nurse call adapter to the alarm.



STEP 3:

Verify the wireless nurse call adapter has a blue light, showing that it is paired to the alarm.

- Blank:** No power (or batteries dead)
 Blue: Paired
 Yellow: Powered on, not paired



STEP 4:


Tap to pair the wireless chair sensor pad to the alarm.



STEP 5:

Verify the wireless chair sensor pad is paired by seeing the blue light on the alarm.

- Blank:** Not paired
 Blue: Paired

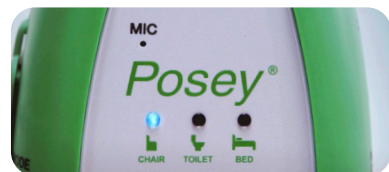

STEP 6:

If the alarm has a green light, then it is actively monitoring your patient.



Status Light Codes: Alarm

- Red (Hold):** Monitoring paused for 30 seconds
 Yellow (Ready): Sensor will begin monitoring once pressure applied
 Green (Active Monitoring): Sensor is monitoring