














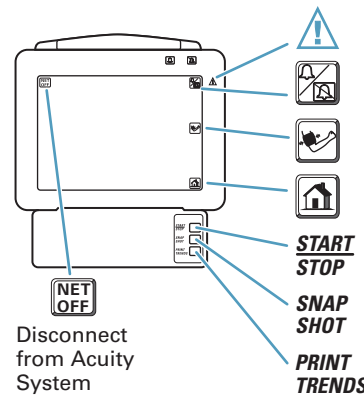
- Change patient mode \_\_\_\_\_  SETUP, MORE, CHANGE
- Change ECG1 lead \_\_\_\_\_  ECG, ECG1 LEAD
- Change RESP lead \_\_\_\_\_  ECG/RESP, MORE, (NEXT, CHANGE)
- Specify auto or manual NIBP \_\_\_\_\_  NIBP, AUTO/MANUAL
  
- Specify waveforms for display \_\_\_\_\_  SETUP, MORE, WAVE SELECT, (NEXT, ON/OFF)
- Change HR/PR tone volume \_\_\_\_\_  SETUP, MORE, (NEXT, CHANGE)
- View vital sign trends \_\_\_\_\_  SETUP, TRENDS, (NEXT TREND)
- Change time/date \_\_\_\_\_  SETUP, MORE, MORE (NEXT, UP or DOWN)

### Alarms and Alerts

- Silence alarm or alert tone for 90 sec or until a new alarm or alert occurs \_\_\_\_\_  or SILENCE
- Inhibit alarm and alert tones for 4 min \_\_\_\_\_  SETUP, ALARMS, 4 MIN SUSPND
- Re-enable a silenced alarm or alert tone \_\_\_\_\_  or RESUME
- Change alarm tone volume \_\_\_\_\_  SETUP, MORE, (NEXT, CHANGE)
- Change alarm limits \_\_\_\_\_  SETUP, ALARMS, LIMITS



## Propaq® CS Vital Signs Monitor



- Consult Directions for Use
- Silence/Re-enable a patient alarm or equipment alert
- Start/Stop an NIBP reading
- Main Menu
- Start/Stop printing
- Print previous eight seconds
- Print specified trends

Disconnect from Acuity System

©2006 · Printed in USA · P/N: 810-1103-02 Rev. A 6/05

800 289 2500 · 503 530 7500 · Beaverton, Oregon USA

