



**GAMMA COMPATIBLE
MATERIALS
REFERENCE GUIDE**

Gamma Compatible Materials

Radiation Stability of Selected Medical Grade Polymers

MATERIAL	TOLERANCE LEVEL (KGY)	COMMENTS
Thermoplastics		
Acrylonitrile/Butadiene/Styrene (ABS)	1,000	Protected by Benzene ring structure. Avoid high dose on high impact grades.
Aromatic Polyesters (PET, PETG)	1,000	Very stable, retains excellent clarity. Drying is essential. Good in luer connectors.
Cellulosics		
Esters and Ethers	100	Paper and natural fibers scission, discolor and embrittle.
Paper, Card, Corrugated, Fibers	100-200	Retains good clarity and impact.
Cellulose Acetate Propionate and Butyrate	100	
Fluoropolymers		
Tetrafluoroethylene (PTFE)	5	Liberates fluorine gas, disintegrates to powder. Avoid use.
Polychlorotrifluoroethylene (ECTFE)	200	
Polyvinyl Fluoride	1,000	
Polyvinylidene Fluoride (PVDF)	1,000	
Ethylene-Tetrafluoroethylene (ETFE)	1,000	
Fluorinated Ethylene Propylene (FEP)	50	Avoid use.
Polyacetals (Delrin, Celcon)	5	Avoid use due to embrittlement.
Polyacrylics		
Polymethylmethacrylate	100	Yellows at 20-40 kGy; clarity recovers partially on aging.
Polyacrylonitrile	100	Yellows at 20-40 kGy.
Polyacrylate	100	Yellows at 20-40 kGy.
Polycyanoacrylate	100	Adhesives function at 100 kGy with less than 30% degradation.
Polyamides (Nylons)		
Aliphatic & Amorphous Grades	50	Discolours. Avoid thin films and fibers. Dry before molding.
Aromatic Polyamide/Polyimide	10,000	High heat/strength grade.
Polycarbonate	1,000	Discolours, clarity recovers after aging. Dry before molding.
Polyethylene (LDPE, LLDPE, HDPE, UHMPE, UHMWPE)	1,000	Crosslinks to gain strength, loses some elongation. All polyethylene radiation stable, low density most resistant.

GAMMA COMPATIBLE MATERIALS REFERENCE GUIDE

MATERIAL	TOLERANCE LEVEL (KGY)	COMMENTS
Polyimides	10,000	
Polymethylpentene	20	Subject to oxidation degradation. Avoid use.
Polyphenylene Sulfide	1,000	
Polypropylene, Radiation Stabilized		
Homopolymer	20-50	Subject to orientation embrittlement. Validate with real time aging.
Copolymers of Propylene-Ethylen	25-60	More stable than Homopolymer.
Polypropylene, natural	20	Avoid use of unstabilized polypropylene.
Polystyrene	10,000	All styrenes are stabilized by Benzene ring structure.
Polysulfone	10,000	Amber colour before irradiation.
Polyurethane	10,000	Excellent clarity and chemical resistance to stress-cracking. Drying is essential.
Polyvinylbutyral	100	Yellows.
Polyvinylchloride (PVC)	100	Yellows, can be tinted for colour correction.
Polyvinylidene Chloride (PVDC)	100	Yellows, releases HCL.
Styrene/Acrylonitrile (SAN)	1,000	Yellows at 40 kGy.
Thermosets		
Allyl Diglycol Carbonate (Polyester)	5,000-10,000	All thermosets as a class are highly resistant.
Epoxies	1,000	Many good formulations available. Success depends on joint design and application process.
Phenolics	50,000	
Polyesters	100,000	
Polyurethanes	100-1,000	Wide formulation variations for urethanes.
Elastomers*		
Butyl	50	All thermosets as a class are highly resistant.
Ethylene-Propylene Diene Monomer (EPDM)	100-200	Crosslinks, yellows slightly.
Fluoro Elastomer	50	Avoid multiple sterilization.
Natural Rubber (Isoprene)	100	Very stable with sulfur or resin cure systems.
Nitrile	200	Avoid multiple sterilization.
Polyacrylic	50-200	Avoid multiple sterilization.
Polychloroprene (Neoprene)	200	Avoid multiple sterilization.
Silicones (Peroxide & Platinum Catalyst Systems)	50-100	Crosslink density increases more in peroxide systems than in platinum systems.
Styrene-Butadiene	100	Avoid multiple sterilization.
Urethanes	100-200	Wide variations in urethane chemistry applied to medical devices.

*Elastomers: 1) Radiation tolerance is affected by the base polymer and the curing system used. Sulfur and resin cures are more durable.

2) All elastomers are subject to cross-linking. Avoid folds, coils, curves in the shape packaged. Typical sterilization processing dose: 20-50 kGy (2.0-5.0 Mrads).



Our Innovation. Your Opportunity.

Foxx Life Sciences is a world leader in Class 7 cleanroom contract manufacturing. With 10 years of ISO 13485 certification, five cleanrooms across three locations, a triple inspection program, a fast-track program, FDA registration, and the shortest leads in the industry, Foxx Life Sciences is the perfect partner for your next custom medical assembly. From design to development to manufacturing, you can trust Foxx to deliver! Locations are in Salem, NH; Londonderry, NH; and Hyderabad, India.

www.ShopFLS.com



CORPORATE HEADQUARTERS

12 N. Wentworth Ave.
Londonderry, NH, USA 03053
Tel: (603) 890-FOXX (3699)

6 Delaware Drive
Salem, NH, USA 03079
Tel: (603) 890-FOXX (3699)

ASIA HEADQUARTERS

SN-65, Rajabolaram Village
Medchal, Telangana, India, 501401
Tel: +91 (8418)- 661210 / 15