



Ultra Pulse

Talking Ear and Forehead Thermometer with Fingertip

Pulse Meter

Model No.: 70203



MEASUREMENT PREPARATIONS

- For proper measurements always store the device in same area being used for at least 30 Minutes before use. An unexpected fluctuation of ambient temperature is likely to lower measuring accuracy.
- Keep ear clean since an unclean ear canal may cause inaccuracy.
- Do not talk or move body during measurement.
- Temperature is likely to increase immediately after exercise, bathing, or other physical exertion.
 - If measurement appears to be incorrect please wait 5-15 minutes before taking measurement again.
- Make sure to use same finger when re-measuring Heart Rate.
- Replace battery when low battery indicator is On.

LED Mini Flashlight

- Press and hold the center of the SET and TALK buttons for 2 seconds and the LED Mini Flashlight will turn ON. It will turn OFF automatically after 5 seconds.

Ear Temperature Mode Operation

- Pull off the forehead probe cover from the unit, and insert the slimmer Ear Probe tip just slightly into the ear opening, then press the scan button to measure the body temperature.
- Once you hear two short “beeps”, the display will show the body temperature in large numbers either in °F or °C depending on the setting.

Forehead Temperature Mode Operation

- Make sure the Forehead probe cover is on the unit, then gently touch the entire tip of the probe to the center of the forehead, midway between the eyebrow and the hairline, then press the SCAN button to measure the body temperature.
- Once you hear two short “beeps”, the body temperature will be shown in large numbers either in °F or °C... depending on the settings.

Note: In case of losing Forehead probe cover, could be purchased from our website at: getmobi.com/accessories

Display of Measurement Results

- **Temperature Range:**
 - Results shown from 89.6 °F to 109.3 °F (32.0 °C to 42.9 °C). “HI” is displayed for all results above 109.3 °F (42.9°C), and “LO” displayed for result below 89.6 °F (32.0°C).
- **Fever Diagnosed:**
 - A Red LED light with results from 100.4°F (32.0°C). to 109.3 °F (42.9°C).
- **No Fever:**
 - A Green LED light with results from 96.0 °F (35.6°C) to 100.3 °F (37.9 °C).


Note:

- Thirty Seconds after a measurement the unit automatically returns to Clock Mode and Room temperature.
- In Sleep Mode the display will automatically turn off one minute after the last button is pressed.

Pulse /Heart Rate Mode Operation

To measure your Pulse:


- With your index finger centered on the sensor, press and release the Heart Rate Button once to activate feature. Rest finger lightly on the sensor until unit beeps and the Pulse count will appear on the display.
- As you let your finger rest gently for 10 – 15 seconds, avoid any movement or talking for better accuracy. The large Heart Symbol on the display will begin to flash in sync with your heart beats.

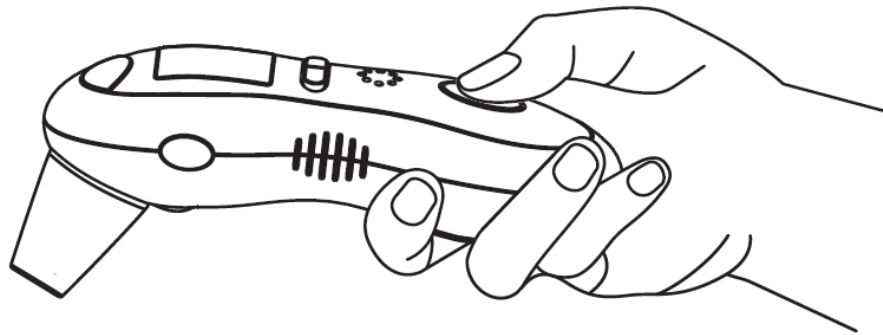

Note: The Heart Rate is usually read within 15 seconds if the Large Heart symbol is flashing. If it remains constantly ON (not flashing) it means the connection between your finger and the sensor is not optimal and will result in an ERR (error) message on the display indicating the Heart Rate was not measured. In that case press and release the button again to restart the reading and be sure to leave your finger resting on the button light enough to keep it from being depressed again.

Pressing the SET button – Allows you to go again to the display feature so you can repeat Pulse Reading process again.

Installing/Replacing batteries

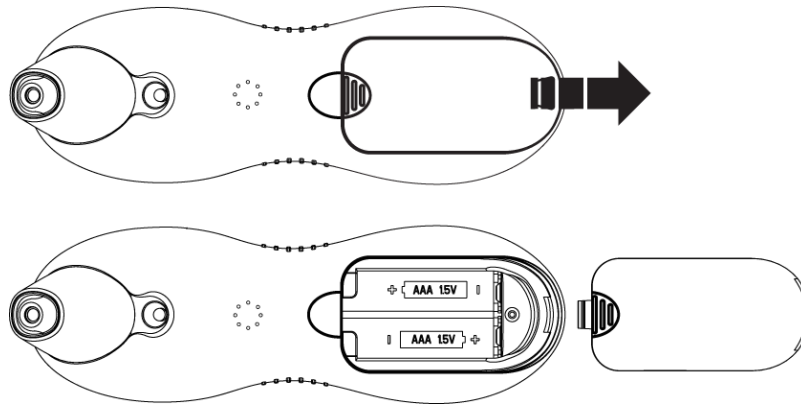

- 1- Put batteries into compartment according to the + and – symbols inside and use only new batteries (2 x AAA – Alkaline recommended).
- 2- Replace the battery compartment door tightly.

Settings

Clock Mode:


The display will show Time & Date as: Year, Month, Date, Hour and Minute with “:” between the hour and minute flashing each second. At rest, the LCD screen displays the current Time and Room Temperature, alternating every 5 seconds. Otherwise Press and release TALK and the unit will visibly display and audibly speak, the current Time and Room Temperature in 3 Languages “English – Spanish – French”.

Language Setting:


To change the language, Press and hold the TALK button for 3 seconds and you will toggle between Languages and the symbol “ - - ” will silent the Language mode.

Memory:

- 1- Press SET button to retrieve the most recent reading recorded in Memory. Starting with the highest number It will show the Body Temperature, the Type of Reading (Ear or Forehead), the Fever/No Fever colored light, the Date it was recorded and the Memory number

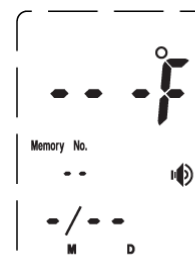

Memory Mode
(no previous recorded readings in memory)

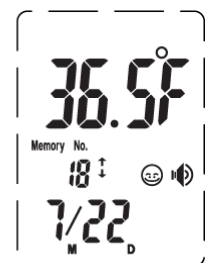

Memory Mode
(No.18 of previous recorded readings)

Time and Date:

- 1- Press and hold the SET button for 2 seconds. The word SET will appear on the display. Then use the TALK button to select the Year, Month, Hour, Minute, etc.
- 2- Use the SET button change the date/time values. To exit settings, use SCAN button.



Fahrenheit or Celsius:

- 1- Press the SCAN button and TALK button together. The unit will cycle between Fahrenheit and Celsius modes.
- 2- This setting change will be spoken if the Talking Feature has not been disabled.



Limited Product Warranty

MOBI® warrants its products to be free from defects in material and workmanship for period of (1 Year) from the date of purchase. Should the product prove to be defective at any time during the warranty period, we would either replace or repair the product at our option without charge. After the warranty period, a service charge will apply for replacement of parts and labor costs. To obtain warranty service, you must return the product complete with its original parts & accessories along with a copy of the original purchase, gift or registry receipt to us. This warranty does not cover damage caused by any tampering, accident, misuse, abuse, improper maintenance, unauthorized modification, or connection to an improper power supply. There will be a charge for repair caused for such damage. This warranty excludes all incidental or consequential damages and any liability other than stated above.

Questions?

Please visit: www.ProductsByMobi.com

Email: service@getmobi.com

Fax: 424-672-6628