

Warnings and Instructions for Use

- Enclosed Face Masks are NOT Intended to Provide Liquid Barrier Protection
- This mask is not a surgical device or a respirator and should not be used as a substitute for either of these devices.
- Not for use in any surgical setting or where significant exposure to liquid, bodily or other hazardous fluids, may be expected.
- Not for use in a clinical setting where the infection risk level through inhalation exposure is high.
- Not for use in the presence of high intensity heat source or flammable gas.
- 65% Polyester/35% Cotton Note: These masks are not made of medical grade material
- For adult use only. Machine wash (hot) and machine dry (high heat) these masks prior to first wear. Wash routinely depending on the frequency of use. Do not microwave. Do not put in the oven.
- Use of this mask is intended to be an addition to and not in substitution of all other measures recommended for the prevention of communicable diseases.
- If unsure, consult your personal physician before wearing.
- When washed and dried, secure the clean mask to your face to cover both your nose and mouth.



Secure mask with earloops,
fully covering nose and mouth



If using optional nose piece,
adjust to desired snugness

WARNING: Gaps between the mask and your face, or a loose fitting mask, will reduce the effectiveness of the mask. Once secured, refrain from touching the mask and your face in order to adjust its fit. Any hand contact with your face, or the mask itself once donned, can increase your risk of contracting or communicating an infectious disease.

These guidelines are based on this FDA document. The guidelines may be subject to change. Please refer to the FDA website for the most up-to-date information.

<https://www.fda.gov/regulatory-information/search-fda-guidance-documents/enforcement-policy-face-masks-and-respirators-during-coronavirus-disease-covid-19-public-health>