

Walker Holder

B6214

DIESTCO Walker Bag mounts to the back of your scooter to carry your basic style walker. Made from high quality, water resistant polyester.

Fits seatbacks 7" - 24" tall.

Mounting Instructions:

1. Remove the strap from the metal loop and pass it through the opening between the seat and the seatback. Be sure to pull the strap through completely and that it is not twisted.
2. Re-insert the strap through the metal loop and pull through.
3. Pull strap down tightly and fasten the loop end to the hook portion of the strap. Excess strap can be cut once you have your Walker Holder is mounted.
4. Place walker in the pocket, handles down.
5. Pull velcro strap at the top of the pocket through the metal ring, pull tight and secure.
6. Wrap the velcro strap (with 300-yard reflective tape) around the walker's legs to secure it in place.

If you have any issues mounting the holder, please call us at (800)795-2392 for assistance.



DIESTCO Mfg. Corp.
P.O. Box 6504
Chico, CA 95927
(800) 795-2392
www.diestco.com

We strongly encourage you to call us at (800)795-2392 if you have any problems, questions, or comments. We are always developing new products for wheelchairs, scooters, crutch, and walker users...so let us know if there's something special you need.