

# Oxygen Tank Holder

B6211

DIESTCO Oxygen Tank Holder mounts to the back of your scooter or powerchair with a 2" hook and loop strap. Made from high quality, water-resistant polyester with 300-yard visible reflective safety tape.

Fits seatbacks 7" - 24" tall.

## Mounting Instructions:

- 1) Remove the strap from the metal loop.
- 2) Pass the strap through the opening between the seat and the seatback.  
Be sure to pull the strap through completely and that it is not twisted.
- 3) Re-insert the strap through the metal loop and pull through.
- 4) Pull strap down tightly and fasten the loop end to the hook portion of the strap.
- 5) Excess strap can be cut once you have your Oxygen Tank Holder mounted.

**If you have any issues mounting the holder, please call us at (800)795-2392 for assistance.**



DIESTCO Mfg. Corp.  
P.O. Box 6504  
Chico, CA 95927  
(800) 795-2392  
[www.diestco.com](http://www.diestco.com)

We strongly encourage you to call us at (800)795-2392 if you have any problems, questions, or comments. We are always developing new products for wheelchairs, scooters, crutch, and walker users...so let us know if there's something special you need.