

Instructions for installing SnapIt! System™ Mounting Arm on flange type Invacare® Powerchair Armrests.

- 1. Remove screws from SnapIt!™ mounting arm.**
- 2. Raise armrest.**



3. Locate mounting holes in armrest.

4. Slide SnapIt!™ arm between flanges.

5. Line up screw holes between flange and arm.

6. Screw supplied screws through hole in flange into matching holes in SnapIt!™ mounting arm. 2 screws from one side and 1 screw from the other.

7. Use #2 Philips screwdriver. Hand tighten.

8. Insert SnapIt!™ accessory into the clip on the mounting arm.

Enjoy!