

Anti-Snoring Noseclip

User Manual



DARTWOOD

a WASSERSTEIN brand

Product Overview

The Dartwood Anti-Snoring Noseclip is a BPA free product. It is a simple and effective solution that can help reduce snoring and improve sleep quality. It is designed to gently open nasal passages, providing better airflow that can reduce snoring.

In the Box




6 x anti-snoring
noseclips



6 x storage cases

How to Use


1. Thoroughly wash your hands with soap and water before handling the Dartwood Anti-Snoring Noseclip.
2. Gently pull the flexible arms open.
3. Carefully fit the Noseclip into your nostrils, with one Noseclip pad in each nostril.

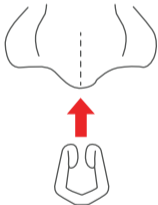

4. Adjust the Noseclip to rest comfortably and securely up against your nose septum.



5. To remove the Noseclip, gently pull the Noseclip arms open and pull them away from your nostrils.
6. Be sure to wash the Noseclip after each use, and dry them thoroughly with a soft cloth.
7. Store the nose clips: Store the nose clips in the provided storage case to keep them clean and safe when not in use.

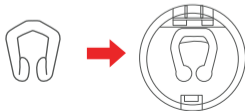

Important Safety Information

- Do not use the Noseclip if you have a medical condition that affects your nasal passages or breathing. Consult with your doctor before using the Noseclip if you have any concerns.
- If you experience any discomfort or irritation while using the Noseclip, immediately discontinue use and consult with your doctor.
- Do not use the Dartwood Anti-Snoring Noseclips if you have allergies to the materials in the Noseclips.
- Keep the the Dartwood Anti-Snoring Noseclips out of reach of children and pets, as they can be a choking hazard.
- To prevent the spread of germs and infections, do not share your Dartwood Anti-Snoring Noseclips with others.

- Clean the Dartwood Anti-Snoring Noseclips after each use, and dry them thoroughly with a soft cloth. Do not use harsh chemicals or abrasive materials to clean the Noseclips.
- Put the snoring device 5 to 15 minutes before bedtime, and wear it all night in order to get the best result.
- A small number of users may feel uncomfortable at the beginning of wearing, The discomfort will gradually disappear in about 3 to 5 days. If the discomfort continues, please stop using it and consult with your doctor if necessary.

Cleaning Instructions

To maintain proper hygiene and ensure the longevity of your Dartwood Anti-Snoring Noseclips, we recommend cleaning them after each use. To clean, simply wash the Noseclip, and then thoroughly dry your Noseclip with a soft cloth.

Specifications

Dimensions: 0.78 x 0.55 x 0.2 in

Certification: BPA free

Material: Soft plastic

Get More Support

Scan the QR code below or email us at contact@wasserstein-home.com



CONTACT US

DW DARTWOOD
a WASSERSTEIN brand

www.wasserstein-home.com

Made in China