

Operation of Controls



Power – Turns the main power to the table on or off.



Vibration – Turns the vibration mode on or off.



Heat – (Option available on Model 400H)
(See page 5)



Direction – Determines the counter-clockwise or clockwise direction of the roller when in motion. Standard position is counter-clockwise rotation.



Lock – Prevents patients from changing the table settings. To unlock, hold in button for three seconds.



Treatment Time Indicator



Start/Stop – Push once to start selected therapy. Give to the patient to use as a remote stop.




Travel – Controls the tracking of the rollers from Cervical to Lumbar. The ON position is when the button is lighted, which allows the rollers to track the full range. The OFF position is when the button is not lighted, halts the horizontal movement of the rollers, and allows treatment to take place in a selected area.



Timer/ Roller Height Control – Timer may be set from 1 to 30 minutes by using the arrow buttons. Toggle up or down to set the height of the rollers



when the timer is active. Rollers will retract to lowest position when time expires.

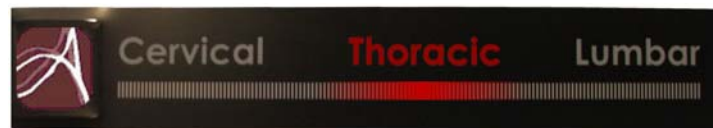

Roller location indicator panel –

Red light behind Cervical – Thoracic – Lumbar panel indicates horizontal position of rollers.



NOTE:

Remote control automatically defaults to last settings used