

## Installation Instructions – SKF STF01-LU1000-X100 / Armedica 03106



Medical Equipment - The Physical Medicine Sourcing Center

Phone: (888) 493-CORE <https://coremedicalequipment.com>

1. **IMPORTANT:** Remove the power connection from the actuator motor to the wall outlet. Electrical shock could occur if not unplugged.
2. Gently pull the old cable out, while slightly wiggling it up and down, as it is released from the plastic clips on the actuator. Be careful not to break the plastic clips on the actuator.
3. Make sure the arrows on the new cable are on the top, going into the actuator connector. The cable will only go in one way. Don't try to force it the other way.



*Put in the cable into the plug in the correct way  
(left: three arrows on the top have to be in line); otherwise the plug in the device will damaged*

Gently push the new foot switch cable end into the actuator connector until it snaps in place by the plastic clips on the actuator base. Again, be careful not to break the clips.

4. Plug the power cable from the actuator motor back into the wall outlet. Test the foot switch pedals to insure proper operation.

This should be a quick replacement. But, if you do not feel comfortable doing this, please consult a qualified maintenance person or electrician for assistance.