



# RECREATEDDESIGNS

**CONTACT** CAROL LYNN LAPOTKA  
*innovator & head recreator*

**WEBSITE** [recreatedesigns.com](http://recreatedesigns.com)

**EMAIL** [info@recreatedesigns.com](mailto:info@recreatedesigns.com)

**PHONE** 406.214.9078

**LOCATION** 212 Main Street, Polson, MT

CLOTHING  
ACCESSORIES  
HOME GOODS

HANDCRAFTED  
IN MONTANA  
FROM UPCYCLED  
MATERIALS



**WEBSITE:** <https://www.recreatedesigns.com/pages/covid-19-mask-making-mission>

**PATTERN DOWNLOAD:** <https://tinyurl.com/sspq496>

**YOUTUBE TUTORIAL:** <https://youtu.be/7cktX379IGg>

**MASK MAKING MISSION FORM:** <https://tinyurl.com/wxwwe79>

**MASK REQUEST FORM:** <https://tinyurl.com/sjy974q>

**FACEBOOK:** <https://www.facebook.com/REcreatedesigns/>

Clean (freshly washed) 100% cotton can be brought to our shop at 212 Main Street, Polson, MT.

## STAY SAFE. STAY HOME. HELP IF YOU CAN.

## MASK MAKING MISSION : FABRIC WITH INSERT POCKET

### ADULT SUPPLY LIST


FABRIC: 15" X 8.5"  
 BINDING STRIP: 2 - 2" X 4.25"  
 PLASTIC TWISTIE(S) 6"  
 ELASTIC:  
 OVER THE HEAD: 30" (1/8" OR 1/4")  
 BEHIND THE EAR: 2- 13" (1/8" OR 1/4")








### CHILD SUPPLY LIST

FABRIC: 13" X 6.5"  
 BINDING STRIP: 2 - 2" X 3.5"  
 ELASTIC:  
 OVER THE HEAD: 25" (1/8" OR 1/4")  
 BEHIND THE EAR: 2- 10" (1/8" OR 1/4")


This basic pattern works for both an adult or child mask using different sizes of fabric and elastic. Please start with clean washed fabric and do not use fabric softeners or dryer sheets. The pocket created can be used with an insert or a N95 mask. Around the head and back of the neck elastic is a more secure fit and less painful on the ears. These masks alone will not prevent individuals from getting the Covid-19. ALWAYS follow proper precautions (hand washing, social distancing, etc) and adhere to the CDC guidelines for PPE. Pattern adapted from several online versions. Feel free to share.

**WEBSITE:** <https://www.recreatedesigns.com/pages/covid-19-mask-making-mission>

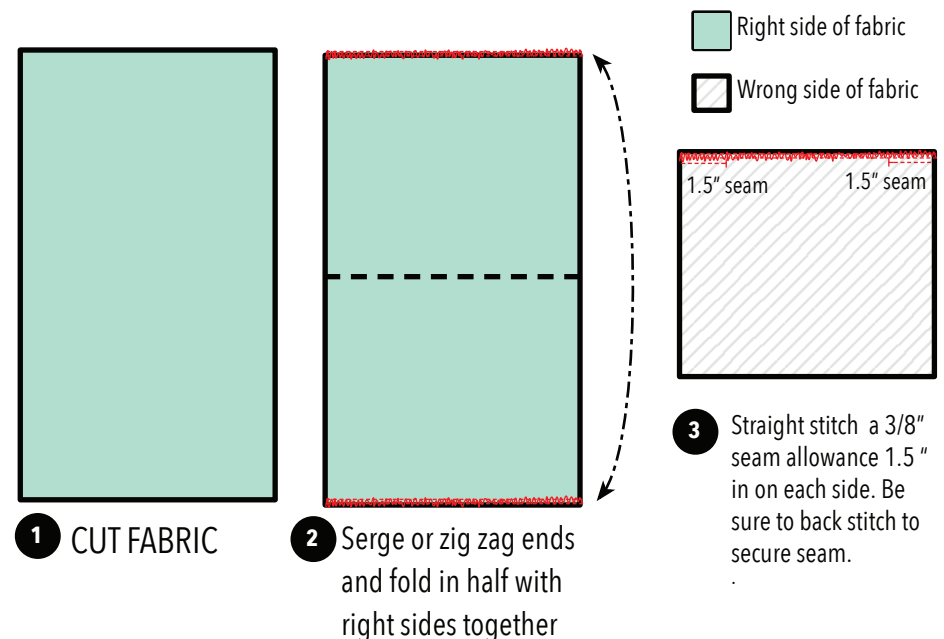


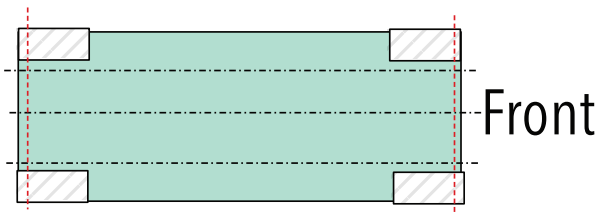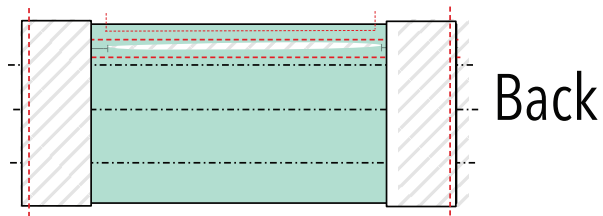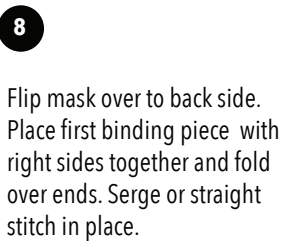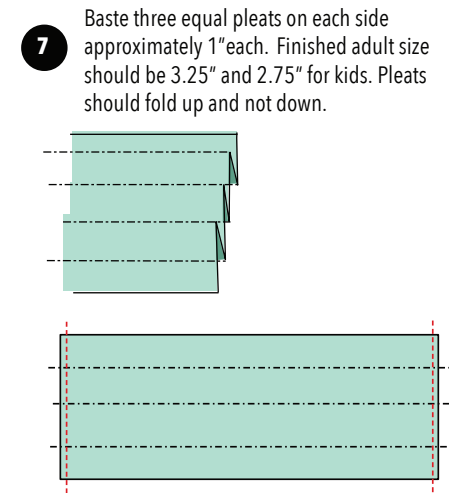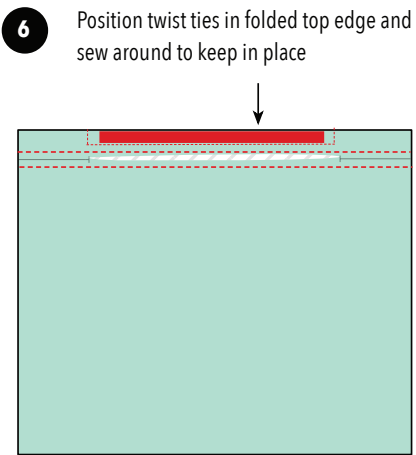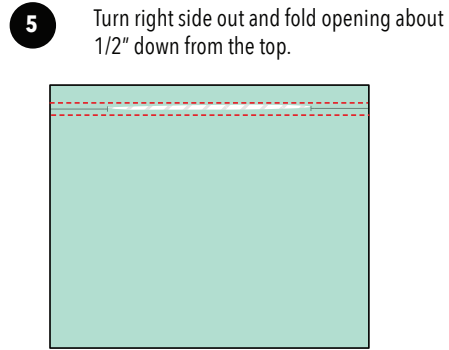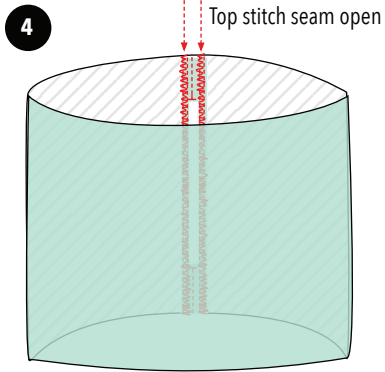

Flip over to the front and turn binding strip right side out

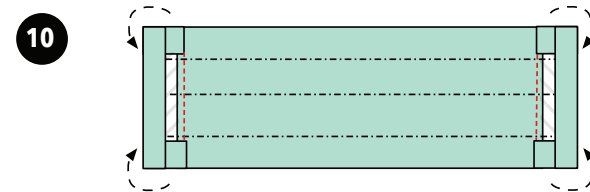

Fold edges over .25", then fold again. Place elastic in channel before stitching closed. DO NOT stitch down elastic.

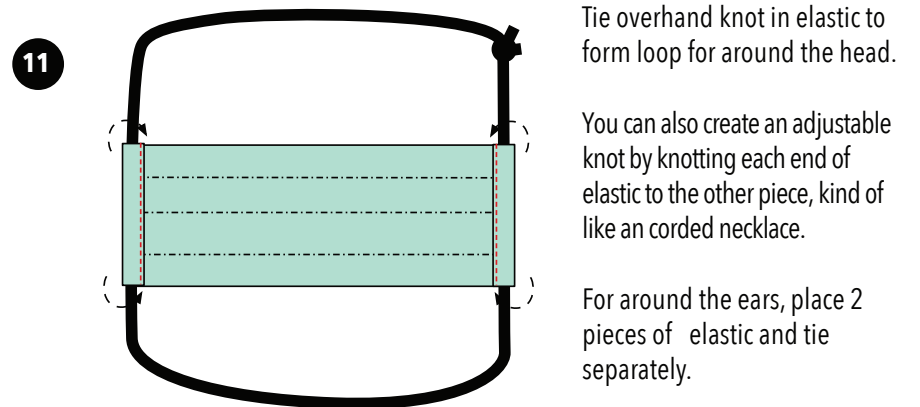

Tie overhand knot in elastic to form loop for around the head.

You can also create an adjustable knot by knotting each end of elastic to the other piece, kind of like an corded necklace.

For around the ears, place 2 pieces of elastic and tie separately.

Rotate knot into side casing. Easy to adjust

It is easy to make one continuous loop and user can have options to make behind the ears. Different head sizes, face size, ear sizes... adjustable is key.

