



ITest4 is a Division of Boss Medical Ltd.
 14823 Yellowhead Trail NW,
 Edmonton, Alberta, Canada, T5L 3C4
 Ph: 780-250-7773
 Web: www.BossMedical.ca

ITEST4™ ONE STEP MULTI-DRUG SCREEN TEST CUP (URINE)



TEST	DESCRIPTION	CUT-OFF LEVEL NG/ML
AMP	Amphetamine	1000
BAR	Barbiturates	300
BZO	Benzodiazepines	300
BUP	Buprenorphine	10
COC	Cocaine	300
MDMA	Ecstasy/Molly	500
FEN	Fentanyl	300
THC	Marijuana	50
MTD	Methadone	300
MET	Methamphetamine	1000
OPI	Opiates	2000
OXY	Oxycodone	100
PCP	Phencyclidine	25

Instructions:

1. Open the pouch, remove the Test Cup.
2. Open Test Cup lid. Urinate directly into the Test Cup. Be sure to fill the cup with urine sample between a min of 30ml to a max of 110ml.
3. After urine specimen has been collected, close the lid securely.
4. Peel off label to reveal test results. Read test results at 5 minutes. Results expire after 10 minutes.

Interpretation of results:

Negative - Two colour lines appear. One in the Control (C) region and one in the Test (T) region.

Positive - One colour line appears in the Control (C) region.

Invalid - Control line fails to appear. In case of invalid results, review procedure and repeat test using a new test device.

Important Information:

Do not use after the expiration date.

Results are preliminary and positive results do not always mean that drugs are present in the donor. Certain foods and medicines, diet plan drugs, and nutritional supplements may yield a false positive test result with this product.

Laboratory tests confirm the preliminary screen results.