



Little Martin's Breastcare Thermopads

User Manual

For Model BBT1708

*Little
Martin's*
DRAWER



Warranty Certificate

This certificate promises our customers that they will receive free repair. If you notice any problems listed below within the warranty period, please contact our customer service with this certificate.

1. Once you have used this product, we basically will not offer exchange.

2. When the product is damaged (after proper use) within the warranty period which is a year from the purchase date, we will repair it for free. Contact us via email support@littlemartinsdrawer.com, and send the product over to our customer center.

3. Even within the warranty period, in the following situation the repair process will come with a fee.

- Natural discoloration of the plastic area.
- Damage due to misuse of the product.
- Damage due to fire, water, earthquake and other natural disasters.
- Damage due to commercial or business usage.
- The shipping fee for paid repair.

4. This service will apply only inside United States.

5. Once the product is out of production and certain parts are out of stock, we will not be able to offer certain repairs.



CONSULT INSTRUCTIONS FOR USE



100% RECYCLEABLE



Thank you for choosing Little Martin's Drawer. We advise that you carefully read this instruction manual to ensure product efficiency and correct use.

- Please keep this manual in a safe place.
- If giving as a gift, please be sure to include this manual.

Contents

Precautions.....	1
Operating Instructions	2
Product Details.....	2
Warranty Certificate	on the back cover

Precautions

- For external use only.
- Keep out of reach of children.
- Do not use on infants or allow infants to chew on packs.
- Do not use for purposes other than intended.
- Overheating or prolonged application may cause injury.
- Do not apply to sensitive skin, insensitive skin or open wounds.
- Always test pack temperature prior to application.
- Use only under adult supervision.
- Do not puncture pack and/or ingest contents.
- Do not use if product is damaged.
- Do not fall asleep using pack.
- Do not heat beyond instructed times.
- If you experience extreme, prolonged or chronic pain, contact a physician.

Operating Instructions

Thermopad should only be applied for 20 minutes. To use with breast pump, wrap heated pack around outside of flanges, or use prior to pumping breasts.

• For cold therapy:

Freeze pack for at least 2 hours

• For heat therapy:

Place one pack inside microwave lying flat for time indicated in table below. Heat packs separately. After testing pack temperature, you may heat for longer in 5 second increments as needed until desired temperature is reached. Do not heat more than 25 seconds in total.

ALWAYS test prior to application

Suggested heating times for Breastcare Thermopad
700 Watts - 16 seconds
1,000 Watts - 13 seconds
1,250 Watts - 11 seconds

Product Details

- Designed to be used in conjunction with breast pump.
- Cold therapy: relieves engorgement, swelling and pain.
- Heat therapy: relieves plugged ducts and mastitis, encourages milk let-down.
- Microwave safe.
- Free of BPA and BPS.
- Reusable, comes with soft washable covers.