



*Little Martin's* Baby Snot Sucker

# User Manual

For Model BSS1703



## Warranty Certificate

This certificate promises our customers that they will receive free repair. If you notice any problems listed below within the warranty period, please contact our customer service with this certificate.

1. Once you have used this product, we basically will not offer exchange ment.
2. When the product is damaged (after proper use) within the warranty period which is a year from the purchase date, we will repair it for free. Contact us via email [support@littlemartinsdrawer.com](mailto:support@littlemartinsdrawer.com), and send the product over to our customer center.
3. Even within the warranty period, in the following situation the repair process will come with a fee.
  - a. Natural discoloration of the plastic area.
  - b. Damage due to misuse of the product.
  - c. Damage due to fire, water, earthquake and other natural disasters.
  - d. Damage due to commercial or business usage.
  - e. The shipping fee for paid repair.
4. This service will apply only inside United States.
5. Once the product is out of production and certain parts are out of stock, we will not be able to offer certain repairs.




100% RECYCLABLE



- Quickly and gently removes mucus from baby's nose by mouth suction
- Biocompatible medical grade mouth-piece
- Easy to use, easy to clean, easy to carry
- Cute pen-shaped design

## Name of Parts

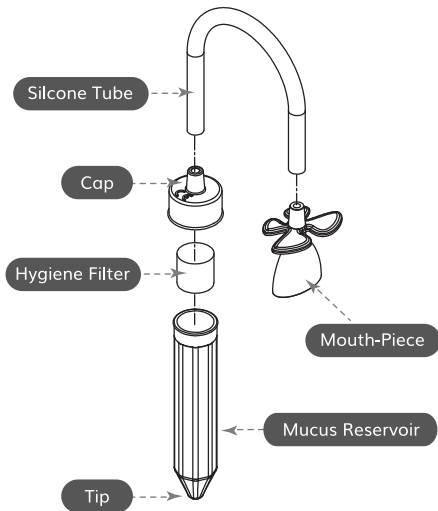

1

## How to Use

- ① Lay your baby face-up on a changing table or hold him on your lap with his head on your knees.
- ② Insert the tip of snot sucker gently into baby's nostril and place the mouth-piece in your mouth.
- ③ Inhale through the mouth-piece. Start slowly and gradually increase the suction strength until the mucus is sucked into the snot sucker.
- ④ If the mucus is too dry or too thick, use some saline to soften the mucus before applying suction.

## ⚠ Warnings

- ① Do not insert the tip into the nostril too deeply.
- ② Do not use the snot sucker in baby's mouth or ears.
- ③ Do not allow children to play with the snot sucker because it is not a toy. Keep out of reach of children.
- ④ Do not sterilize the snot sucker with boiling water, steam, or microwave.
- ⑤ Do not use the snot sucker if there is any wound, symptom of inflammation, or bleeding at the nose.

## Cleaning Instructions

- ① Clean the parts with warm water and soap after each use.
- ② Leave the parts dry before re-joining them.

2