



*Little Martin's* Washable Nursing Pads

# User Manual

For Model WNP1708

Thank you for choosing Little Martin's Drawer. We advise that you carefully read this instruction manual to ensure product efficiency and correct use.

- Please keep this manual in a safe place.
- If giving as a gift, please be sure to include this manual.

## Contents

Precautions .....	2
Product Features .....	2
Instructions For Use .....	3
Specifications .....	4

## Precautions

- Regularly replace product to maintain healthy breasts.
- Ensure to clean breasts with warm cotton towel or cloth prior to breastfeeding.
- Do not use if product is damaged.
- Do not use product for purposes other than intended.
- Keep out of reach of children.
- If any discomfort occurs, stop using this product immediately and consult your doctor.

## Product Features

Little Martin's Washable Nursing Pads are bigger, thicker, and have a stronger absorbent capacity than other brands. We are confident our product is top quality and will provide moms with leak-proof performance and full breast support.

Nursing pads can be prepared in advance to avoid the rush during lactation and prevent clothes from getting wet. At only 0.4cm thick, our nursing pads are comfortable and breathable so your nipples will stay protected.

### Instructions For Use

1. Remove nursing pads from packaging.
2. Wash before use by rinsing with cool to warm clean running water and drying in low temperature conditions.
3. Place inside bra cup so it is centered on your nipple.
4. Adjust bra and straps to fit comfortably.
5. Wash after use by machine or hand washing, then dry in low temperature conditions.

### Note:

- Do not soak nursing pads in water.
- Avoid sun exposure and high water temperatures.
- Do not bleach, iron, or dry clean nursing pads.

### Tip:

We recommend using a front button bra.

### Specifications

Dimensions:	Diameter 12 cm, Thickness 0.4 cm
Material:	Cotton