

Nano Wall System

Assembly Guide

Warnings and Safety

Please review the following safety information prior to assembling your Joovv Nano Wall System. Attempting to assemble and use this equipment without fully understanding its features and functions may result in unsafe operating conditions. Some Warnings and Cautions are also inserted within the manual where they are most meaningful. Notes are also located throughout the manual to provide additional information related to specific features. If you have a question regarding the installation, set up, operation, or maintenance, please contact Joovv Inc. at help@joovv.com.

WARNINGS identify conditions or practices that could result in serious adverse reactions or potential safety hazards. **CAUTIONS** identify conditions or practices that could result in damage to the device or other equipment.

Warnings

- No modification of this system is allowed.
- System is only meant to support applicable Joovv products.

Cautions

- Read user manual prior to use and keep for future reference.
- Choking Hazard, small parts. Keep away from small Children.
- For interior use only, storage or installation of the system in direct sunlight may cause the system to yellow or break down over time. Normal storage or installation in window-lit rooms is acceptable.
- Do not leave the system in any place that could be damaged by children, pets, pests, or liquids. If you suspect your system of devices is damaged, contact customer service at help@Joovv.com.
- Do not install system near heated surfaces such as nearby fireplaces or ceramic heaters.
- Do not install or store system in humid or wet locations such as a sauna or wet sauna.
- Remove the device before adjusting height of Slider Pole Mount.
- When loosening the adjustment knob, do not exceed more than 1.5 turns counterclockwise to loosen. Additional turns can cause the Slider Pole Mount to fall.
- Do not use power tools to assembly parts in this kit.
- Do not use system without wall brackets installed.
- Do not mount more than 2 devices per pole upright.


Safety Standards

Conforms To: IEC 62471, 60601-2-57, 60601-1-11, 60601-1-6 & AAMI STD ES60601-1


Certified To: CSA STD C22.2 #60601-1


Table of Contents


Notices	2
Parts List	4
Nano Wall Configurations	5
1 Pole Setup	9
2 Pole and 3 Pole Setups	17
Adding Spacers to Mounted Devices	26
Revision History	27
Warranty	27
Return Policy	27
Contact Information	27

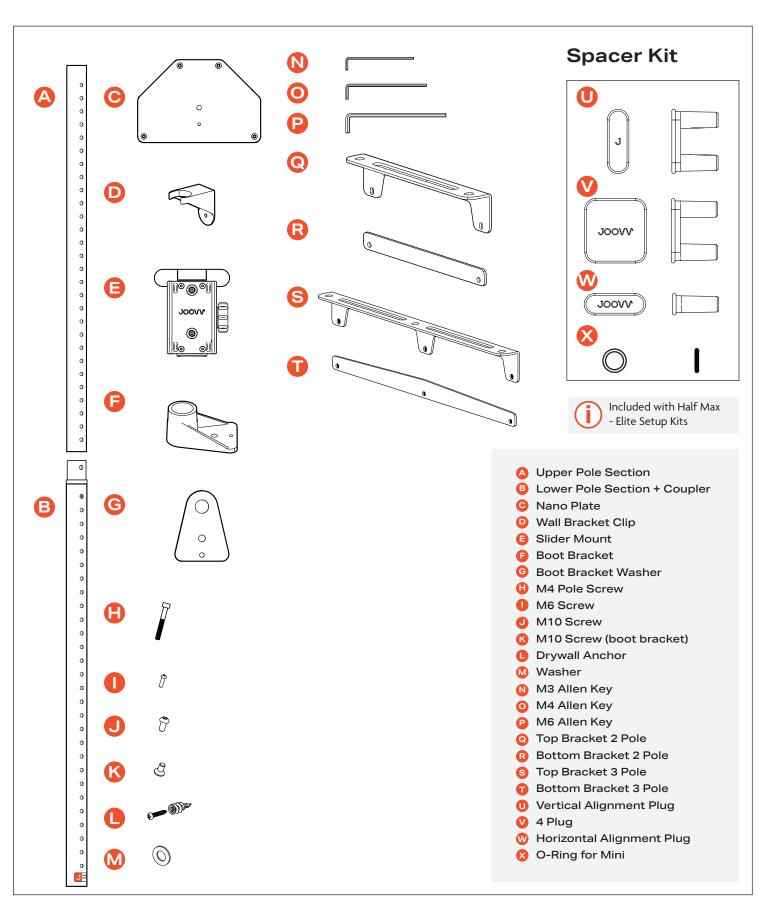


Nano Wall Configurations

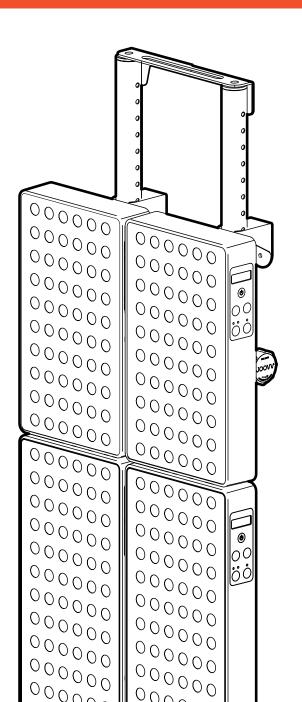

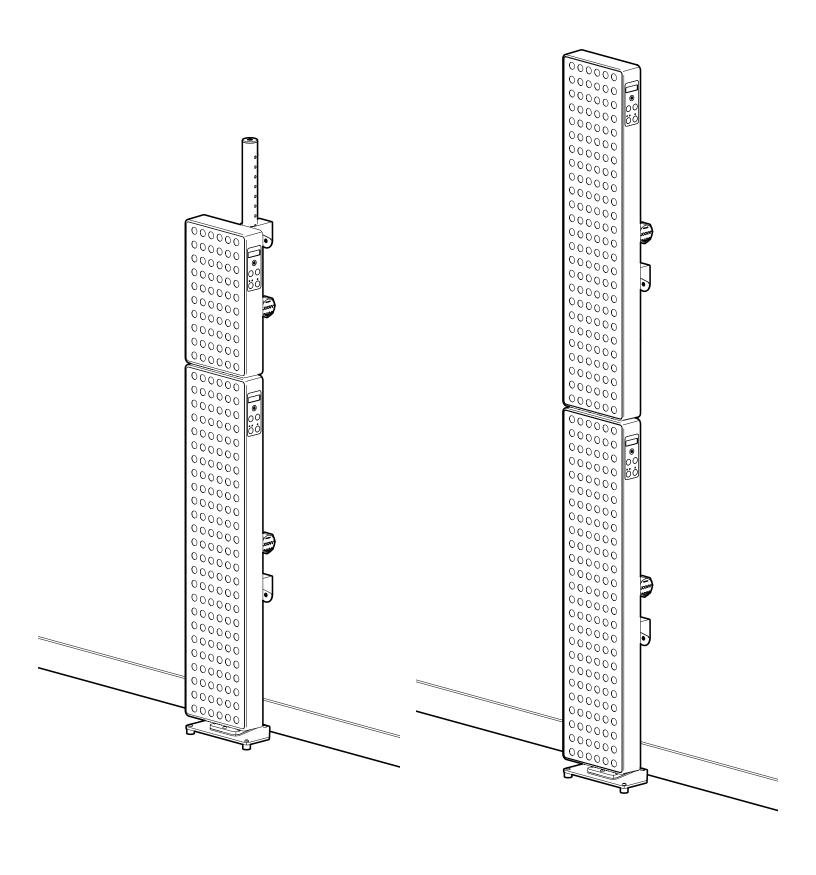

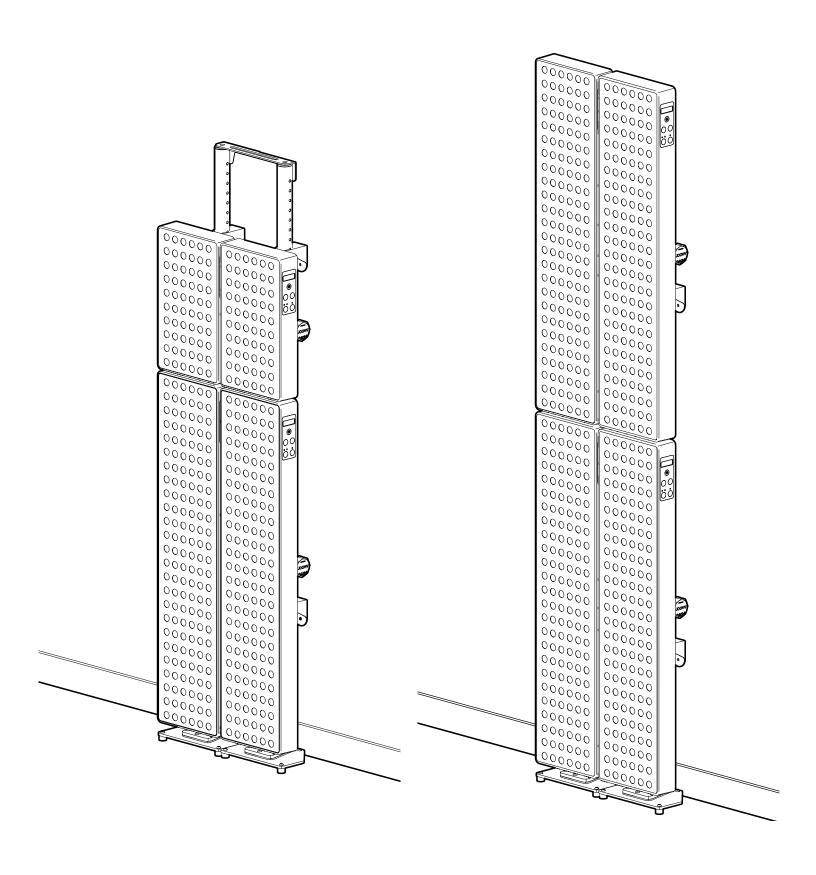

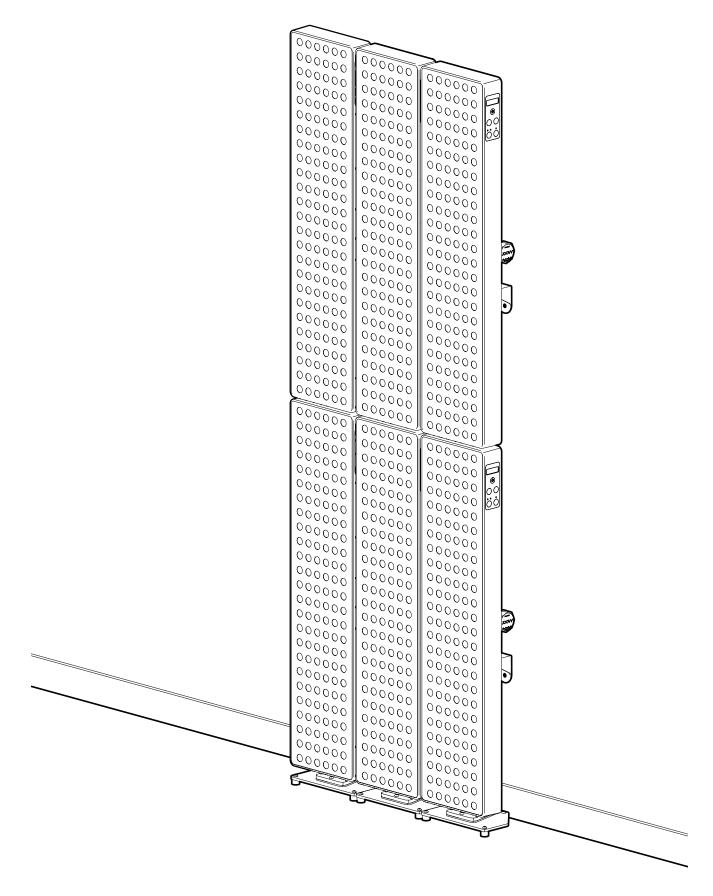

1 Pole Setup


Step 1 - Connect Pole Upright

Connect the Top and Bottom sections of the Pole Upright by inserting the Top section into the coupler portion of the Bottom section and securely fastening using the M4 screw (H) as shown.

IMPORTANT PLEASE READ

When initially installing the Slider Mount keep the M4 screws (H) semi tight to allow the Pole Uprights to flex slightly. After positioning the Slider Mount to the dedicated height, the M4 screws can be fully tightened.


Note, over tightening the M4 screw (H) can make it difficult for the Slider Mount to pass through the coupler section of the Pole Upright.

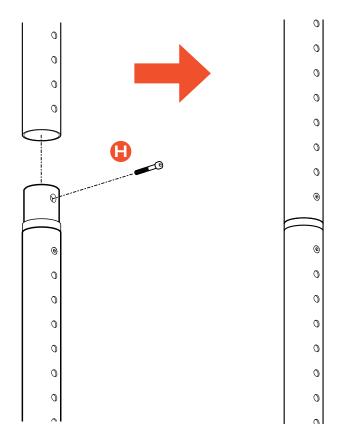


Secure the Boot Bracket to the Pole Upright. To do so, attach the bracket on the side of the pole that is marked with the J label. Use screws I and K to attach the Boot Bracket as shown.

Connect the Nano Plate to the Pole Upright. To do so, place the Boot Bracket Washer on the Nano Plate and connect using screws I and J as shown.

Note: Screw J comes in two lengths. You will use the shorter length for this step.

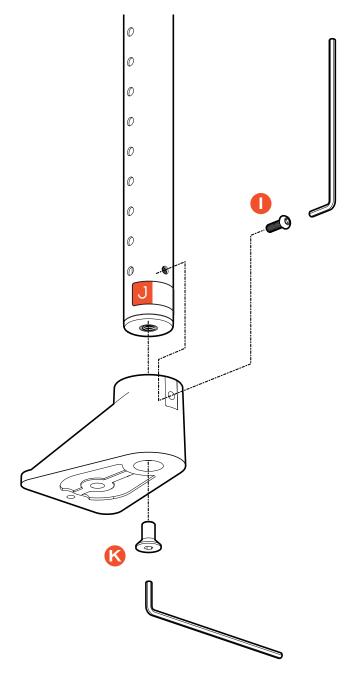


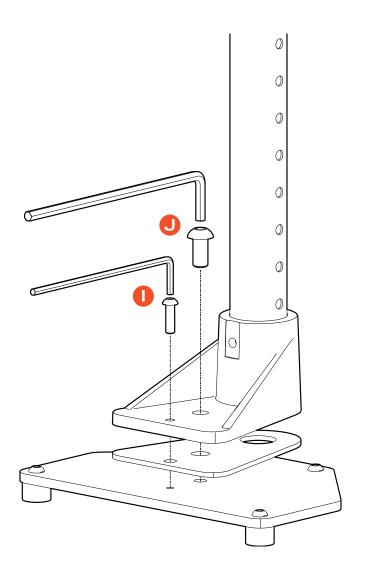

Install Slider Mount(s) to Pole Upright and secure Pole Insert using screw I, then adjust Slider Mount to the desired height.


Top of wall bracket should be half way between hole 7 from the top Top of wall bracket should be half way between hole 13 from the bottom

Clip Wall Brackets to Pole Upright and position system up against the wall.

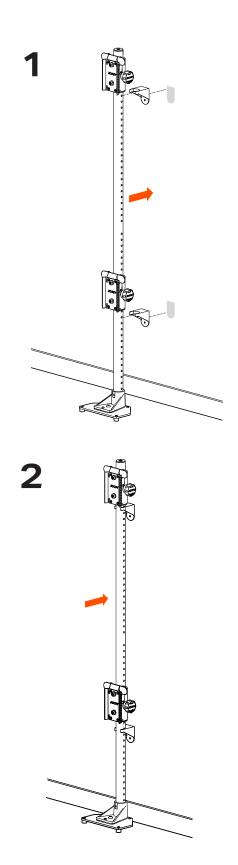



Step 7


Mark Drywall Anchor locations for both Wall Brackets with a pencil or marker as shown. Set Nano Wall System aside and install Drywall Anchors in previously marked locations.

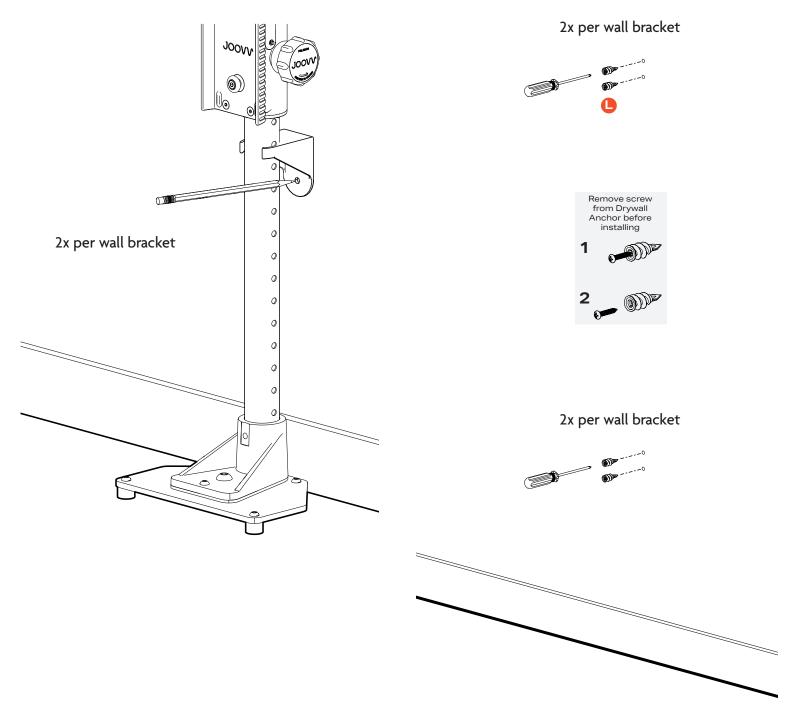


Install Wall Brackets using the Drywall Anchor Screw and Washer as shown. Repeat for each Wall Bracket. Clip Pole Upright system into installed wall brackets.

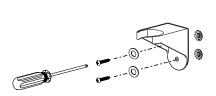

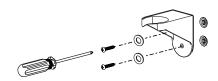

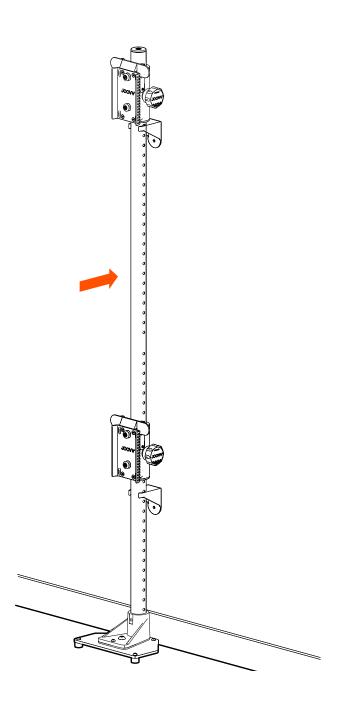

To adjust the Slider Mount, turn the knob counterclockwise ~1 turn and pull out. Slide the mount up or down based on your desired height. To secure the Slider Mount tighten the knob by turning it clockwise until hand tight.

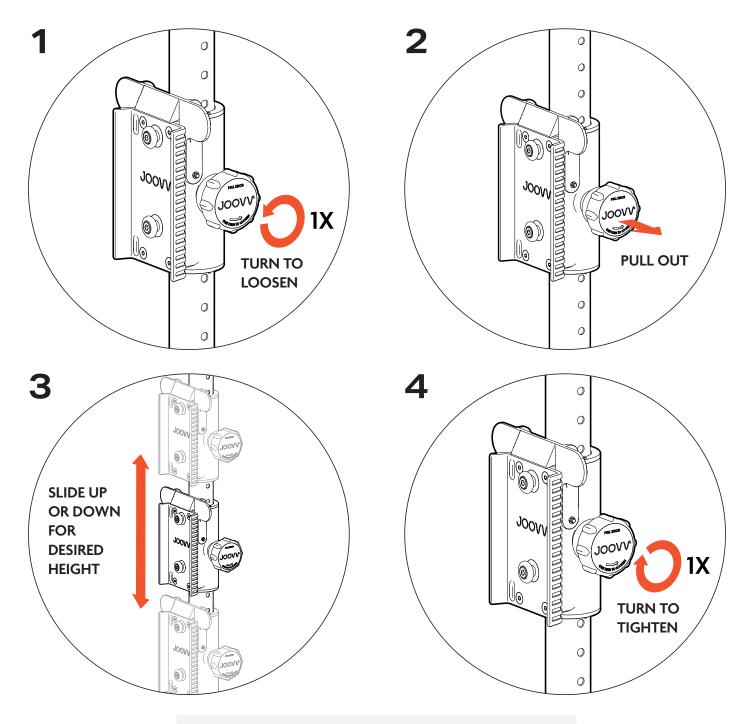




WARNING!


When loosening the adjustment knob, do not exceed more than 1.5 turns counterclockwise to loosen.

Additional turns can cause the Slider Pole Mount to fall.

While holding the device as shown below, align the device over the Slider Mount keyholes and slide the device down onto the Slider Mount to secure.

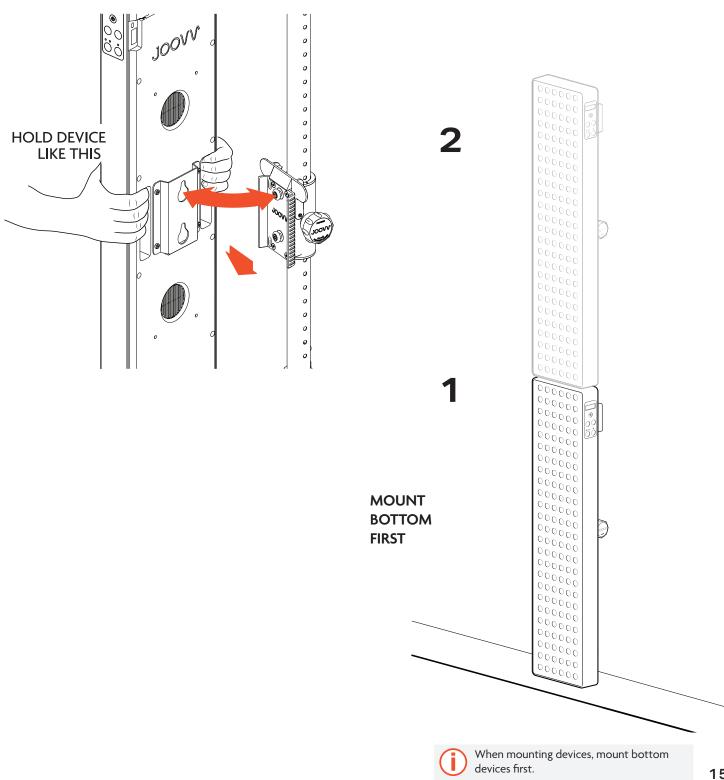

To remove the Solo or Mini Device from the Slider Mount, follow the steps illustrated below.

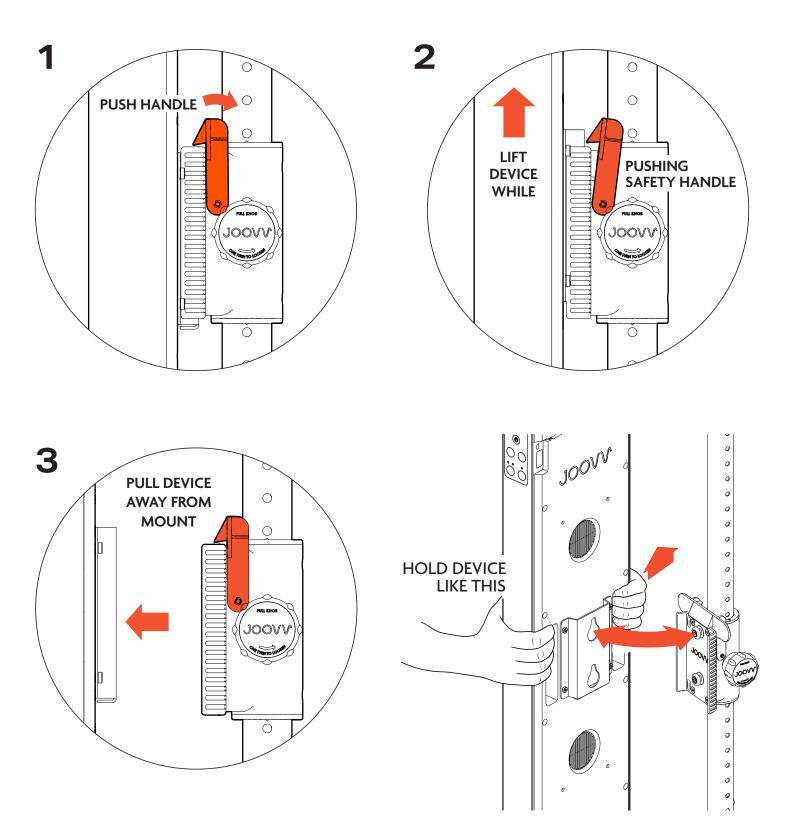

2 Pole and 3 Pole Setups


Step 1


Refer to page 9 on how to connect the Pole Upright. Secure Boot Bracket to each Pole Upright. To do so, attach the bracket on the side of the pole that is marked with the J label. Use Screw K to attach each Boot Bracket as shown.

Step 2


Connect the Nano Plate to each Pole Upright. To do so, place the Boot Bracket Washer on the Nano Plate and connect using screws I and J as shown.

Note: Screw J comes in two lengths. You will use the shorter length for this step.

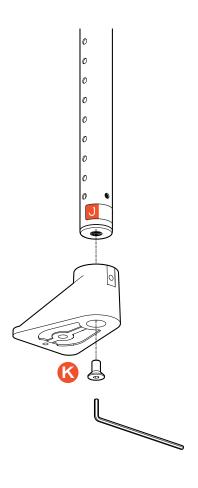

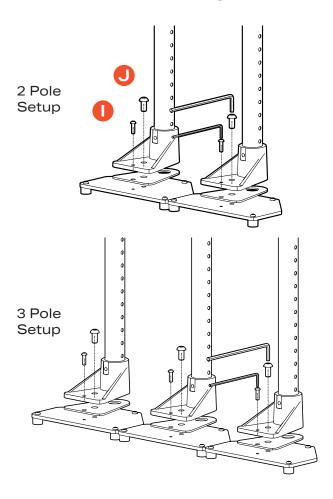


Install Slider Mounts to Pole Uprights and adjust to the desired heights.

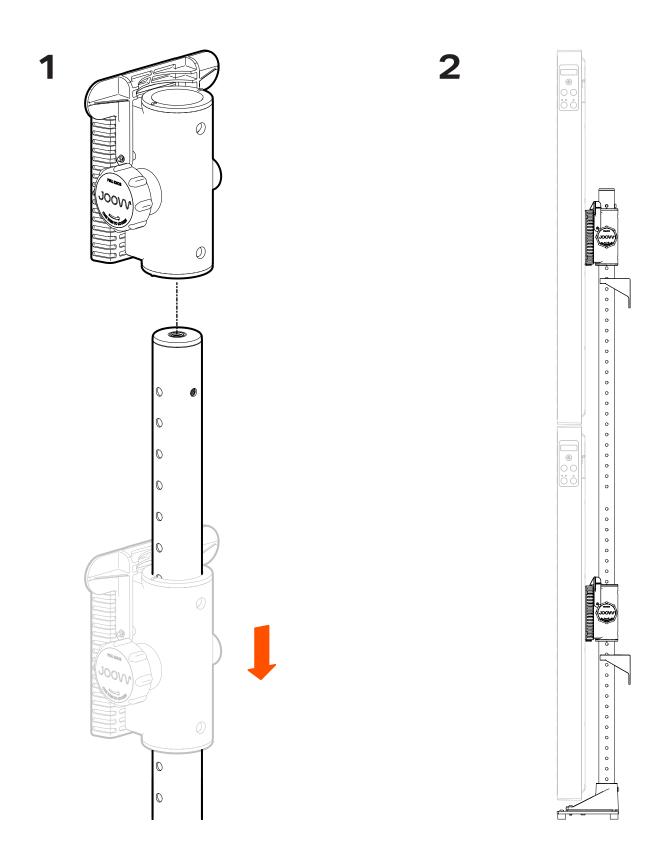


Install Top and Bottom Brackets using screws I and J. Clip the Wall Brackets to the Pole Uprights and position the system up against the wall.

Note: Screw J comes in two lengths. You will use the longer length for this step.

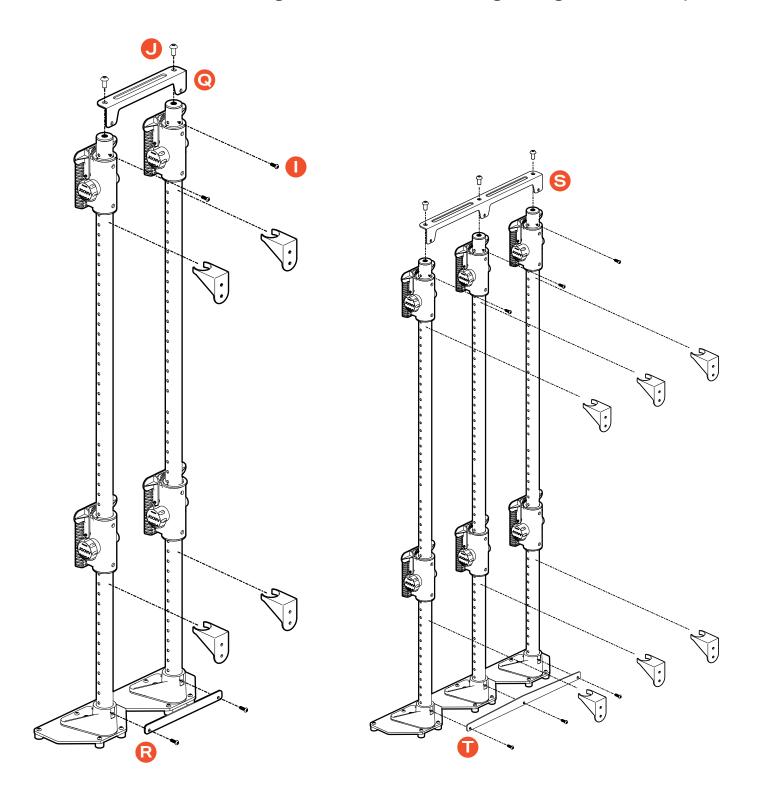


Step 6


Mark Drywall Anchor locations for both Wall Brackets with a pencil or marker as shown. Set Nano Wall System aside and install Drywall Anchors in previously marked locations.

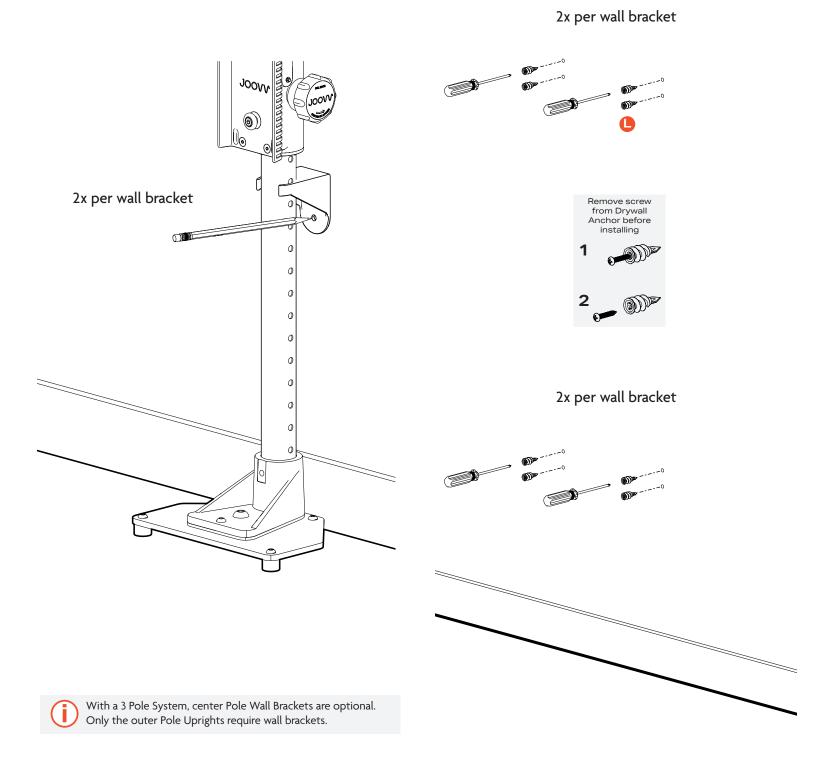

Install Wall Brackets using the Drywall Anchor Screw and Washer as shown. Repeat for each Wall Bracket. Clip Pole Upright system into installed wall brackets.

1

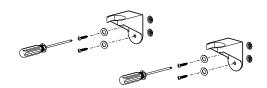


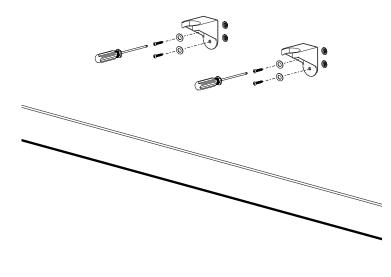



With a 3 pole system, center pole wall brackets are optional. Only the outer pole uprights require wall brackets.

To adjust the Slider Mount, turn the knob counterclockwise ~1 turn and pull out. Slide the mount up or down based on your desired height. To secure the Slider Mount tighten the knob by turning it clockwise until hand tight.

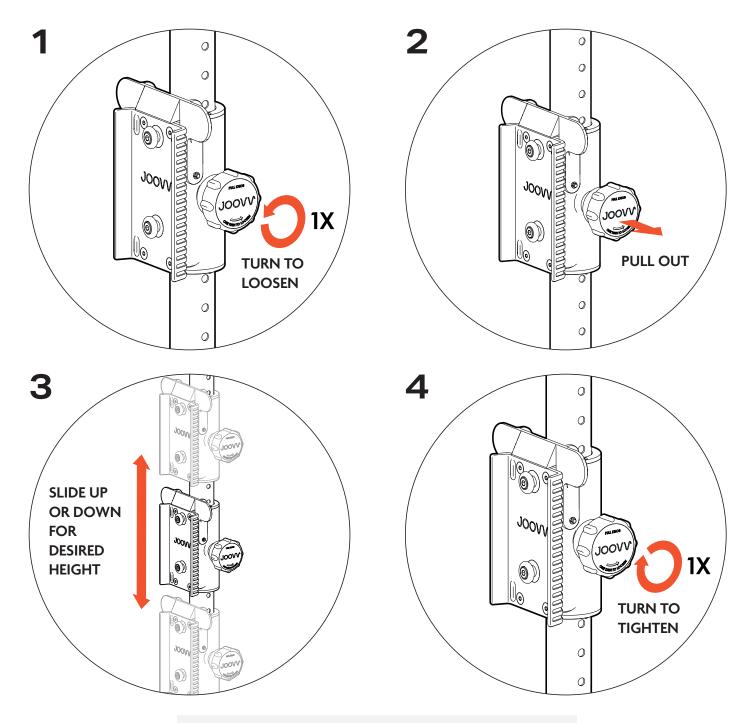




WARNING!

When loosening the adjustment knob, do not exceed more than 1.5 turns counterclockwise to loosen.


Additional turns can cause the Slider Pole Mount to fall.

While holding the device as shown below, align the device over the Slider Mount keyholes and slide the device down onto the Slider Mount to secure.

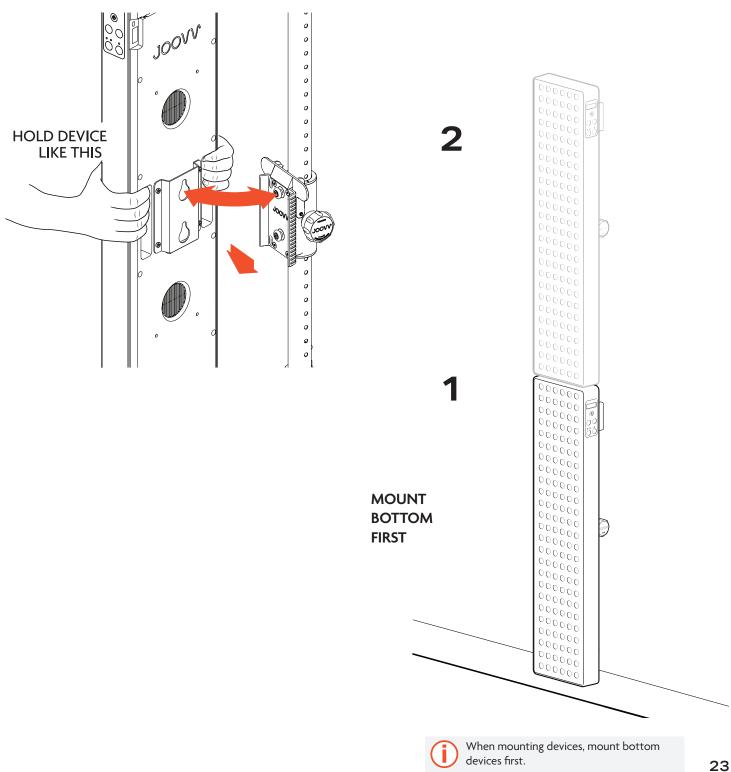


With the help of a 2nd person, pull loose the Nano Wall System from the Wall Bracket Clips. Spin 90 degrees to gain access to the backside to install cables and Spacers. See page [26] for instructions on Spacers.


Once complete, reposition the Nano Wall System back into place by securing each Wall Bracket Clip to the Pole Uprights.

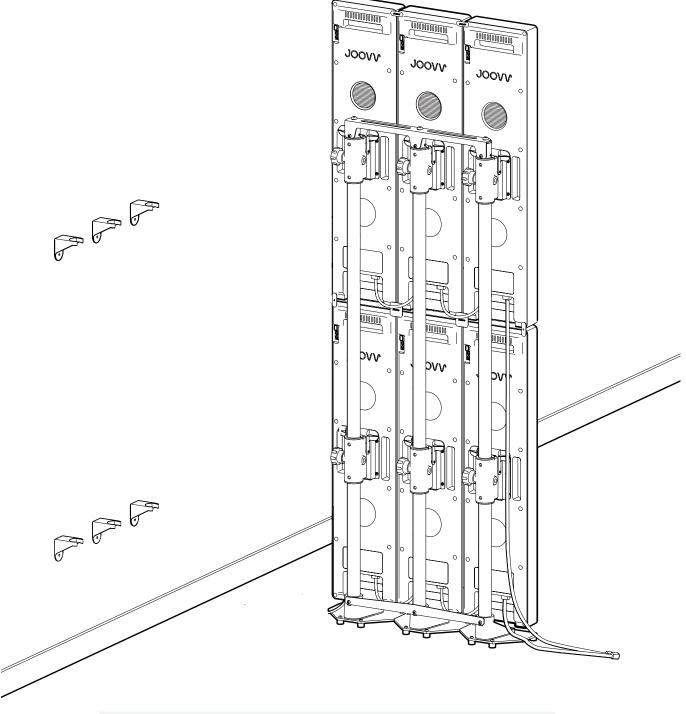

- $\bullet\,$ Refer to Mini and Solo user manual for wire diagrams regarding cable installation.
- If power outlet is located behind your Joovv unit, plug in system before attaching unit to wall.

To remove the Solo or Mini Device from the Slider Mount, follow the steps illustrated below.

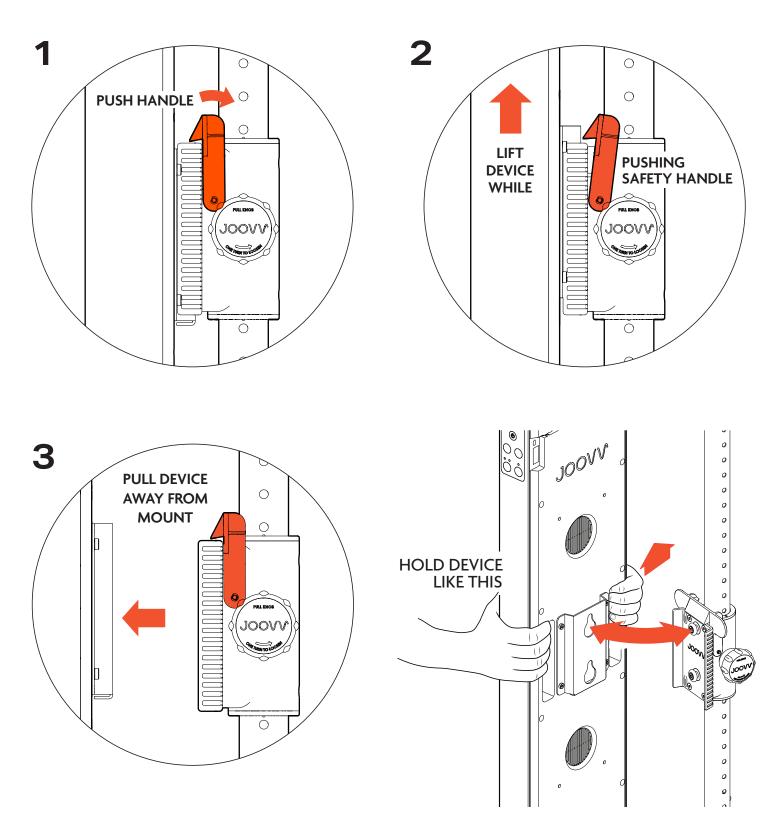

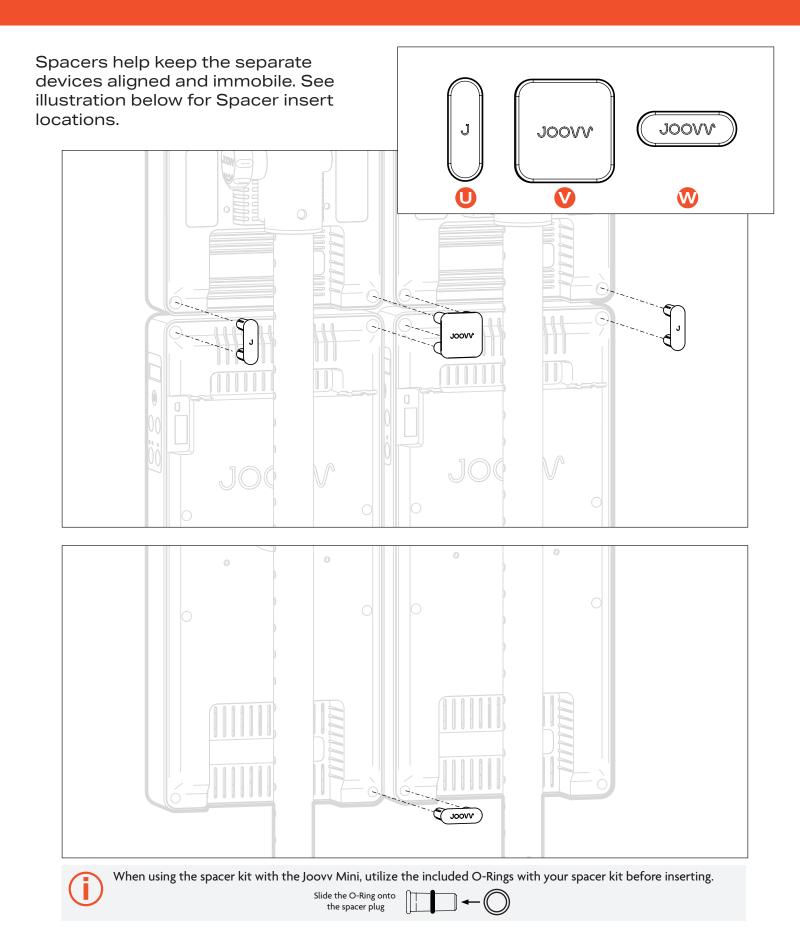

Revision History		
Date	Revision #	Description of Changes
November 2020	JV-MKT-DOC-2805 (Ver.0)	Ver.0 Release to market
June 2021	JV-MKT-DOC-2805 (Ver.1)	Addition to pages 2, 4, 9, 17 - New Pole Kit
March 2022	JV-MKT-DOC-2805 (Ver.2)	Addition to page 9
October 2022	JV-MKT-DOC-2805 (Ver.3)	Corrected letter identifiers throughout, Updated contact information.

Warranty

All Joovy mounting accessories are covered by a 2-year limited warranty. Please visit our website at https://joovv.com/pages/warranty for details of the limited warranty. To register your product, visit https://joovv.com/pages/warranty for instructions. Failure to register will not affect your limited warranty rights.

Return Policy

All mounting accessories come with a 14-day return period (60-day return period when purchased as part of a device kit). If you're not satisfied with your purchase within the first 14 days, please visit our website at https://joovv.com/pages/returns for additional details.

Contact Information



Joovy, Inc.

3579 US-50, Suite 307 Carson City, NV 89701

USA

Email us: help@joovv.com Website: www.joovv.com

Online Setup instructions: www.joovv.com/setup

