

Padded Legs

The padded legs help to prevent the material roping and creasing under legs. They also help to reduce skin injury or discomfort during sling application.



Head Support

Head Support offers patient more comfort and protection during transfer from sitting to lying position.



Polyester Material

Disposable Material

| Material Type | Part Number | Description | Size | Capacity |
|------------------|----------------|---|-------------|----------|
| Polyester | SL-MAA4000M-M | General Purpose Clip Sling With Padded Legs. | Medium | 600LB |
| | SL-MAA4000M-L | | Large | |
| | SL-MAA4000M-XL | | Extra Large | |
| Disposable Woven | SL-MFA1000M-M | Patient Specific Clip Sling With Padded Legs. | Medium | |
| | SL-MFA1000M-L | | Large | |
| | SL-MFA1000M-XL | | Extra Large | |

Sling User Guide

— Lifting from a chair —




1. Position the sling against the back with the clip straps on the outside. Ensure the person's spine is covered and the head support area is behind the head. The shoulder strap should be at shoulder level.



2. Position each leg piece under and between the person's legs.



3. Position the cradle just below shoulder level.

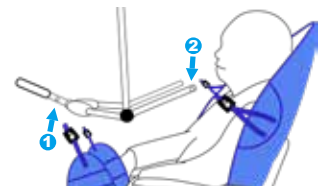

4. Attach the leg clips to the spreader bar first, then attach the shoulder strap attachment clip to the cradle.

