

VEVOR[®]
TOUGH TOOLS, HALF PRICE

HYDROGEN OXYGEN
FACIAL MACHINE

MODEL:H202-7







NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

	Warning-To reduce the risk of injury, user must read instructions manual carefully.
Warning:	It means that due to a mistake,death or serious injury could result
Danger:	Indicates that due to misoperation, death and injury may occur
	Tokens represent mandatory items
	The mark indicates prohibited items
	The protection type of the device is Type B

1. Brief introduction:





Aqua facial
handle



Ultrasound handle



Skin scrubber



RF handle



Spray gun



Cold hammer



Mask

1.1 Please use it safely

The following are the symbols used in the instrument and instructions. In order to use the instrument correctly, prevent injury to operators and patients, protect the instrument from being damaged, and avoid property damage, please pay attention to the types and meanings of the following symbols. Precautions for use

Warning:

1. Please do not use and connect the socket, so as to avoid fire.
2. When replacing the fuse, be sure to turn off the power switch to avoid fire and electric shock.
3. 100-240V power supply and effectively grounded socket must be used;
4. There is danger of fire and electricity in using damaged and processed power lines.
5. Do not over-examine and twist the electro-hydraulic wire, so as to avoid fire and electric shock accidents:
6. If the instrument feels loose, falls or is damaged, please contact the company in time;
7. Do not use this instrument in unstable environment such as tit, vibration and attack
8. No water or debris can enter the instrument. in case of foreign matter entering the instrument, please contact the company in time;

9. The power plug must be unplugged when the instrument moves;
10. Vibration should be avoided as far as possible when the instrument is moving;
11. The instrument should be placed in a stable place, and a space of at least 5cm should be set aside around it to ensure air circulation and facilitate the heat dissipation of the instrument;
12. Do not cover anything on the instrument to prevent cold blockage of the vent;
13. Avoid the use of this instrument in the following environments: sunlight; High-temperature environment; Near water environment; Vibration, dumping place; Strong magnetic field environment; Places where electromagnetic wave and impulse voltage occur; The storage place of chemical drugs; Corrosive gas field;
14. The instrument shall not be placed and used near radio, TV, copier, fax machine, etc.
15. Do not use "such as microwave" with other instruments, so as not to interfere with each other and cause misrepresentation.
16. If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
17. This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.

1. 2 Precautions before use

Warning:

1. Before starting, be sure to read the instruction manual carefully. The author must be familiar with the instruction manual and operation method, and the instrument must be managed by specially-assigned person.
2. Please set the operating parameters under the guidance of professional personnel;

Note:

1. Confirm whether the instrument is in normal operation before use;
2. Confirm whether the grounding wire is connected correctly;

3. Make sure that all wires are correctly connected and safe;
4. When the instrument is used, it must be used correctly, otherwise it may be dangerous and must be paid great attention to.

1.3 Precautions for use

Warning:

When abnormal conditions occur in the operation of the instrument or patient, the operator shall immediately leave the operating handle from the patient and press the stop switch to stop the use of the instrument:

Note:

During the operation of the instrument, it is necessary to constantly observe the situation of the beauty guests and the instrument, and pay attention to whether there is any abnormal situation at any time;

The instrument shall be used correctly by skilled operating technicians. Personnel other than technicians shall not use the instrument.

1.4 Precautions after use

Note:

1. After the operation, put the control handle back on the control handle hanger.
2. After the handle is used, the probe should be cleaned for the next use.

1.5 Precautions for falling, storage and inspection

Warning:

1. Please do not use the instrument when it is abnormal. In case of smoke burnt smell, can not be mentioned, if continue to use, there is a risk of fire electric shock. Please press the stop button quickly, turn off the power, unplug the power, and immediately contact the company or distributor;
2. Personnel other than technical personnel must not maintain modified instruments. This instrument is completely not open inside;

Note:

1. Instruments and components must be checked regularly;
2. Do not use cleaning tools that damage the protective coating of the instrument, such as brush, broom, etc;

3. If the instrument is not used for more than 2 weeks, the instrument must be carefully checked before being used again.
4. Warning and suggestive instructions

Warning:

When the bright frequency device and this instrument are connected to the same patient at the same time, the electrode position of the instrument may cause burns to the skin tissue of the patient, and may also cause damage to the instrument. When the instrument is used near short wave or microwave equal (1m), it is affected by microwave interference shadow. Ring, instrument output may be unstable. Using electrodes close to the chest increases the risk of heart deterioration.

SAVE THESE INSTRUCTIONS

2. Caution

1. Please turn off the power if needs to change the fuse.
2. Please pull out the power cable if needs to move the machine.
3. The instrument should be placed on the temperature between 5-40 degree, and placed in clean indoor, also avoid direct sunlight. In and make sure the vents are not blocked. Do not put anything on the instrument at any situation even temporary
4. Install the instruments with probes with the picture below and install the handles on the accessory holder and put in them right places.
5. Do not use any irritating skin care products, cosmetic within 24 hours after treatment and pay attention to the sunlight.
6. Using moisturizing mask for a week after treatment.
7. Those do not use the machine:
 - a. Pregnant women
 - b. Cardiovascular disease, kidney disease, cancer, diabetes.
 - c. Bacterial infection site.
 - d. Kelo cote patients.
 - e. Whom with heart pacemaker.
 - f. Whom with extreme sensitivity to skin diseases.

- g. Whom with Erythema.
 - h. Whom with thin skin cuticle cannot use hydro facial handle.
 - i. People with thin skin cuticle cannot use RF for more than 20 minutes.
 - j. Cannot use the machine with severe allergy or severe acne.
8. If the patient feels uncomfortable, please turn off the machine in case of dangerous.

3. How to install the instrument:

3.1 Description for the Back of the instrument




3.2 When moving the instrument, please remove all the liquid on the Bottle A,B,C,D,otherwise, liquid will enter the inside of the instrument during moving, causing damage to the instrument.

3.3 When using the instrument, if the instrument sounds beep beep, please observe the interface for the wasted bottle. If it is twinkling, please pour out the wasting water and install the wasted bottle to the instrument after done pouring out.

4. Operation procedures and methods

4.1 Using for Aqua facial handle

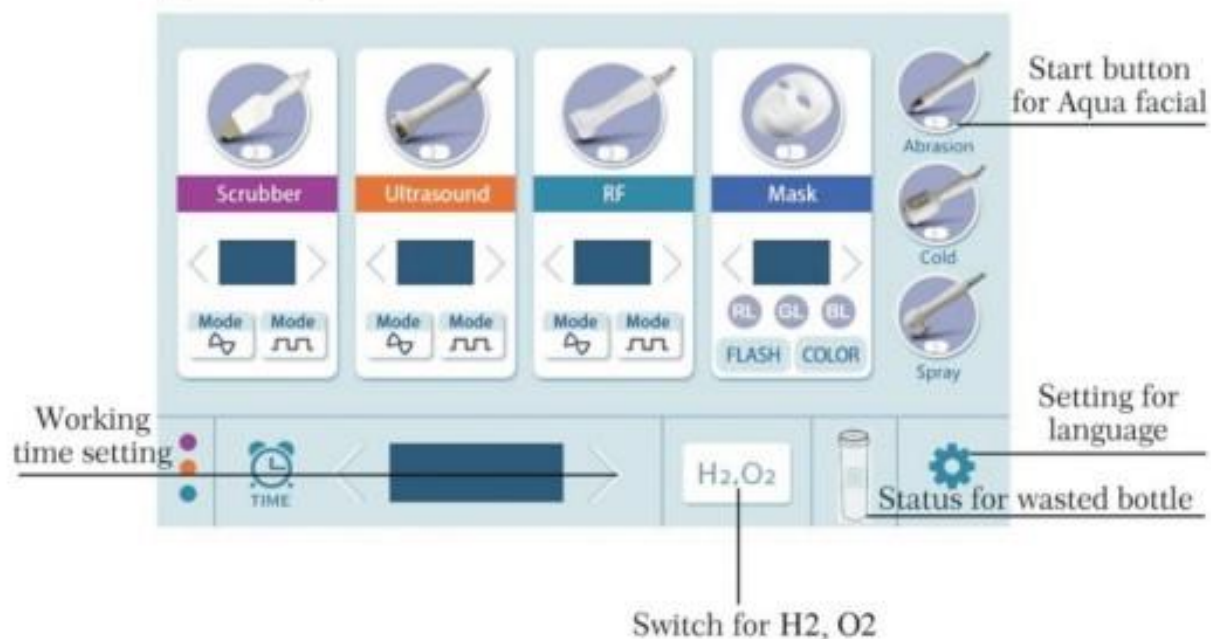

4.1.1 Cleaning the face by using facial cleanser.


4.1.2 Choosing the aqua facial cleaning head according to the situation for the skin. Choosing the liquid (A,B,C) by rotating the knob or the H₂, O₂.

4.1.3 Adjusting the flow level and pressure lever to start working, the whole cleaning procedure can be working for about 10 minutes.

Caution:

When using the H₂O₂ from the bottle D which is on the back of the machine , first select the knob to the D position, and press the Button (H₂O₂), then please wait for about 3 minutes, to make sure there are enough (H₂O₂)inside the bottle.

4.1.4 When done using the instrument, select the knob to position C to clean the aqua facial tube. First, select the flow level to the maximum intensity, and then select the pressure level to the maximum. Then turn on the machine.putting on the aqua facial head to the aqua facial probe. Then the tubes inside the machine will be cleaned automatically.

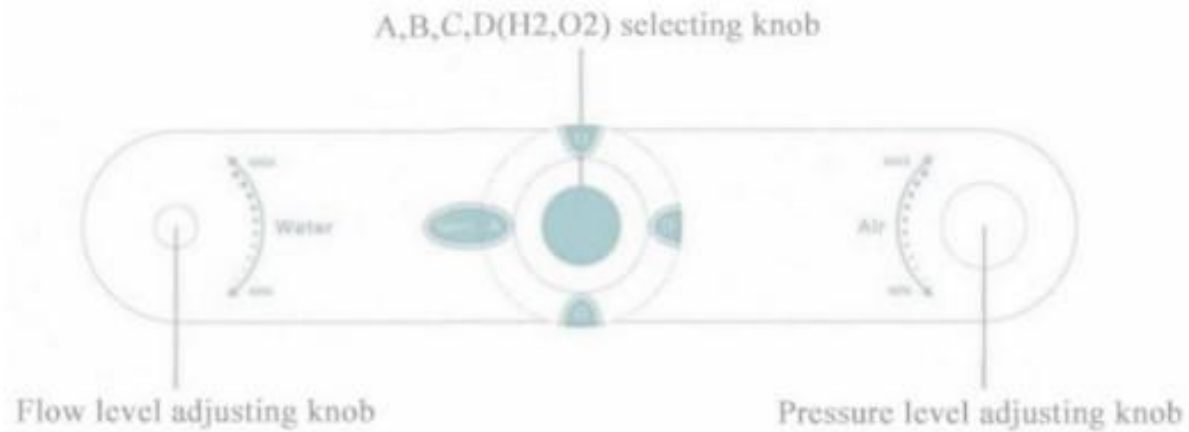

Caution:

When the liquids on the bottles are been using out, please input warm water to bottle A,B,C,D and cleaning the tubes by the methods for 4.1.4, to avoid blocking the tubes.

4.2 Using for Spray handle




4.2.1 Cleaning the treatment area before using the handle.

4.2.2 Input the dilution essence to the spray bottle.

4.2.3 Press the start button for the spray gun on the screen.

4.2.4 Press the button on the spray gun then the gun will start working and aim at the treatment area.

4.3 Using for Cold hammer

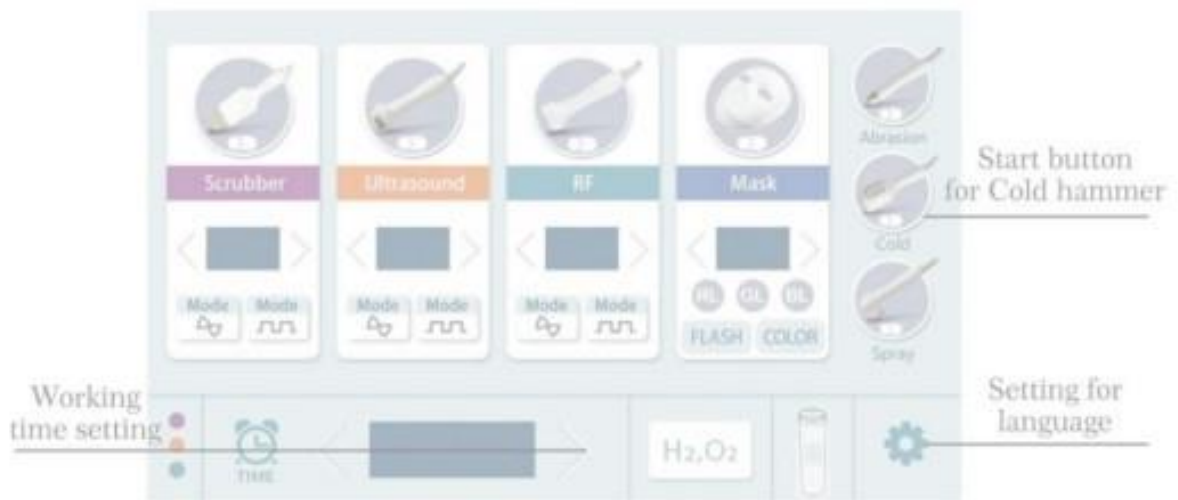

4.3.1 Cleaning the treatment area before using the cold hammer.


4.3.2 Press the start button on the screen.

4.3.3 Using the cold hammer on the treatment area.


Treatment on the face:

Start from ophryon and spread away, move upwards. As the photo shows

1. From lower jaw to ear
 2. From lower jaw to nose.
 3. From nose to Temple
 4. From the inner corner to the temple.
- do not get too close to the eyes and be tender.

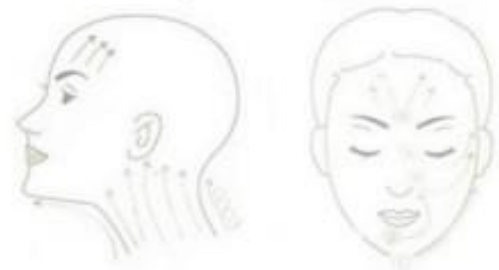

4.4 Using for RF

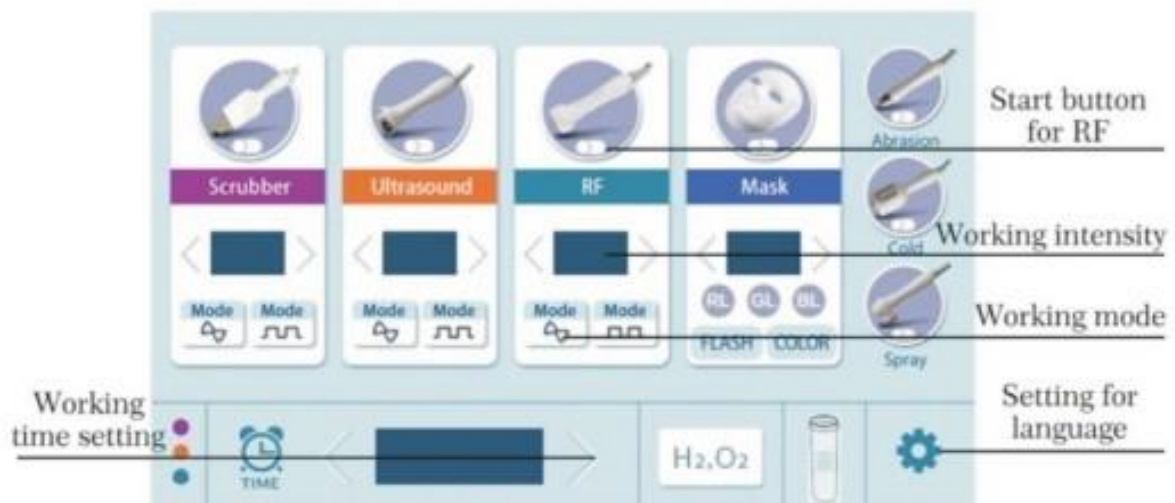


4.4.1 Cleaning the treatment area before using the instrument.

4.4.2 Apply suitable gel to the treatment area.

4.4.3 Adjusting the working time, RF intensity and RF mode before using. Using the RF handle stay close on the treatment area, and press the start button on the screen to start the RF function.

4.4.4 According to the facial line and neck, do circling from bottom to top to realize detox. And operation time for each part should be around 10-15 minutes.

4.5 Using for Ultrasonic

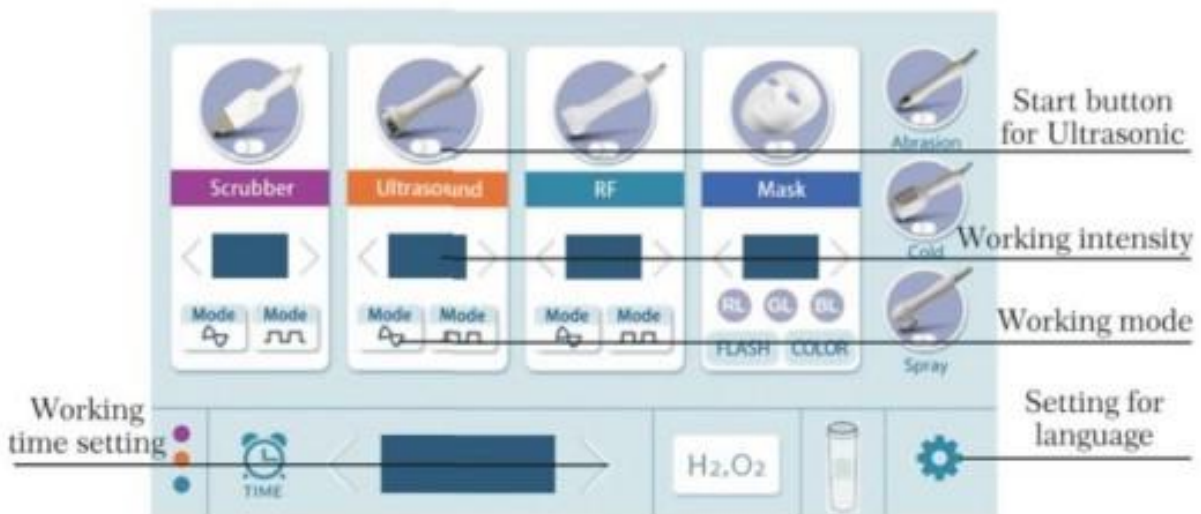

4.5.1 Cleaning the treatment area before using the treatment.


4.5.2 Apply suitable gel on the treatment area.

4.5.3 Adjusting the working time, ultrasonic intensity and ultrasonic mode before using.

4.5.4 Using the ultrasonic handle stay close on the treatment area, and press the start button on the screen to start the ultrasonic function.

4.5.5 According to the contour line and do circling for the treatment area at about 5-8 minutes.

4.6 Using for Skin scrubber

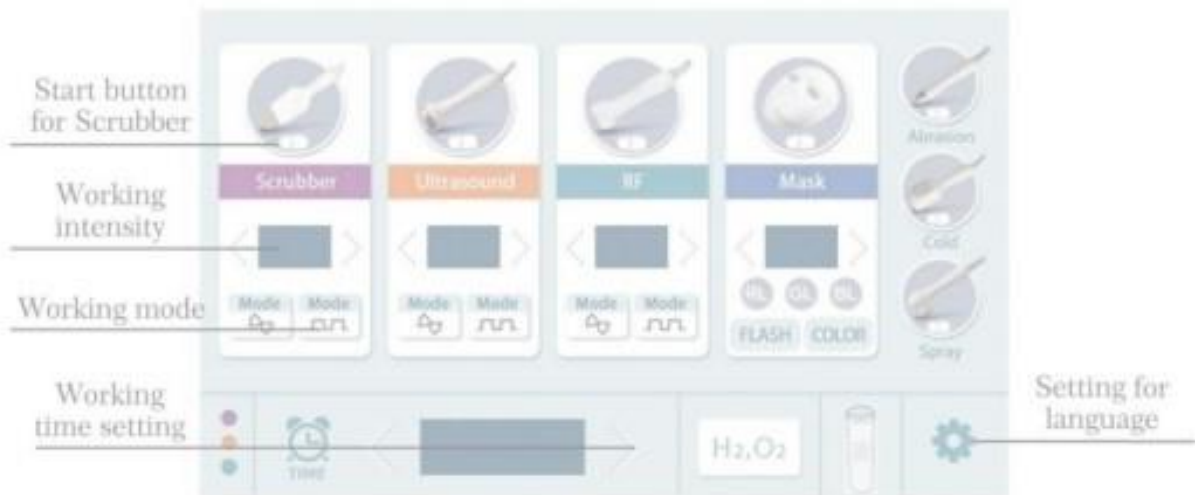

4.6.1 Cleaning the treatment area before using the treatment.


4.6.2 Adjusting the working time, intensity and mode before using.

4.6.3 Using the cleaning method to remove the dirt from the skin surface.

- 1) Using the facial cleanser to clean the skin.
- 2) Sterilizing the skin scrubber before using, and let the customer hold the sticker when using.
- 3) Applying enough water on the skin.
- 4) The blade is tilted at 45 degrees, make sure it is close to the skin, moving up and down at moderate speed.
- 5) The whole working time is 20 minutes.

4.6.4 Using the touching method to do lymphatic massage

- 1) Using the facial cleanser to clean the skin.
- 2) Sterilizing the skin scrubber before using, and let the customer hold the sticker when using.
- 3) Applying enough skin care products on the skin.
- 4) Make the skin scrubber head flating with the skin and moving it according to the texture of the muscles.
- 5) The whole working time is 15 minutes.

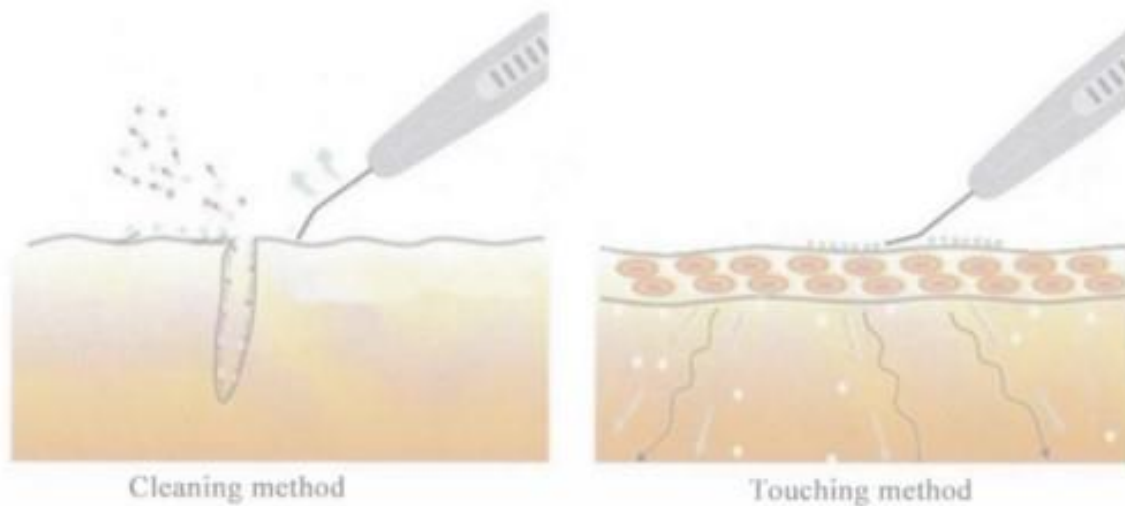

4.7 Using for Mask



4.7.1 Solves the whitening, skin aging problem fundamentally, Let increased activity of cells;promote cell metabolism, make the skin secretion of collagen and fibrous tissue to fill itself. Accelerate blood reflux, increase skin elasticity and improve skin complexion, dull conditions,so as to achieve whitening, skin rejuvenation, wrinkle, repair, etc. Blemish effect.Instruments were based on the original technology upgrade, using seven different colors cold light, namely red, blue, green, yellow, purple, blue, laser light, each light has a different effect, you can handle different skin problems, the effectiveness of a variety of instruments, practical and affordable.

4.7.2 Note

- 1) Don't watch the LED light source directly in long time.
- 2) If use disposable mask or cosmetics together with the LED mask, please choose high quality natural ingredients products.
- 3) The mask material is medical silicon, which will have some smell right after opening the package, but smell will disappear very soon.
- 4) There are LED light sources and PCB board inside of the silicon mask. NO fold the mask, NO squeeze, No heavy press.
- 5) Use the LED mask on bed, after you finish treatment put the led mask back on the box, no left it on bed.
- 6) Don't use water to clean the LED mask, no dip into water. Please use wet cloth to clean LED mask. The soften mask can be cleaned by water.
- 7) Don't use any chemical liquid to clean the LED mask, and the soften mask.
- 8) Don't expose to sun light directly.

5. Maintenance service

5.1 The instrument is under warranty for one year except the accessories and the consumables. We would provide maintain service after the warranty period, but payable.

5.2 If you want to replenish or purchase the consumables and accessories, please contact the distributor for advice.

5.3 Breakdown cause by human as below is out of our warranty.

- a) Breakdown cause by dismounting or transforming the instrument by customer himself/herself.
- b) Breakdown cause by incorrect or unreasonable maintenance.
- c) Breakdown cause by beating or dropping the instrument.
- d) Breakdown cause by incorrect operation.
- e) Damage cause by irresistible disaster like earthquake, fire disaster.

5.4 Welcome for any comment and suggestion.

5.5 When cleaning machines, please use dry soft towel to swipe the machine for the dust, and use wet soft towel to swipe the machine for the dirty.

6. FCC Information

CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment!

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This product may cause harmful interference.
- 2) This product must accept any interference received, including interference that may cause undesired operation.

WARNING: Changes or modifications to this product not expressly approved by the party responsible for compliance could void the user's authority to operate the product.

Note: This product has been tested and found to comply with the limits for a Class B digital device pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This product generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this product does cause harmful interference to radio or television reception, which can be determined by turning the product off and on, the user is encouraged to try to correct the interference by one or more of the following measures.

- Reorient or relocate the receiving antenna.
- Increase the distance between the product and receiver.
- Connect the product to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for assistance.

Sanven Technology Ltd.

Address: Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730.

Made In China