


THE LEADERS IN RECREATION, FITNESS AND ABILITY!

9889 Garrymore Ln Missoula, MT 59808 888-687-3552 | +1-406-549-0769 www.aquacreek.com

# Extended Adapter Base Plate

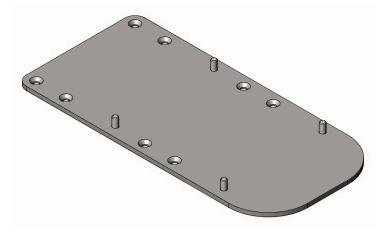

PART #: F-PPEBP

**MANDATORY** LEAVE THIS MANUAL WITH LIFT OWNER

Read and follow all instructions. Lift safety can only be ensured if the lift is installed and operated according to these instructions.

- · NEVER permit children to play on or around the lift
- Do not allow children to use the lift without adult supervision
- NEVER apply direct water pressure to the electronics
- NEVER use the lift with a dry pool

#### **READ CAREFULLY**


Check entire box and all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.

It is critical that all parts be carefully inspected prior to installation. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.

Proper installation cannot be overstressed, as an improper installation voids Aqua Creek's warranty and may affect the safety of the user.

## The Extended Adapter Assembly


The extended adapter base plate assembly is to be used for any Pro Pool-AT or Ranger-AT lift when the anchors must be set back further than the standard lift allows for. The diagram below gives an overview of the assembly, and shows how the lift and adapter plate are bolted to the deck.

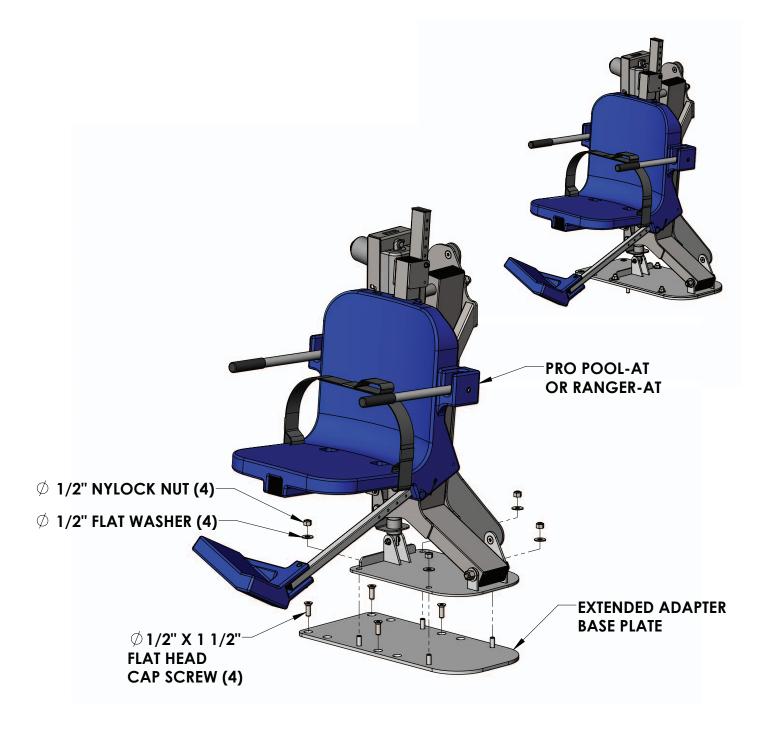

2

## Step 1: Bolt the Plate to the Deck

The Extended Adapter plate bolts directly to the concrete anchor inserts with four (4) 1/2" x 1 1/2" flat head cap screws. Installation will require a 5/16" Hex wrench or Allen Key.

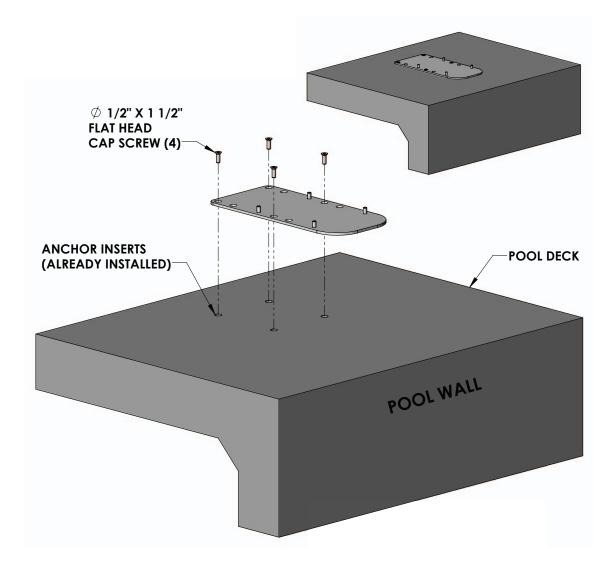


NOTE:

The Extended Adapter plate has two sets of holes that can be used to anchor the plate to the pool deck. The front set is for setback distances up to 23" from the pool wall and the second set is for distances up to 26" from the pool wall.

### **Step 2: Install The Lift**

With the Extended Adapter plate bolted to the pool deck, the lift is ready to be installed on the plate. The diagram below shows the process, use the four (4) welded studs on the plate. Install the lift onto the plate using;

3/4" WRENCH OR SOCKET ANTI-SEIZE LUBE

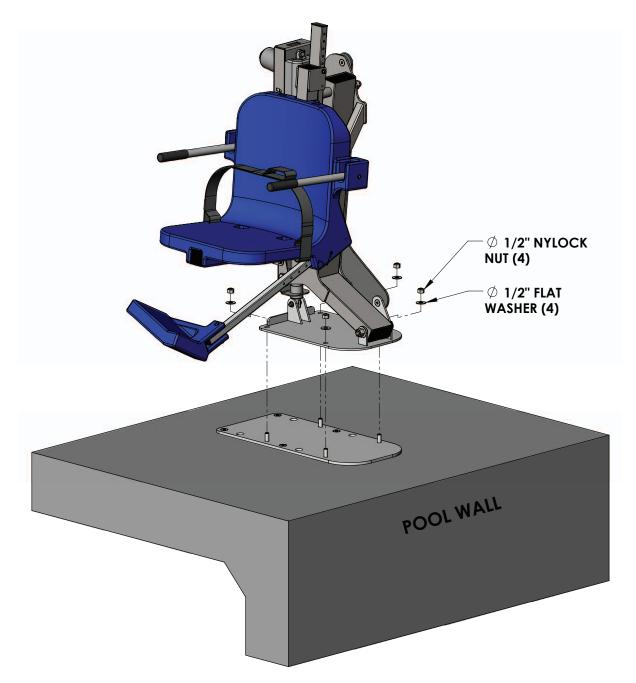

Apply the ANTI-SEIZE lube to the welded stud threads before installing. This will reduce corrosion and prevent the stainless steel from galling.

4