




# Installation User Manual

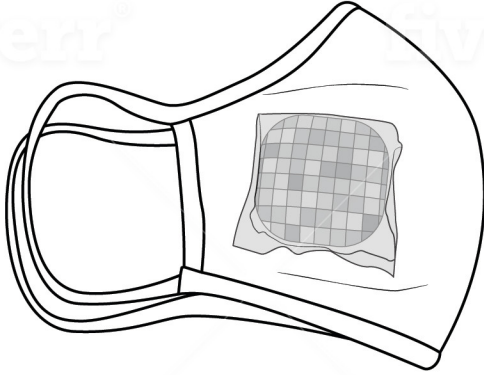


**Step 1:** Take the mask out of the package. Momz Halo “2.0 electric air valve” is attached to the mask.

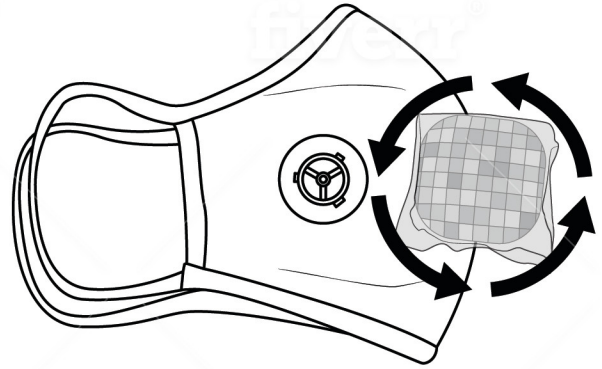


**Step 2:** Twist the “2.0 electric air valve” ‘counterclockwise’ which will detach from the air valve’s rim of the mask. Process shown on the photo.

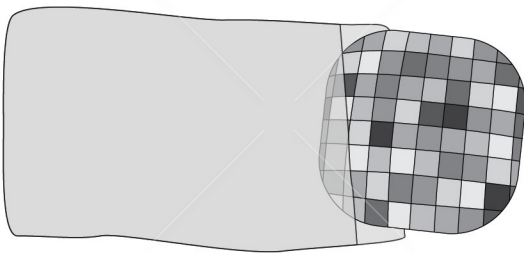


**Step 3:** Remove the protective cover from “2.0 electric air valve”.

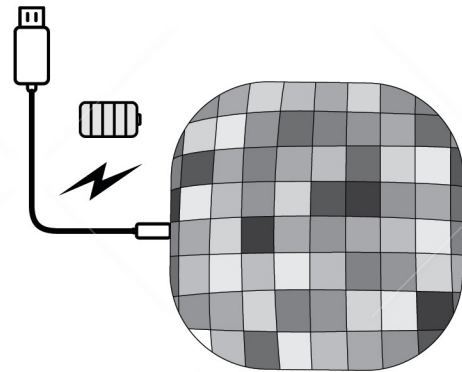

**Step 4:** We recommend to have the “2.0 electric air valve” fully charged to experience better performance.

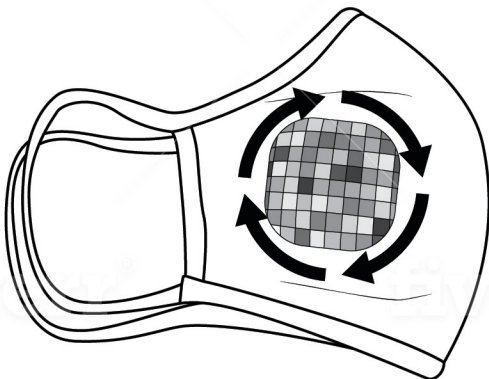


**Step 5:** Remove the protective cover from “2.0 electric air valve”.

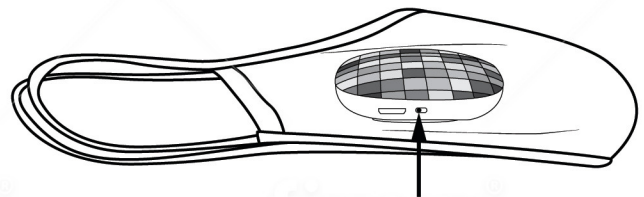


**Step 6:** Turn on the switch of the “2.0 electric air valve” as shown. Then wear the mask.


# Steps to charge

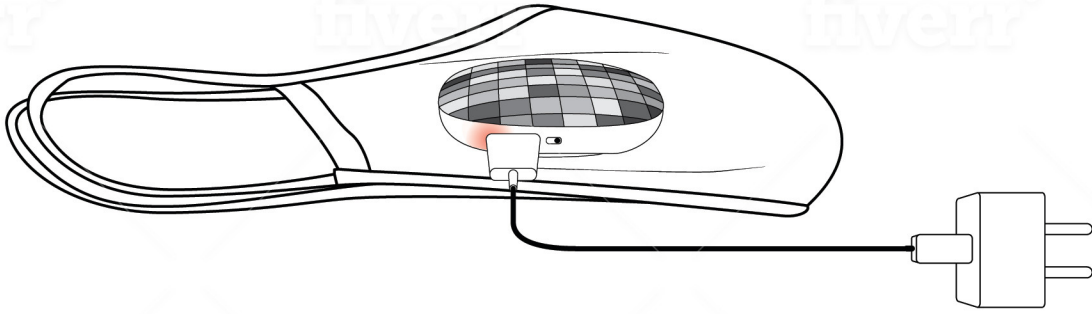

Plug the Micro USB cable in the micro-port OF THE “2.0 Electric air valve”. Plug the other end into a USB port on your computer or other USB charging device. Typical charge time on an empty battery takes 1 hour.

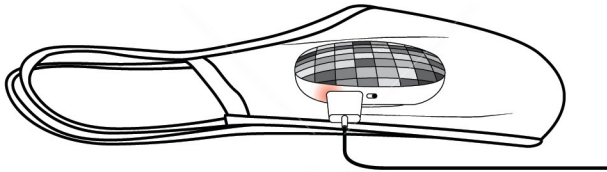

RED “2.0 electric air valve needs charging

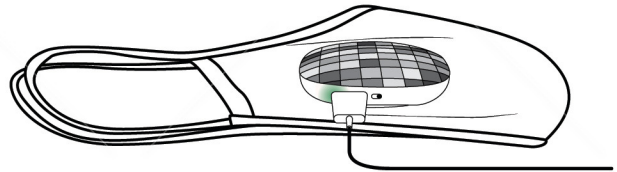

GREEN 2.0 electric air valve is fully charged

**Washing Instructions:** Please remove (twist counter-clockwise) the “2.0 electric air valve” before washing.

Wash the mask in warm water below 40 degrees Celsius. Hang to dry or tumble dry low. Do not Iron, do not Dry clean and Do not bleach.