

## Making A Perfect Teeth Impression

We would like to thank you for your trust in TruSmile Veneers Company and we assure you that making a dental impression is a very simple process that anyone can make. More than 90% of our customers get it right from the first attempt.

### Very important notice:

Please, don't attempt to take the impression if you suffer or have any of the following: Bleeding Gums, Gum Disease, loose teeth, loose fillings, loose crowns, badly decayed teeth and/or accumulated calculus or tartar over your teeth.

Please consult your dentist first.

Please carefully read all the instructions and watch our Impression making video available on TruSmile YouTube Channel before you proceed.

## STEP 1

Brush your teeth and rinse your mouth.

Have a mirror ready (Bathroom mirror is okay).



## STEP 2

Fit the different impression trays on your teeth, until you decide which size to use (Small or Medium) (5 or 6).

The tray should overlap all your teeth and surrounding gums.



# STEP 3

Get the impression material out of one of the containers with lable 1 & one of the containers with lable 2 and mix them quickly by kneading and folding for 30 seconds only until it becomes orange color.





## STEP 4

Roll the putty and make it as tall as your finger and then quickly load the tray evenly and fully with it.

It's very important to mix the material & load it in the tray in less than 50 seconds.



Try to swallow all the saliva in your mouth to make your mouth as dry as possible.

## STEP 5

Insert the tray inside your mouth. Make it fully inside your mouth before touching any teeth.



## STEP 6

Aim your front teeth to the middle part of the tray or slightly backwards. Make sure your front teeth are away from the front wall.

Look closely into the mirror to make sure your teeth are in the right place.

# Keep your teeth away from Front Wall Aim your teeth to the highlighted zone

## STEP 7

Push the tray all the way against your teeth in a straight line without any rocking or wiggling. Push so hard all the way until you can't push anymore



## STEP 8

Use your fingers to firmly massage around the borders of the tray to push the excess putty back inside the tray, only for 10 sec Please hold the tray firmly in place using your thumbs while massaging like shown in the picture.

## STEP 9

Keep the impression perfectly still using your thumbs without moving or rocking for 4 minutes, try not to speak or swallow to keep the tray still. Then take it out in a straight line.



## **STEP 10**

After making the first impression, and before making the rest of the impressions please, e-mail us a photo of it to guide you. Please make impression for top & bottom as both are needed.


## STEP 11

Fill the details of the Return Card and put it along with impression back in the box. Put the box in the return shipping bag and Leave it in your mail box or drop it in any Postal Service Office.



Home impression making is a very simple process. Perfect impression = Perfect Veneers, that's why sometimes we would send you another impression kit to make the impression again to get more details or a more accurate impression. Kindly we charge \$25 for an extra impression kit to cover a part of the shipping and expensive materials cost.

If you have enough materials, please make extra impressions for Top and Bottom to choose the best one for your veneers.

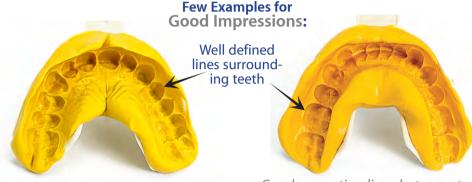

Well demarcated teeth borders

Good separation lines between teeth



Distorted impression and showing walls of the tray

# Examples of Rejected impressions:



Showing dragging lines because the material was too soft when it was pulled out from the mouth



Too shallow impression because the material was too hard before inserted in the mouth

In Most of the cases it's very Important to have photos of the customers mouth and teeth to make the most suitable and matching veneers. So, please take few minutes to take those few photos and upload them to your online account or send them to photos@trusmileveneers.com



## **Return Card**

Please, Don't forget to cut the return card, Fill it and Send it along with your impressions.

This return card is very important in order to recognize your impressions & don't mix them with other customers. No cases will be accepted without this return card.

Please Note that after making the impressions & manufacturing your veneers, this sale will be final & no returns or refunds will be accepted. If you aren't satisfied with your veneers, we will remake them once for you for free.

Making your custom made veneers require a lot of work, expensive materials and labor because it's custom made exclusively for your teeth. That's why we don't offer refunds or returns.

By filling this card and sending your impressions, you agree to all our terms and conditions mentioned above and the terms available on our website: www.trusmileveneers.com

Full Name :			
Full Address :			
Phone :			
Email :			