Nitrile examination gloves, powder-free

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NON-STERILE. FOR SINGLE USE ONLY. Aurelia's Bold nitrile examination gloves were tested for exposure to **fentanyl** in accordance with **ASTM D 6978-05 Standard Practice for Assessment of Gloves to Permeation by Chemotherapy Drugs**. The gloves were specifically tested for breakthrough time with **Fentanyl Citrate Injection**. Breakthrough time is the point at which the chemical reaches a predetermined threshold on the opposite side of the glove film. In testing for **fentanyl**, Bold tested to **No Breakthrough greater than 240 minutes**.

INSTRUCTION FOR USE AND STORAGE

Before putting on gloves hands must be clean and dry. Gloves do not replace the need for hand hygiene. Hands must be washed every time gloves are removed to avoid contamination. Gloves are for single use. Change gloves if moving from a contaminated area to a non-contaminated area. Be sure to wear gloves that fit properly, gloves should fit snugly around wrists and hands. Different sizes are available.

Store in a cool, dry place. Avoid excessive heat (40°C, 104°F). Shield open boxes from direct sun and fluorescent lighting. To order, please refer to a specific size and mention the product code.

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