QUICK START GUIDE

Quick Ready Mounting Plate for Assistive Technology

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN December 10, 2021 11:18 AM

Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Attach threaded bolt on Quick Ready Mounting Plate to mounting arm.
2. Tighten flat side of locking nut against mounting arm to secure Quick Ready Mounting Plate.
3. Apply Dual Lock to mounting plate surface and to the device you are mounting. Press down firmly on Dual Lock to ensure proper adhesion.
4. Attach device to Quick Ready Mounting Plate.

To request a different mounting arm, clamp, or mounting plate, contact ableCARE at ablecare@ablenetinc.com or (651) 294-3101.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.


3

Device Overview

Mounting Plate Surface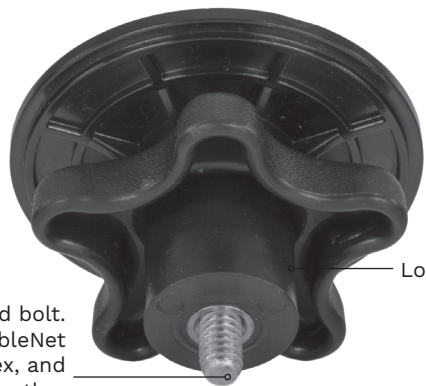

Locking Nut

1/4"-20 male threaded bolt.
Compatible with AbleNet Latitude 360, One, Flex, and Cling mounting arms, or other mounting arms with a 1/4"-20 female threaded receiver.

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