QUICK START GUIDE

All-Turn-It® Spinner

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN June 18, 2021 1:51 PM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Insert two alkaline C batteries into the battery compartment.
2. Remove the clear plastic disc and arrow.
3. Place the small overlay over the plastic spinner wheel.
4. Replace the arrow and clear plastic disc.
5. Place the large overlay on the device, aligning the three tabs into their corresponding slots.
6. Your devices is now ready to use. Spin the arrow and small overlay by activating the built-in switch or an external switch plugged into the switch jack on the side of the device.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.


ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

3 Device Overview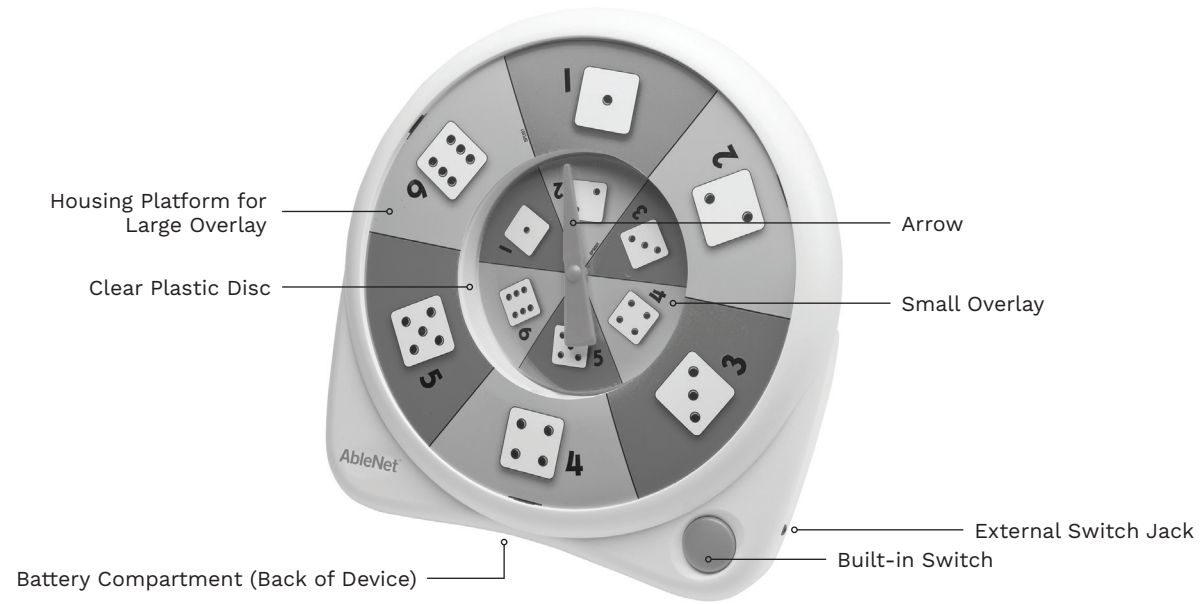

Create printable picture symbol overlays for your device with the FREE AbleNet Symbol Overlay Maker app. Scan the QR code to download today!

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