**ablenet**<sup>®</sup>

## Quick Start Guide

### Hitch<sup>®</sup> 2.0


#### Computer Switch Interface

AbleNet, Inc.  
2625 Patton Road  
Roseville, MN 55113  
www.ablenetinc.com

*This device has been tested to comply with  
FCC standards for home or office use.*

PN 98000053 (06/16 Rev. A - FN 70500247)

## Product Overview

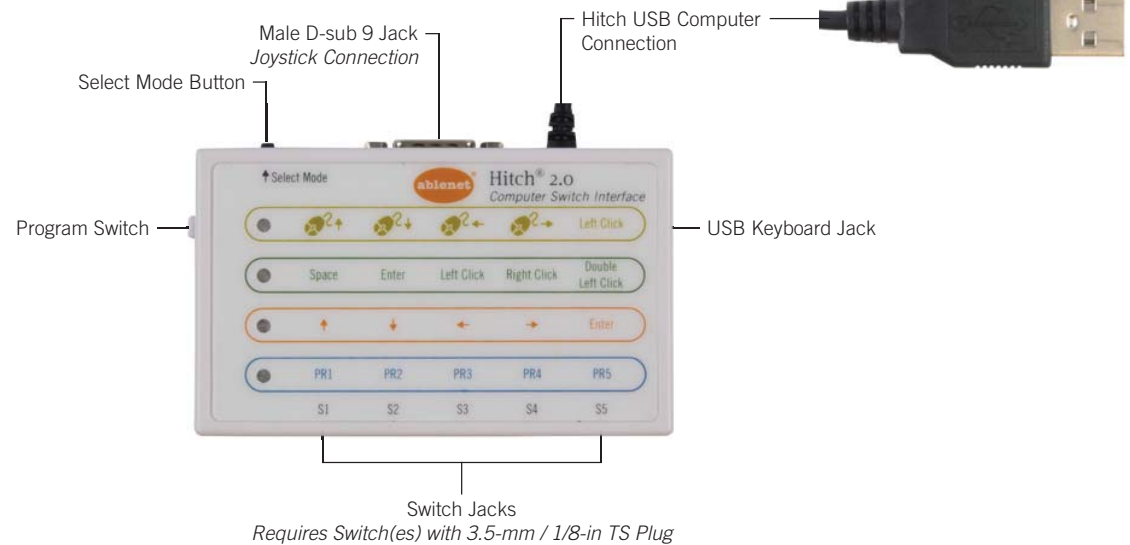## Getting Started

Hitch 2.0 requires a computer with a USB jack and is compatible with most Windows, Mac OS X, Google Chrome, or Android operating systems.

1. Connect Hitch 2.0 to USB port on computer. One LED light on Hitch 2.0 will turn on when connected.
2. Activate Select Mode button to select desired row of functions
3. Attach switch(es) with 3.5-mm / 1/8-in TS plug or joystick with female D-sub 9 connector to Hitch 2.0
4. Activate switch(es) or joystick to send function to computer

## Warranty

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return / repair information is available at [www.ablenetinc.com](http://www.ablenetinc.com).

## Custom Program (Optional)

The functions PR1 to PR5 found in the fourth / blue mode on Hitch 2.0 can be programmed to send one to four keyboard keystrokes of your choice to a computer. Complete the following steps to program Hitch 2.0.

1. Connect Hitch 2.0 to USB port on computer. One LED light on Hitch 2.0 will turn on when connected.
2. Activate the Select Mode button and select the fourth / blue mode with the functions PR1 to PR5
3. Attach switch(es) with 3.5-mm / 1/8-in TS plugs into Hitch 2.0
4. Move the Program switch on Hitch 2.0 to On
5. Plug a USB keyboard into Hitch 2.0
  - a. Note: If using a Mac OS X computer, please make sure you first plug the USB keyboard directly in the Mac OS X computer and go through the Keyboard Setup Assistant so that the computer can identify the keyboard. Once the Keyboard Setup Assistant is complete, unplug the USB keyboard from the Mac OS X computer and plug it directly into Hitch 2.0. Failure to do so may result in the keyboard not working with Hitch 2.0.
6. Select the switch plugged into Hitch 2.0 that you would like to program and hold it down until the blue LED light begins to blink on and off
7. While the blue LED light is blinking on and off, select one to four keystrokes on the USB keyboard
  - a. If you select one, two, or three keystrokes, activate the same switch a second time to exit recording mode
  - b. If you select four keystrokes, Hitch 2.0 will automatically exit recording mode
8. Repeat steps 6 and 7 if you would like to program additional switches connected to Hitch 2.0.
9. After the programming of each switch is complete, move the Program switch on Hitch 2.0 to Off and disconnect the USB keyboard from Hitch 2.0
10. Hitch 2.0 is now ready to use your custom keystrokes with each switch

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