


Quickstart Guide

Responsible Switch-Adapted Emergency Wall Communicator (EWC)

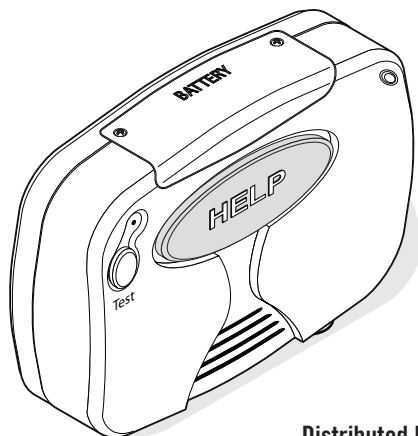


Distributed by AbleNet




www.ablenetinc.com

Installing Batteries

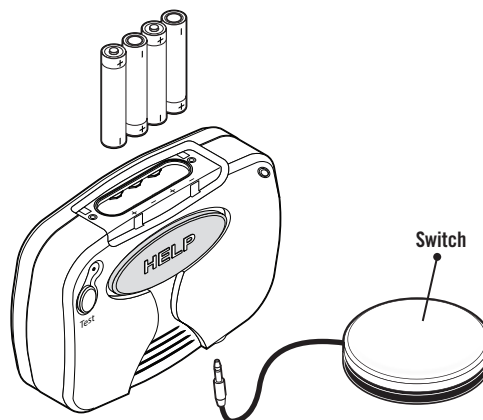

1. Unscrew battery cover from EWC and insert batteries as shown.
2. Replace battery cover.
3. Plug any AbleNet switch into jack on front of device
 - This switch can now be used for the same functions as the Red “Help” button.


Pairing EWC with Base Unit

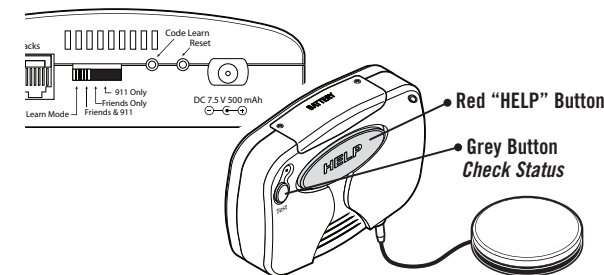

1. Press and Release Red “Code Learn” button on back of Responsible Base Unit.
 - Device will speak, “Pendant Learning”
2. Press and Hold Gray then Red buttons on EWC until you hear, “Pendant Learning.” Release buttons.
3. When successfully paired, device will speak, “All systems are OK” then announce current dialing mode.


Notes: If Code Learning fails, wait 30 seconds, then retry by starting the process over.

Basic Use

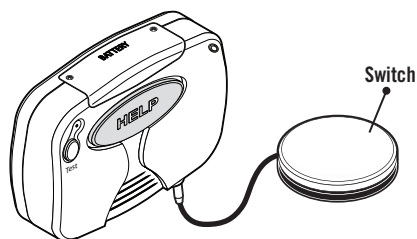

1. To call out, Press and Hold the switch for 2 seconds.
 - Responsible will dial the emergency number(s) based on the calling mode selected. (See Responsible System Manual for modes.)
2. To hang up, Press and Release the Gray button one time.
 - The other caller can also end the call by pressing 9 on their keypad (preferred) or by simply hanging up.
3. To answer an incoming call, Press and Release the switch one time.

Battery and Systems Check

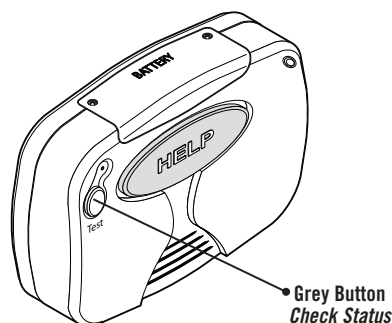

To test battery status - Press and Release the device's Gray button. The device will speak its current battery condition.

To test system status - Press and Hold the device's Gray button for 4 seconds. The device will speak the current system conditions.

Note: See Responsible System Manual for status details.

Mounting, Warranty and Contact

Mounting the EWC

EWC can be mounted using screws or hook and loop fastener included.

- Position screws exactly 3 1/8-inches apart.
- Velcro® should be adhered to a clean and dry surface.
- Use AbleNet's Small Rectangular Mounting Plate and mounting systems for wheelchair or bed rail mounting.

Warranty

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text is available at www.ablenetinc.com.

AbleNet, Inc.

Mpls./Saint Paul, MN 55113
800-322-0956
www.ablenetinc.com



www.ablenetinc.com