

AL-Gard

Deluxe Heavy Duty Back Support Belt

- S 27"-32"
- M 32-37"
- L 35"-40"
- XL 40"-45"
- XXL 45"-52"

- Adjustable shoulder strap

- Rubberized stays keep belt in place

- 9-inch elastic band with 5 non-conductive stays for firmer support



- Fully detachable comfortable lumbar pad for extra support and stability

- 2-stage hook and loop closure for fully customisable fit

Purchase online at www.safetysolutions.com.sg



Email: sales@quantumsafety.com.sg
Phone: +65 6352 2422 www.quantumsafety.com.sg
Quantum Safety Network Pte Ltd
998 Toa Payoh North #03-14/15 Singapore 318993


Deluxe Heavy Duty Back Support Belt




STEP 1: Put both arms through suspenders. Belt should be at waist level. Close the right side over the left.



STEP 2: To tighten belt, grasp outer elastic bands and pull forward.

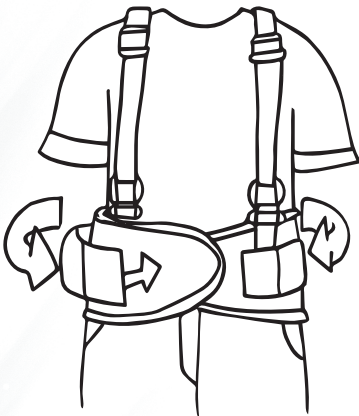

STEP 3: To fasten, attach outer elastic bands to the front of the belt.

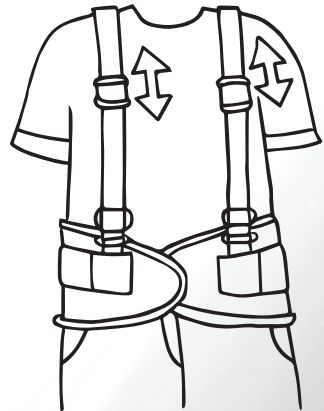

STEP 4: Adjust suspenders by sliding buckles up or down to get a comfortable and secure fit.

Warning: Only use this back support belt for intended use. Use of this back support belt will not prevent injury. Always use proper and safe techniques. Do not exercise or lift heavy items without proper instruction and supervision.